


AUXILOCK[®] AC JOINT KIT

AC Joint Repair

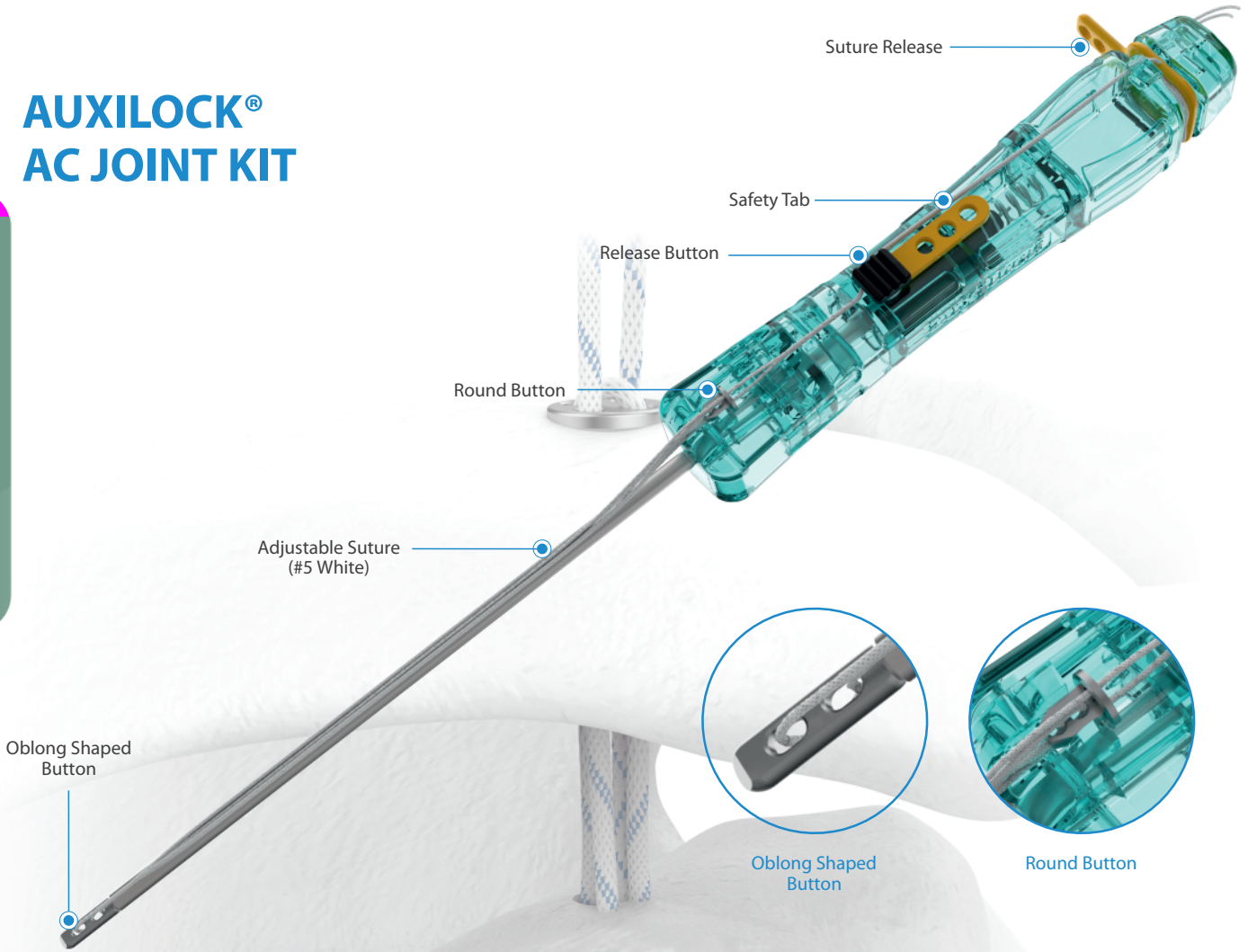

AUXILOCK[®] AC Joint Kit can be used for Acromioclavicular Joint (AC Joint) separation repair. The kit comprises of adjustable loop with round shaped button (Dia. 5.5mm) and oblong shaped button (L 12mm X W 3.2mm X H 1.5mm). A tunnel is made through the clavicle and coracoid using 1.5mm K-Wire and 3.5mm Cannulated Drill Bit. The kit is passed through 3.5mm drilling tunnel from clavicle to coracoid provides the fixation of round button on the clavicle and oblong shaped button on the coracoid. The AC Joint Kit provides a double locking mechanism which eliminates the need for knot tying and also provides very strong compression on the clavicles.

Ordering Information

Code	Product Description
6-041-02	AUXILOCK [®] AC Joint Kit

Related Instrumentation (Include in Kit)

Code	Product Description
7-147-01	Cannulated Drill Bit with Quick Coupling End, Ø3.5mm x Length 175mm
7-147-02	Ø1.5mm K-Wire

STERILEEO