


auxilock[®]


powered by  **Auxein**[®]

Surgical Technique


Single Row Rotator Cuff Repair using
PEEK Screw-in Suture Anchor

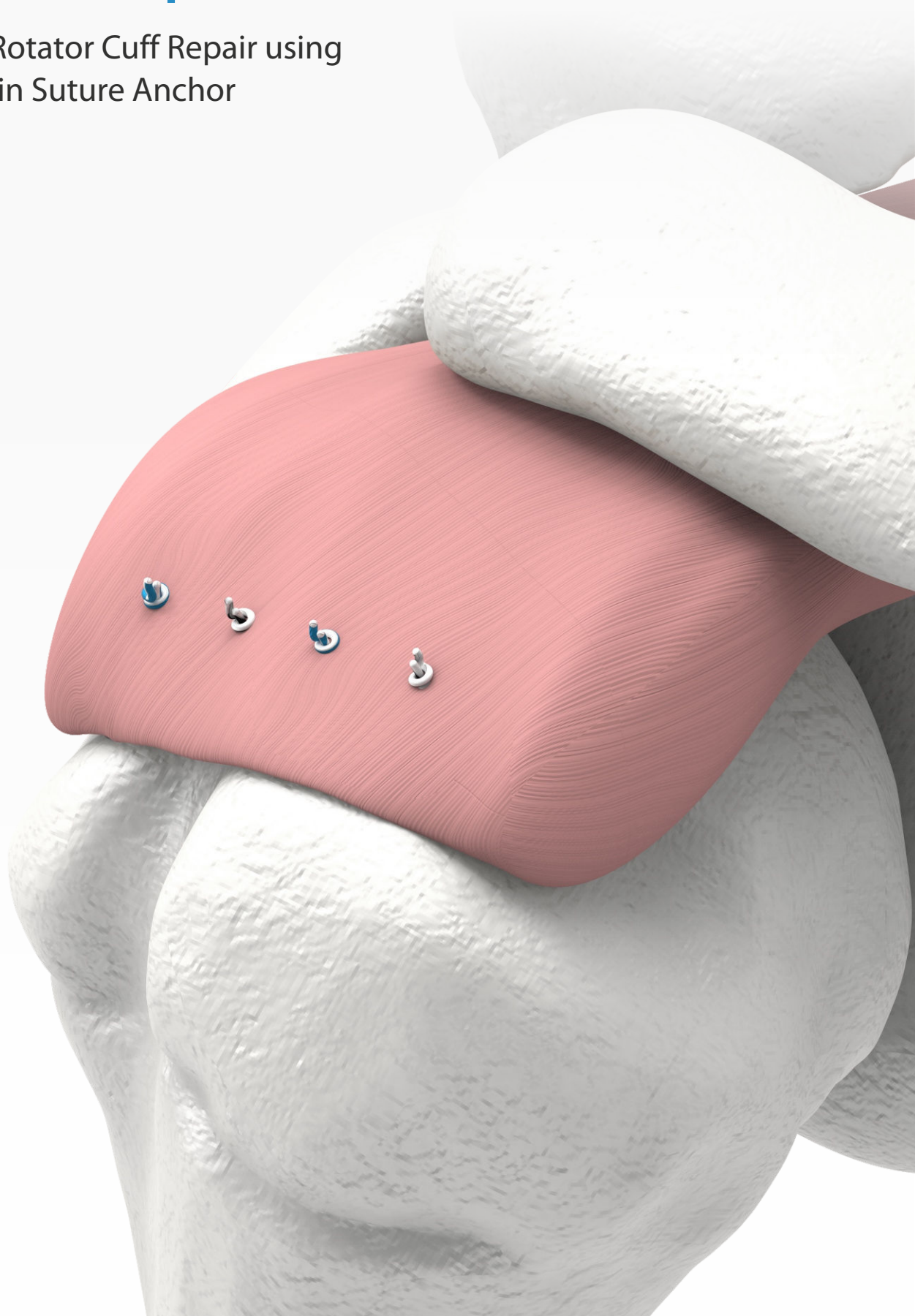

PRODUCT OVERVIEW

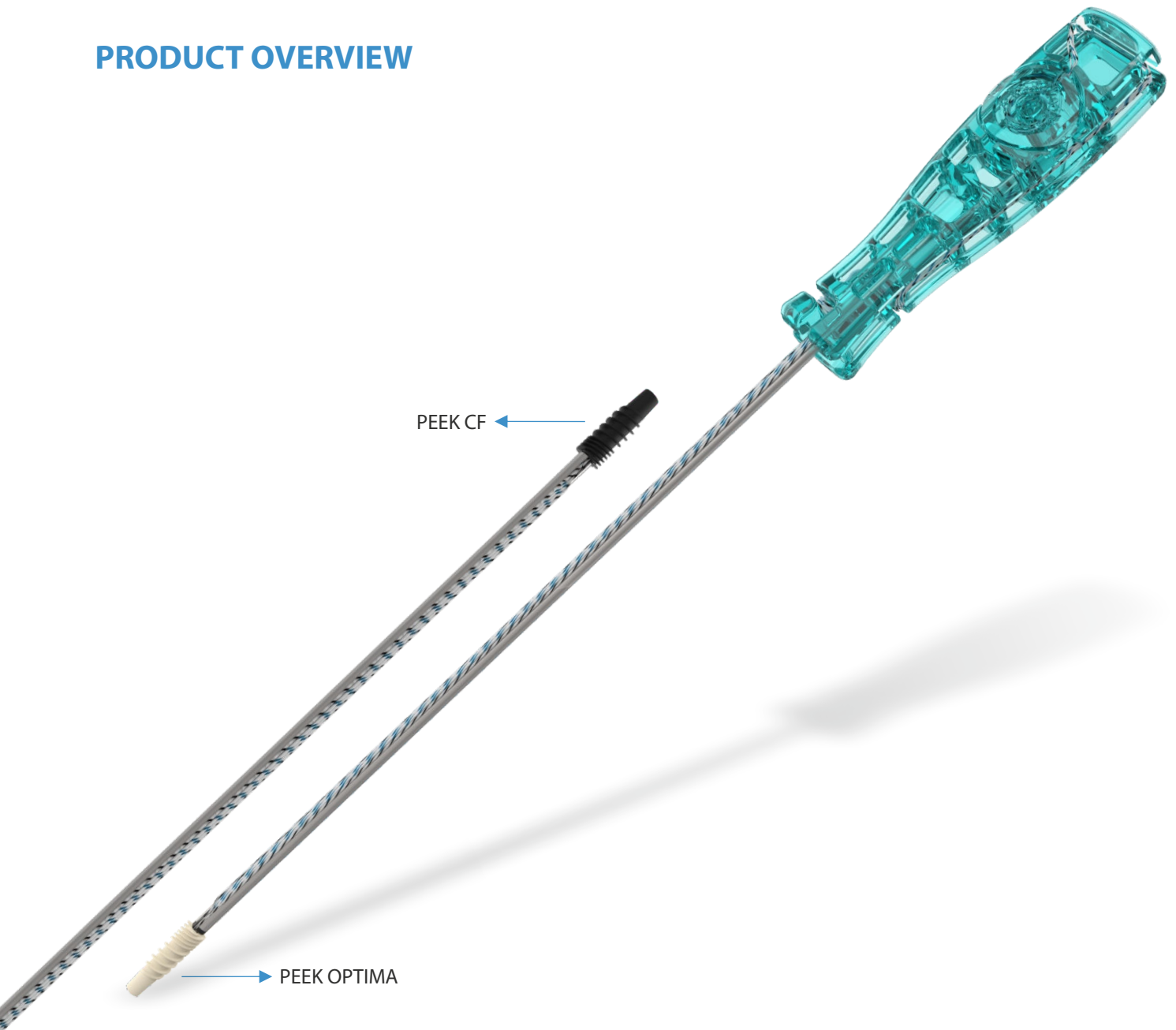

AUXILOCK® PEEK Screw-In Suture anchor is a fully threaded suture anchor featuring dual threads to maximize cortical and cancellous fixation. The anchor is available in PEEK OPTIMA and PEEK CF material. It is available in diameter of 4.5, 5.5 and 6.5mm with double and triple loaded options. It is also available with various BioBraid and suture tape configurations. It has a flat tip to protect the sutures and to facilitate the insertion. The anchor is particularly suitable for repairing rotator cuff and associated pathologies. The anchors are also available with needles which are ideal for mini-open procedures.

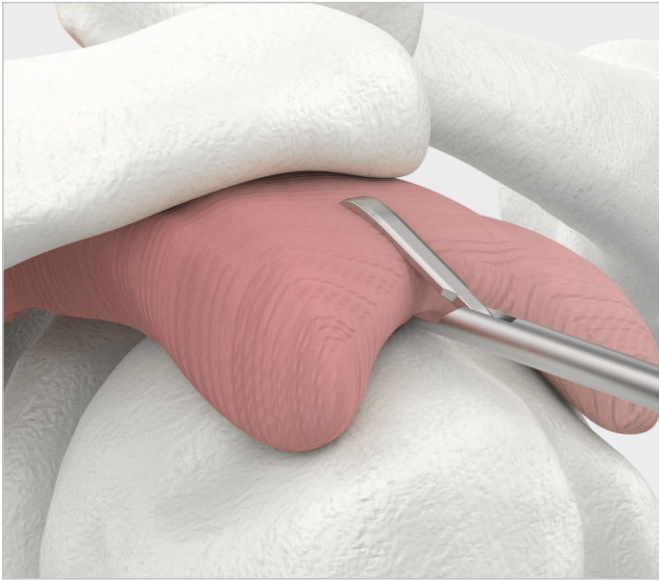


The rotator cuff footprint is prepared using the Rasp or Shaver, taking care not to completely decorticate bone.

With tissue grasper, reduce cuff to lateral aspect of tuberosity to determine medial-to-lateral mobility of defect.



Place Awl for 4.5/5.5mm PEEK Screw-In Anchor at a 45° on desired anchor location in the superior humerus just lateral to the humeral cartilage.

(Note: For semi hard or hard bone, use Awl/Tap or Tap to make a hole for the anchor).

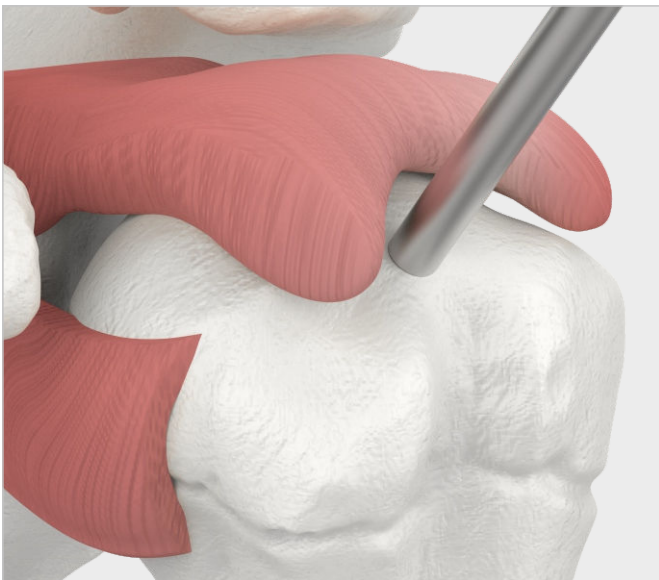


Advance the Awl to appropriate depth and create a hole for anchor.




Remove awl and insert PEEK Screw-in Suture anchor into hole and screw-in the anchor in clockwise direction. Continue to advance anchor until the back of the anchor is 1-3mm below the surface of the bone as indicated by the distal end of the laser mark on the driver shaft.



Once the anchor is fully seated into the bone, pull the round cap on the side of the handle. Then remove driver by twisting or pulling it. Confirm anchor is fully seated by direct visualization.

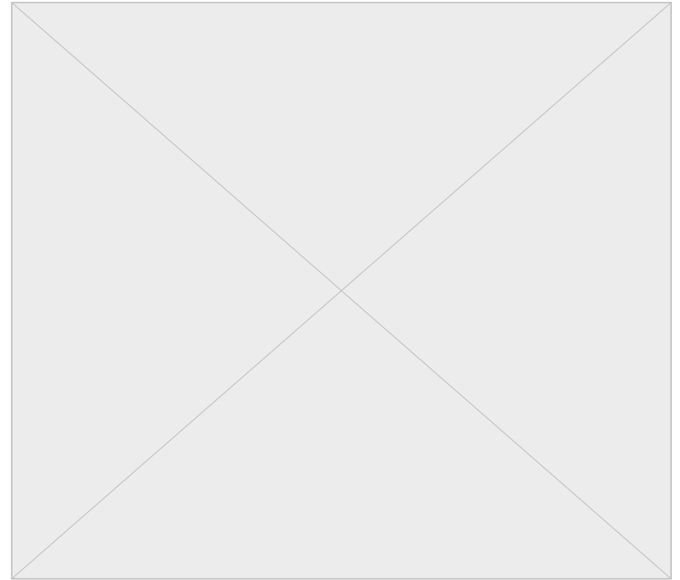

Repeat previous steps and insert the second PEEK Screw in Suture anchor.

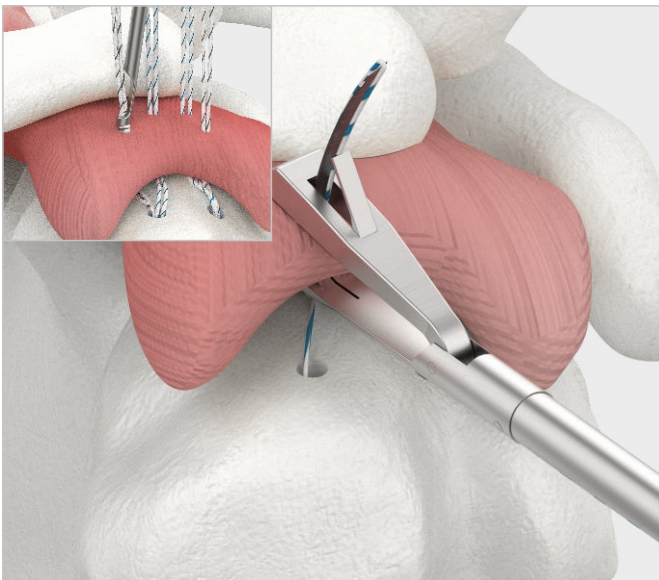

Beginning anteriorly, pass sutures through torn cuff tissue using Escorpion suture passer or surgeon's preferred method.


















Beginning Posteriorly, secure torn tissue using standard arthroscopic knot tying techniques.










Cut off the excess suture using the suture cutter.











ORDERING INFORMATION

PEEK OPTIMA Screw-In Suture Anchor

Code	Product Description	Suture/Tape Colour	Needle
6-016-05	AUXILOCK® 4.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black		
6-016-07	AUXILOCK® 5.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black		
6-016-08	AUXILOCK® 6.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black		
6-016-13	AUXILOCK® 4.5mm PEEK OPTIMA Screw-In Suture Anchor With Three #2 BioBraid: White, White/Blue & White/Black		
6-016-14	AUXILOCK® 5.5mm PEEK OPTIMA Screw-In Suture Anchor With Three #2 BioBraid: White, White/Blue & White/Black		
6-016-15	AUXILOCK® 4.5mm PEEK OPTIMA Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Blue & White/Black		
6-016-16	AUXILOCK® 5.5mm PEEK OPTIMA Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Blue & White/Black		
6-016-17	AUXILOCK® 6.5mm PEEK OPTIMA Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Blue & White/Black		
6-016-18	AUXILOCK® 4.5mm PEEK OPTIMA Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-016-19	AUXILOCK® 5.5mm PEEK OPTIMA Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-016-20	AUXILOCK® 6.5mm PEEK OPTIMA Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-016-01	AUXILOCK® 4.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		
6-016-03	AUXILOCK® 5.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		
6-016-04	AUXILOCK® 6.5mm PEEK OPTIMA Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		

PEEK CF Screw-In Suture Anchor

Code	Product Description	Suture/Tape Colour
6-046-01	AUXILOCK® 4.5mm PEEK CF Screw-In Suture Anchor With Three #2 BioBraid: White, White/Blue & White/Black	
6-046-02	AUXILOCK® 4.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black	
6-046-03	AUXILOCK® 5.5mm PEEK CF Screw-In Suture Anchor With Three #2 BioBraid: White, White/Blue & White/Black	
6-046-04	AUXILOCK® 5.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black	
6-046-05	AUXILOCK® 6.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White/Blue & White/Black	
6-046-06	AUXILOCK® 4.5mm PEEK CF Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Black & White/Blue	
6-046-07	AUXILOCK® 5.5mm PEEK CF Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Black & White/Blue	

Code	Product Description	Suture/Tape Colour	Needle
6-046-08	AUXILOCK® 6.5mm PEEK CF Screw-In Suture Anchor With One #2 BioBraid and One 1.4mm Suture Tape: White/Black & White/Blue		
6-046-09	AUXILOCK® 4.5mm PEEK CF Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-046-10	AUXILOCK® 5.5mm PEEK CF Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-046-11	AUXILOCK® 6.5mm PEEK CF Screw-In Suture Anchor With Two 1.4mm Suture Tape: White/Blue & White/Black		
6-016-21	AUXILOCK® 4.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		
6-016-22	AUXILOCK® 5.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		
6-016-23	AUXILOCK® 6.5mm PEEK CF Screw-In Suture Anchor With Two #2 BioBraid: White & White/Blue, With Needles: MO-6		

RECOMMENDED INSTRUMENTATION

Code	Product Description	Bone Density	Compatible Implants
7-013-25	Awl For 4.5/5.5mm PEEK Screw-In Anchor	Soft Bone	6-046-01, 6-046-02, 6-046-03, 6-046-04, 6-046-06, 6-046-07, 6-046-09, 6-046-10, 6-046-21, 6-046-22
7-013-26	Awl For 6.5mm PEEK Screw-In Anchor		6-046-05, 6-046-08, 6-046-11, 6-046-23
7-013-27	Awl/ Tap For 4.5/5.5mm PEEK Screw-In Anchor	Normal Bone	6-046-01, 6-046-02, 6-046-03, 6-046-04, 6-046-06, 6-046-07, 6-046-09, 6-046-10, 6-046-21, 6-046-22
7-013-28	Awl/ Tap For 6.5mm PEEK Screw-In Anchor		6-046-05, 6-046-08, 6-046-11, 6-046-23
7-013-29	Tap For 4.5/5.5mm PEEK Screw-In Anchor	Hard Bone	6-046-01, 6-046-02, 6-046-03, 6-046-04, 6-046-06, 6-046-07, 6-046-09, 6-046-10, 6-046-21, 6-046-22
7-013-30	Tap For 6.5mm PEEK Screw-In Anchor		6-046-05, 6-046-08, 6-046-11, 6-046-23

USA**Auxein Inc.**

1500 Nw 89th Court, Suite 107-108
Doral, Florida 33172
Tel: +1 305 395 6062
E Fax: +1 305 395 6262
Email: USoffice@auxein.com

MEXICO**Auxein México, S.A. de C.V.**

Tepic 139 int 801, Colonia Roma Sur,
Alcaldía Cuauhtémoc, CDMX,
México, C.P. 06760
Tel: +521 55 7261 0318
Email: info@auxein.mx

INDIA**Auxein Medical Pvt. Ltd.**

Plot No. 168-169-170, Phase-4,
Kundli Industrial Area,
HSIIDC, Sector-57, Sonapat - 131028, Haryana
Tel: +91 99106 43638 | Fax: +91 86077 70197
Email: info@auxein.com