Auxein

2.4mm AV-WISELOCK Distal Radius Fracture System

G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER		STRONG	GROWIN	STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRON
G (GROWING	STRONGER	GOING	STRONG	GROWIN		STRONGER	GOING	STRONG	GROWIN	STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STROP
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRON
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STROI
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRO
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	S STRO
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO1
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO1
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO1
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (BROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (GROWING	STRONGER		STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G	GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
G	GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G	GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G	GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRON
	ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRON
	ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRON
	ROWING	STRONGER	GOING	STRONG	GROWIN	IG §	STRONGER	GOING	STRONG	GROWIN	G STRON
G (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
G	BROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
3 (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	3 STRO
3 (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRO
3 (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRO
3 (GROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STROM
3 (GROWING	STRONGER		STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	G STRON
3 (ROWING	STRONGER	GOING	STRONG	GROWIN	IG S	STRONGER	GOING	STRONG	GROWIN	STRON
1	DOWING	CTRONCER	COLNIC	CTDONO	CDOWIN	10	STRONGER	COLNIC	CTDONO	CDOWING	CTDO

NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG		STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
								GOING STRONG	
								GOING STRONG	
NG	STRONG	GROWING	STRONGER	GOING	STRONG	GROWING	STRONGER	GOING STRONG	GROWI
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
								GOING STRONG	
JG	STRONG	GPOWING	STRONGER	GOING	STRONG	GROWING	STRONGER	COING STRONG	CPOWI





2.4mm AV-WISELOCK DISTAL RADIUS SYSTEM LOW PORFILE PLATES

The content of this catalogue is intended for healthcare professionals only. All product information, illustrations, descriptions, and specifications provided here are for general informational purposes and are subject to change without notice. Auxein Medical Pvt. Ltd. reserves the right to improve or discontinue products at any time without prior notification.

This document does not constitute medical advice. Surgeons and medical practitioners must exercise their own professional judgment when using any medical devices or instruments, including those provided by Auxein.

INTENDED USE

The implants and instruments in this catalogue are intended for use by qualified orthopedic surgeons trained in trauma surgery and fracture-fixation procedures. Improper selection, misuse, or lack of experience may result in complications.

REGULATORY COMPLIANCE

US FDA 510(k)

CE Mark (Class IIb)

MDSAP (Medical Device Single Audit Program)
ISO 13485:2016 Quality Management System
CDSCO – Indian Medical Device Rules, 2017

EU MDR- European Union Medical Device Regulation, 2017

INTELLECTUAL PROPERTY

All trademarks, trade names, brand names, product names, and logos mentioned in this catalogue are the property of Auxein Medical Pvt. Ltd. Unauthorised use or reproduction is strictly prohibited.

Auxein

ABOUT US

Auxein Medical is an integrated, research based, orthopaedic implants & instruments manufacturing company, producing a wide range of quality, affordable generic implants, trusted by healthcare professionals and patients across geographies. It is the Company's constant endeavour to provide a wide basket of generic and our innovator products that exceed the highest expectations of customers in term of quality and safety. The company has world-class manufacturing unit established in india and serves customers in over 75 countries worldwide.



At Auxein, it is our goal to provide the best in class quality and value to every individual. Consistency is a virtue we pursue with a passion. From the procurement of raw materials to the manufacturing of our range of 1200+ orthopaedic implant products, we employ world-class systems and technology at par with the highest standards of globally recognized evaluation systems. The AV-Wiselock System is part of our advanced portfolio tailored to meet the diagnostic and surgical demands of trauma procedures. Designed with input from global experts, this system ensures optimal fixation, stability, and anatomical alignment for complex fractures & deformity corrections.

OUR ACHIEVEMENTS



















New generation low profile Distal Radius & Ulna Plating System for fracture specific treatment.

Indications

- · Distal Radius fractures & osteotomies.
- Intra-articular and extra-articular fractures of the distal radius.
- · Correction osteotomies of the distal radius.
- · Distal Ulna fractures & osteotomies.
- · Intra-articular and extra-articular fractures of the distal ulna.

Plate Features

- · Low-Profile plates with better anatomical Fit.
- AV-Wiselock Variable angle of ± 15° in all directions in each screw hole.
- Pre-angled Variable Angle holes for oriented screw placement.
- Rounded edges and a smooth surface for soft tissue protection.
- Oblong hole for fracture reduction and length restoration.
- Universal Variable angle drill guides available for easy angulation of screws.
- K-wire holes to assist with temporary plate fixation



Anatomical Contour

First distal row can be bent individually to match the anatomy.



Variable angle locking

Holes allow up to ±15° screw angulation in all directions.



Plates Available in



2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved




	\wedge
	NON STERILE
- 4	STERILLE

STERILE

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
5	Left	11-058-01TIL	11-058-01SSL
5	Right	11-058-01TIR	11-058-01SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
5	Left	11-058-01TIL-S	11-058-01SSL-S
5	Right	11-058-01TIR-S	11-058-01SSR-S

2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate

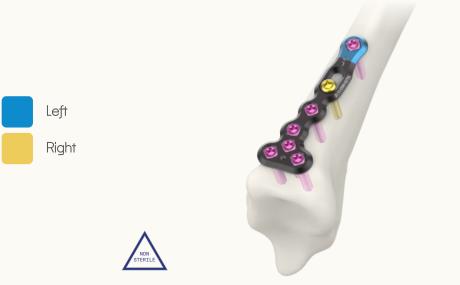

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
7	Left	11-058-02TIL	11-058-02SSL
7	Right	11-058-02TIR	11-058-02SSR

OLES	DIRECTION	TITANIUM	STAINLESS STEEL
7	Left	11-058-02TIL-S	11-058-02SSL-S
7	Dight	11 050 02TID C	11 0E0 0200D C



2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate



STERILE

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
8	Left	11-058-03TIL	11-058-03SSL
8	Right	11-058-03TIR	11-058-03SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
8	Left	11-058-03TIL-S	11-058-03SSL-S
8	Right	11-058-03TIR-S	11-058-03SSR-S

2.4mm AV-Wiselock Lateral Distal Radius Small Fragment M Plate





HOLES	TITANIUM	STAINLESS STEEL
6	11-058-04TI	11-058-04SS

HOLES	TITANIUM	STAINLESS STEEL
6	11-058-04TI-S	11-058-04SS-S



2.4mm AV-Wiselock Distal Radius Volar Rim M Plate



HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
13	Left	11-058-05TIL	11-058-05SSL
13	Right	11-058-05TIR	11-058-05SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
13	Left	11-058-05TIL-S	11-058-05SSL-S
13	Right	11-058-05TIR-S	11-058-05SSR-S

2.4mm AV-Wiselock Volar Lunate Facet M Plate



HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
7	Left	11-058-06TIL	11-058-06SSL
7	Right	11-058-06TIR	11-058-06SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
7	Left	11-058-06TIL-S	11-058-06SSL-S
7	Right	11-058-06TIR-S	11-058-06SSR-S







HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
10	Left	11-058-21TIL	11-058-21SSL
10	Right	11-058-22TIR	11-058-22SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
12	Left	11-058-23TIL	11-058-23SSL
10	Diaht	11 OEO 24TID	11 OEO 24CCD

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
10	Left	11-058-21TIL-S	11-058-21SSL-S
10	Right	11-058-22TIR-S	11-058-22SSR-S

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
12	Left	11-058-23TIL-S	11-058-23SSL-S
12	Right	11-058-24TIR-S	11-058-24SSR-S

2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, Narrow







HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
18	Left	11-058-08TIL	11-058-08SSL
18	Right	11-058-08TIR	11-058-08SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
18	Left	11-058-08TIL-S	11-058-08SSL-S
18	Right	11-058-08TIR-S	11-058-08SSR-S



2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate




HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
20	Left	11-058-09TIL	11-058-09SSL
20	Right	11-058-09TIR	11-058-09SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
20	Left	11-058-09TIL-S	11-058-09SSL-S
20	Right	11-058-09TIR-S	11-058-09SSR-S

STERILE

2.4mm AV-Wiselock Volar Distal Radius M Plate, XL

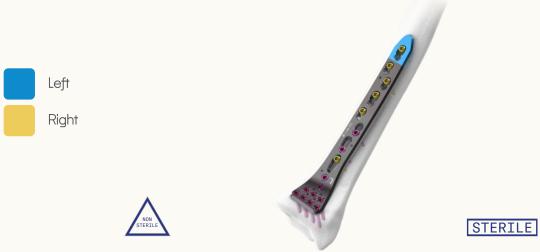


HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
17	Left	11-058-10TIL	11-058-10SSL
17	Right	11-058-10TIR	11-058-10SSR
20	Left	11-058-11TIL	11-058-11SSL
20	Right	11-058-11TIR	11-058-11SSR
24	Left	11-058-12TIL	11-058-12SSL
24	Right	11-058-12TIR	11-058-12SSR

HOLES	DIRECTION TITANIUM		DIRECTION	TITANIUM	STAINLESS STEEL
17	Left	11-058-10TIL-S	11-058-10SSL-S		
17	Right	11-058-10TIR-S	11-058-10SSR-S		
20	Left	11-058-11TIL-S	11-058-11SSL-S		
20	Right	11-058-11TIR-S	11-058-11SSR-S		
24	Left	11-058-12TIL-S	11-058-12SSL-S		
24	Right	11-058-12TIR-S	11-058-12SSR-S		





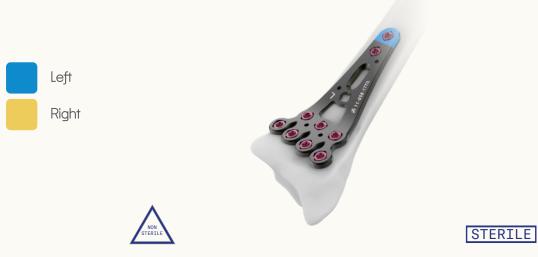

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
11	Left	11-058-17TIL	11-058-17SSL
11	Right	11-058-17TIR	11-058-17SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
11	Left	11-058-17TIL-S	11-058-17SSL-S
11	Right	11-058-17TIR-S	11-058-17SSR-S

2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Narrow, Short



HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
10	Left	11-058-18TIL	11-058-18SSL
4.0	D: 1.	11 050 1050	11 050 10000

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
10	Left	11-058-18TIL-S	11-058-18SSL-S
10	Right	11-058-18TIR-S	11-058-18SSR-S



2.4mm AV-Wiselock Distal Ulna M Plate





HOLES	TITANIUM	STAINLESS STEEL
7	11-058-19TI	11-058-19SS
10	11-058-20TI	11-058-20SS

STERILE

HOLES	TITANIUM	STAINLESS STEEL
7	11-058-19TI-S	11-058-19SS-S
10	11-058-20TI-S	11-058-20SS-S

2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate



HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
12	Left	11-058-07TIL	11-058-07SSL
12	Right	11-058-07TIR	11-058-07SSR

HOLES	DIRECTION	TITANIUM	STAINLESS STEEL
12	Left	11-058-07TIL-S	11-058-07SSL-S
12	Right	11-058-07TIR-S	11-058-07SSR-S



2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column





Holes	Titanium	Stainless Steel
5	11-045-05TI	11-045-05SS
6	11-045-06TI	11-045-06SS

STERILE

Holes	Titanium	Stainless Steel
5	11-045-05TI-S	11-045-05SS-S
6	11-045-06TI-S	11-045-06SS-S

2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column



Right



Direction	Titanium	Stainless Steel
Left	11-046-03LTI	11-046-03LSS
Right	11-046-03RTI	11-046-03RSS
Left	11-046-04LTI	11-046-04LSS

11-046-04RSS

11-046-04RTI



Holes	Direction	Titanium	Stainless Steel
3	Left	11-046-03LTI-S	11-046-03LSS-S
3	Right	11-046-03RTI-S	11-046-03RSS-S
4	Left	11-046-04LTI-S	11-046-04LSS-S
4	Right	11-046-04RTI-S	11-046-04RSS-S







(2 Head Holes)

Left

Right



Holes	Direction	Titanium	Stainless Steel
3	Left	11-047-03LTI	11-047-03LSS
3	Right	11-047-03RTI	11-047-03RSS
5	Left	11-047-05LTI	11-047-05LSS
5	Right	11-047-05RTI	11-047-05RSS

STERILE

Holes	Direction	Titanium	Stainless Steel
3	Left	11-047-03LTI-S	11-047-03LSS-S
3	Right	11-047-03RTI-S	11-047-03RSS-S
5	Left	11-047-05LTI-S	11-047-05LSS-S
5	Right	11-047-05RTI-S	11-047-05RSS-S

(3 Head Holes)



Holes	Direction	Titanium	Stainless Steel
3	Left	11-048-03LTI	11-048-03LSS
3	Right	11-048-03RTI	11-048-03RSS
5	Left	11-048-05LTI	11-048-05LSS
5	Right	11-048-05RTI	11-048-05RSS

STERILE

Holes	Direction	Titanium	Stainless Steel
3	Left	11-048-03LTI-S	11-048-03LSS-S
3	Right	11-048-03RTI-S	11-048-03RSS-S
5	Left	11-048-05LTI-S	11-048-05LSS-S
5	Right	11-048-05RTI-S	11-048-05RSS-S

(Oblique 3 Head Holes)



Holes	Direction	Titanium	Stainless Steel
3	Left	11-049-03LTI	11-049-03LSS
3	Right	11-049-03RTI	11-049-03RSS
5	Left	11-049-05LTI	11-049-05LSS
5	Right	11-049-05RTI	11-049-05RSS

Holes	Direction	Titanium	Stainless Steel
3	Left	11-049-03LTI-S	11-049-03LSS-S
3	Right	11-049-03RTI-S	11-049-03RSS-S
5	Left	11-049-05LTI-S	11-049-05LSS-S
5	Right	11-049-05RTI-S	11-049-05RSS-S



2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes)





Holes	Titanium	Stainless Steel
3	11-050-03TI	11-050-03SS
5	11-050-05TI	11-050-05SS

Holes	Titanium	Stainless Steel
3	11-050-03TI-S	11-050-03SS-S
5	11-050-05TI-S	11-050-05SS-S









Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-1144.06	1144.06
Ø2.4	8mm	TI-1144.08	1144.08
Ø2.4	10mm	TI-1144.10	1144.10
Ø2.4	12mm	TI-1144.12	1144.12
Ø2.4	14mm	TI-1144.14	1144.14
Ø2.4	16mm	TI-1144.16	1144.16
Ø2.4	18mm	TI-1144.18	1144.18
Ø2.4	20mm	TI-1144.20	1144.20
Ø2.4	22mm	TI-1144.22	1144.22
Ø2.4	24mm	TI-1144.24	1144.24
Ø2.4	26mm	TI-1144.26	1144.26
Ø2.4	28mm	TI-1144.28	1144.28
Ø2.4	30mm	TI-1144.30	1144.30
Ø2.4	32mm	TI-1144.32	1144.32
Ø2.4	34mm	TI-1144.34	1144.34





STERILE

·X:

Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-1144.06-S	1144.06-S
Ø2.4	8mm	TI-1144.08-S	1144.08-S
Ø2.4	10mm	TI-1144.10-S	1144.10-S
Ø2.4	12mm	TI-1144.12-S	1144.12-S
Ø2.4	14mm	TI-1144.14-S	1144.14-S
Ø2.4	16mm	TI-1144.16-S	1144.16-S
Ø2.4	18mm	TI-1144.18-S	1144.18-S
Ø2.4	20mm	TI-1144.20-S	1144.20-S
Ø2.4	22mm	TI-1144.22-S	1144.22-S
Ø2.4	24mm	TI-1144.24-S	1144.24-S
Ø2.4	26mm	TI-1144.26-S	1144.26-S
Ø2.4	28mm	TI-1144.28-S	1144.28-S
Ø2.4	30mm	TI-1144.30-S	1144.30-S
Ø2.4	32mm	TI-1144.32-S	1144.32-S
Ø2.4	34mm	TI-1144.34-S	1144.34-S

2.4mm Variable Angle Screw, Self-Tapping, (Star Head)



Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-549.006	549.006
Ø2.4	8mm	TI-549.008	549.008
Ø2.4	10mm	TI-549.010	549.010
Ø2.4	12mm	TI-549.012	549.012
Ø2.4	14mm	TI-549.014	549.014
Ø2.4	16mm	TI-549.016	549.016
Ø2.4	18mm	TI-549.018	549.018
Ø2.4	20mm	TI-549.020	549.020
Ø2.4	22mm	TI-549.022	549.022
Ø2.4	24mm	TI-549.024	549.024
Ø2.4	26mm	TI-549.026	549.026
Ø2.4	28mm	TI-549.028	549.028
Ø2.4	30mm	TI-549.030	549.030
Ø2.4	32mm	TI-549.032	549.032
Ø2.4	34mm	TI-549.034	549.034





Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-549.006-S	549.006-S
Ø2.4	8mm	TI-549.008-S	549.008-S
Ø2.4	10mm	TI-549.010-S	549.010-S
Ø2.4	12mm	TI-549.012-S	549.012-S
Ø2.4	14mm	TI-549.014-S	549.014-S
Ø2.4	16mm	TI-549.016-S	549.016-S
Ø2.4	18mm	TI-549.018-S	549.018-S
Ø2.4	20mm	TI-549.020-S	549.020-S
Ø2.4	22mm	TI-549.022-S	549.022-S
Ø2.4	24mm	TI-549.024-S	549.024-S
Ø2.4	26mm	TI-549.026-S	549.026-S
Ø2.4	28mm	TI-549.028-S	549.028-S
Ø2.4	30mm	TI-549.030-S	549.030-S
Ø2.4	32mm	TI-549.032-S	549.032-S
Ø2.4	34mm	TI-549.034-S	549.034-S





2.7mm Cortical Screw, Self-Tapping, (Star Head)



(
•	a l



Dia	Length	Titanium	Stainless Steel
Ø2.7	6mm	TI-1242.06-S	1242.06-S
Ø2.7	8mm	TI-1242.08-S	1242.08-S
Ø2.7	10mm	TI-1242.10-S	1242.10-S
Ø2.7	12mm	TI-1242.12-S	1242.12-S
Ø2.7	14mm	TI-1242.14-S	1242.14-S
Ø2.7	16mm	TI-1242.16-S	1242.16-S
Ø2.7	18mm	TI-1242.18-S	1242.18-S
Ø2.7	20mm	TI-1242.20-S	1242.20-S
Ø2.7	22mm	TI-1242.22-S	1242.22-S
Ø2.7	24mm	TI-1242.24-S	1242.24-S
Ø2.7	26mm	TI-1242.26-S	1242.26-S
Ø2.7	28mm	TI-1242.28-S	1242.28-S
Ø2.7	30mm	TI-1242.30-S	1242.30-S

Dia	Length	Titanium	Stainless Steel
Ø2.7	6mm	TI-1242.06	1242.06
Ø2.7	8mm	TI-1242.08	1242.08
Ø2.7	10mm	TI-1242.10	1242.10
Ø2.7	12mm	TI-1242.12	1242.12
Ø2.7	14mm	TI-1242.14	1242.14
Ø2.7	16mm	TI-1242.16	1242.16
Ø2.7	18mm	TI-1242.18	1242.18
Ø2.7	20mm	TI-1242.20	1242.20
Ø2.7	22mm	TI-1242.22	1242.22
Ø2.7	24mm	TI-1242.24	1242.24
Ø2.7	26mm	TI-1242.26	1242.26
Ø2.7	28mm	TI-1242.28	1242.28
Ø2.7	30mm	TI-1242.30	1242.30



Optional Screws

2.4mm Wise-Lock Screw, Self-Tapping, (Star Head)



Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-1145.06	1145.06
Ø2.4	8mm	TI-1145.08	1145.08
Ø2.4	10mm	TI-1145.10	1145.10
Ø2.4	12mm	TI-1145.12	1145.12
Ø2.4	14mm	TI-1145.14	1145.14
Ø2.4	16mm	TI-1145.16	1145.16
Ø2.4	18mm	TI-1145.18	1145.18
Ø2.4	20mm	TI-1145.20	1145.20
Ø2.4	22mm	TI-1145.22	1145.22
Ø2.4	24mm	TI-1145.24	1145.24
Ø2.4	26mm	TI-1145.26	1145.26
Ø2.4	28mm	TI-1145.28	1145.28
Ø2.4	30mm	TI-1145.30	1145.30





STERILE

-><

Dia	Length	Titanium	Stainless Steel
Ø2.4	6mm	TI-1145.06-S	1145.06-S
Ø2.4	8mm	TI-1145.08-S	1145.08-S
Ø2.4	10mm	TI-1145.10-S	1145.10-S
Ø2.4	12mm	TI-1145.12-S	1145.12-S
Ø2.4	14mm	TI-1145.14-S	1145.14-S
Ø2.4	16mm	TI-1145.16-S	1145.16-S
Ø2.4	18mm	TI-1145.18-S	1145.18-S
Ø2.4	20mm	TI-1145.20-S	1145.20-S
Ø2.4	22mm	TI-1145.22-S	1145.22-S
Ø2.4	24mm	TI-1145.24-S	1145.24-S
Ø2.4	26mm	TI-1145.26-S	1145.26-S
Ø2.4	28mm	TI-1145.28-S	1145.28-S
Ø2.4	30mm	TI-1145.30-S	1145.30-S

1.8mm Buttress Screw, (Star Head)



Dia	Length	Titanium	Stainless Steel
Ø1.8	12mm	TI-1103.112	1103.112
Ø1.8	14mm	TI-1103.114	1103.114
Ø1.8	16mm	TI-1103.116	1103.116
Ø1.8	18mm	TI-1103.118	1103.118
Ø1.8	20mm	TI-1103.120	1103.120
Ø1.8	22mm	TI-1103.122	1103.122
Ø1.8	24mm	TI-1103.124	1103.124





STERILE

Dia	Length	Titanium	Stainless Steel
Ø1.8	12mm	TI-1103.112-S	1103.112-S
Ø1.8	14mm	TI-1103.114-S	1103.114-S
Ø1.8	16mm	TI-1103.116-S	1103.116-S
Ø1.8	18mm	TI-1103.118-S	1103.118-S
Ø1.8	20mm	TI-1103.120-S	1103.120-S
Ø1.8	22mm	TI-1103.122-S	1103.122-S
Ø1.8	24mm	TI-1103.124-S	1103.124-S

1.5mm Cortical Screw, Self-Tapping, (Star Head)



Dia	Length	Titanium	Stainless Steel
Ø1.5	8mm	TI-1281.08	1281.08
Ø1.5	10mm	TI-1281.10	1281.10
Ø1.5	12mm	TI-1281.12	1281.12
Ø1.5	14mm	TI-1281.14	1281.14





Dia	Length	Titanium	Stainless Steel
Ø1.5	8mm	TI-1281.08-S	1281.08-S
Ø1.5	10mm	TI-1281.10-S	1281.10-S
Ø1.5	12mm	TI-1281.12-S	1281.12-S
Ø1.5	14mm	TI-1281.14-S	1281.14-S





IMPLANTS COMPOSITION

CODE	Item Description	UNIT
	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, Titanium	
11-058-01TIL	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, 5 Holes, Left, Titanium	1
11-058-01TIR	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, 5 Holes, Right, Titanium	1
	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, Titanium	
11-058-02TIL	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, 7 Holes, Left, Titanium	1
11-058-02TIR	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, 7 Holes, Right, Titanium	1
	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, Titanium	
11-058-03TIL	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, 8 Holes, Left, Titanium	1
11-058-03TIR	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, 8 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Lateral Distal Radius Small Fragment M Plate, Titanium	
11-058-04TI	2.4mm AV-Wiselock Lateral Distal Radius Small Fragment M Plate, 6 Holes, Titanium	1
	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, Titanium	
11-058-05TIL	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, 13 Holes, Left, Titanium	1
11-058-05TIR	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, 13 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Volar Lunate Facet M Plate, Titanium	
11-058-06TIL	2.4mm AV-Wiselock Volar Lunate Facet M Plate, 7 Holes, Left, Titanium	1
11-058-06TIR	2.4mm AV-Wiselock Volar Lunate Facet M Plate, 7 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, Titanium	
11-058-07TIL	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, 12 Holes, Left, Titanium	1
11-058-07TIR	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, 12 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 18 Holes, Titanium	
11-058-08TIL	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, Narrow, 18 Holes, Left, Titanium	1
11-058-08TIR	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, Narrow, 18 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Titanium	1
11-058-09TIL	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Left, Titanium	1
11-058-09TIR	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, Titanium	
11-058-10TIL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 17 Holes, Left, Titanium	1
11-058-11TIL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 20 Holes, Left, Titanium	1
11-058-12TIL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 24 Holes, Left, Titanium	1
11-058-10TIR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 17 Holes, Right, Titanium	1
11-058-11TIR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 20 Holes, Right, Titanium	1
11-058-12TIR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 24 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Titanium	
11-058-17TIL	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, 11 Holes, Left, Titanium	1
11-058-17TIR	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, 11 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Titanium	
11-058-18TIL	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Narrow, Short, 10 Holes, Left, Titanium	1
11-058-18TIR	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Narrow, Short, 10 Holes, Right, Titanium	1



IMPLANTS COMPOSITION

CODE	Item Description	UNIT
	2.4mm AV-Wiselock Distal Ulna M Plate, Titanium	
11-058-19TI	2.4mm AV-Wiselock Distal Ulna M Plate, 7 Holes, Titanium	1
11-058-20TI	2.4mm AV-Wiselock Distal Ulna M Plate, 10 Holes, Titanium	1
	2.4mm AV-Wiselock FPL Volar Distal Radius Facet M Plate, Titanium	
11-058-21TIL	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, 10 Holes, Left, Titanium	1
11-058-22TIR	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, 10 Holes, Right, Titanium	1
11-058-23TIL	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, Long, 12 Holes, Left, Titanium	1
11-058-24TIR	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, Long, 12 Holes, Right, Titanium	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, Titanium	
11-045-05TI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, 5 Holes, Titanium	1
11-045-06TI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, 6 Holes, Titanium	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), Left, Titanium	
11-046-03LTI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 3 Holes - Left, Titanium	1
11-046-04LTI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 4 Holes - Left, Titanium	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), Right, Titanium	
11-046-03RTI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 3 Holes - Right, Titanium	1
11-046-04RTI	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 4 Holes - Right, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), Left, Titanium	
11-047-03LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 3 Holes - Left, Titanium	1
11-047-05LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 5 Holes - Left, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), Right, Titanium	
11-047-03RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 3 Holes - Right, Titanium	1
11-047-05RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 5 Holes - Right, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), Left, Titanium	
11-048-03LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes - Left, Titanium	1
11-048-05LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes - Left, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), Right, Titanium	
11-048-03RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes - Right, Titanium	1
11-048-05RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes - Right, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), Left, Titanium	
11-049-03LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 3 Holes - Left, Titanium	1
11-049-05LTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 5 Holes - Left, Titanium	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), Right, Titanium	
11-049-03RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 3 Holes - Right, Titanium	1
11-049-05RTI	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 5 Holes - Right, Titanium	1
	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), Titanium	
11-050-03TI	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes, Titanium	1
11-050-05TI	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes, Titanium	1





SCREWS COMPOSITION

CODE	Item Description	UNIT
	2.4mm Cortical Screws, Self Tapping, (Star Head), Titanium	
TI-1144.06	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 6mm, Titanium	3
TI-1144.08	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	3
TI-1144.10	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Titanium	3
TI-1144.12	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Titanium	3
TI-1144.14	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Titanium	3
TI-1144.16	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 16mm, Titanium	3
TI-1144.18	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 18mm, Titanium	3
TI-1144.20	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 20mm, Titanium	3
TI-1144.22	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 22mm, Titanium	3
TI-1144.24	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 24mm, Titanium	3
TI-1144.26	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 26mm, Titanium	3
TI-1144.28	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 28mm, Titanium	3
TI-1144.30	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 30mm, Titanium	3
TI-1144.32	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 32mm, Titanium	3
TI-1144.34	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 34mm, Titanium	3
	2.4mm Variable Angle Screws, Self Tapping, (Star Head), Titanium	
TI-549.006	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 6mm, Titanium	6
TI-549.008	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	6
TI-549.010	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 10mm, Titanium	6
TI-549.012	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 12mm, Titanium	6
TI-549.014	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 14mm, Titanium	6
TI-549.016	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 16mm, Titanium	6
TI-549.018	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 18mm, Titanium	6
TI-549.020	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 20mm, Titanium	6
TI-549.022	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 22mm, Titanium	6
TI-549.024	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 24mm, Titanium	6
TI-549.026	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 26mm, Titanium	6
TI-549.028	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 28mm, Titanium	6
TI-549.030	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 30mm, Titanium	6
TI-549.032	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 32mm, Titanium	6
TI-549.034	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 34mm, Titanium	6
	1.5mm Cortical Screws, Self-Tapping, (Star Head), Titanium	
TI-1281.08	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	2
TI-1281.10	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Titanium	2
TI-1281.12	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Titanium	2
TI-1281.14	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Titanium	2





SCREWS COMPOSITION

CODE	Item Description	UNIT
	2.4mm Wise-Lock Screws, Self-Tapping, (Star Head), Titanium	
TI-1145.06	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	6
TI-1145.08	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	6
TI-1145.10	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 10mm, Titanium	6
TI-1145.12	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 12mm, Titanium	6
TI-1145.14	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 14mm, Titanium	6
TI-1145.16	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 16mm, Titanium	6
TI-1145.18	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 18mm, Titanium	6
TI-1145.20	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 20mm, Titanium	6
TI-1145.22	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 22mm, Titanium	6
TI-1145.24	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 24mm, Titanium	6
TI-1145.26	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 26mm, Titanium	6
TI-1145.28	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 28mm, Titanium	6
TI-1145.30	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 30mm, Titanium	6
	1.8mm Buttress Screws, (Star Head), Titanium	
TI-1103.112	1.8mm Buttress Screw, (Star Head), Length 12mm, Titanium	3
TI-1103.114	1.8mm Buttress Screw, (Star Head), Length 14mm, Titanium	3
TI-1103.116	1.8mm Buttress Screw, (Star Head), Length 16mm, Titanium	3
TI-1103.118	1.8mm Buttress Screw, (Star Head), Length 18mm, Titanium	3
TI-1103.120	1.8mm Buttress Screw, (Star Head), Length 20mm, Titanium	3
TI-1103.122	1.8mm Buttress Screw, (Star Head), Length 22mm, Titanium	3
TI-1103.124	1.8mm Buttress Screw, (Star Head), Length 24mm, Titanium	3
	2.7mm Cortical Screws, Self-Tapping, (Star Head), Titanium	
TI-1242.06	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 6mm, Titanium	3
TI-1242.08	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Titanium	3
TI-1242.10	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Titanium	3
TI-1242.12	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Titanium	3
TI-1242.14	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Titanium	3
TI-1242.16	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 16mm, Titanium	3
TI-1242.18	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 18mm, Titanium	3
TI-1242.20	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 20mm, Titanium	3
TI-1242.22	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 22mm, Titanium	3
TI-1242.24	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 24mm, Titanium	3
TI-1242.26	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 26mm, Titanium	3
TI-1242.28	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 28mm, Titanium	3
TI-1242.30	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 30mm, Titanium	3





IMPLANTS COMPOSITION

CODE	Item Description	UNIT
	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, Stainless Steel	
11-058-01SSL	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, 5 Holes, Left, Stainless Steel	1
11-058-01SSR	2.4mm AV-Wiselock Distal Radius Small Fragment M Plate, Curved, 5 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, Stainless Steel	
11-058-02SSL	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, 7 Holes, Left, Stainless Steel	1
11-058-02SSR	2.4mm AV-Wiselock T-Distal Radius Small Fragment M Plate, 7 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, Stainless Steel	
11-058-03SSL	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, 8 Holes, Left, Stainless Steel	1
11-058-03SSR	2.4mm AV-Wiselock L-Distal Radius Small Fragment M Plate, 8 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Lateral Distal Radius Small Fragment M Plate, Stainless Steel	
11-058-04SS	2.4mm AV-Wiselock Lateral Distal Radius Small Fragment M Plate, 6 Holes, Stainless Steel	1
	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, Stainless Steel	
11-058-05SSL	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, 13 Holes, Left, Stainless Steel	1
11-058-05SSR	2.4mm AV-Wiselock Distal Radius Volar Rim M Plate, 13 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Volar Lunate Facet M Plate, Stainless Steel	
11-058-06SSL	2.4mm AV-Wiselock Volar Lunate Facet M Plate, 7 Holes, Left, Stainless Steel	1
11-058-06SSR	2.4mm AV-Wiselock Volar Lunate Facet M Plate, 7 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, Stainless Steel	
11-058-07SSL	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, 12 Holes, Left, Stainless Steel	1
11-058-07SSR	2.4mm AV-Wiselock Dorsal H-Distal Radius M Plate, 12 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 18 Holes, Stainless Steel	
11-058-08SSL	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, Narrow, 18 Holes, Left, Stainless Steel	1
11-058-08SSR	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, Narrow, 18 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Stainless Steel	1
11-058-09SSL	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Left, Stainless Steel	1
11-058-09SSR	2.4mm AV-Wiselock Dorsal Frame Distal Radius M Plate, 20 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, Stainless Steel	
11-058-10SSL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 17 Holes, Left, Stainless Steel	1
11-058-11SSL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 20 Holes, Left, Stainless Steel	1
11-058-12SSL	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 24 Holes, Left, Stainless Steel	1
11-058-10SSR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 17 Holes, Right, Stainless Steel	1
11-058-11SSR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 20 Holes, Right, Stainless Steel	1
11-058-12SSR	2.4mm AV-Wiselock Volar Distal Radius M Plate, XL, 24 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Stainless Steel	
11-058-17SSL	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, 11 Holes, Left, Stainless Steel	1
11-058-17SSR	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, 11 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Stainless Steel	
11-058-18SSL	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Narrow, Short, 10 Holes, Left, Stainless Steel	1
11-058-18SSR	2.4mm AV-Wiselock Volar Distal Radius Fracture M Plate, Narrow, Short, 10 Holes, Right, Stainless Steel	1





IMPLANTS COMPOSITION

CODE	Item Description	UNIT
	2.4mm AV-Wiselock Distal Ulna M Plate, Stainless Steel	
11-058-19SS	2.4mm AV-Wiselock Distal Ulna M Plate, 7 Holes, Stainless Steel	1
11-058-20SS	2.4mm AV-Wiselock Distal Ulna M Plate, 10 Holes, Stainless Steel	1
	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, Stainless Steel	
11-058-21SSL	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, 10 Holes, Left, Stainless Steel	1
11-058-22SSR	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, 10 Holes, Right, Stainless Steel	1
11-058-23SSL	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, Long, 12 Holes, Left, Stainless Steel	1
11-058-24SSR	2.4mm AV-Wiselock FPL Volar Distal Radius M Plate, Long, 12 Holes, Right, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, Stainless Steel	
11-045-05SS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, 5 Holes, Stainless Steel	1
11-045-06SS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Radial Column, 6 Holes, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), Left, Stainless Steel	
11-046-03LSS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 3 Holes - Left, Stainless Steel	1
11-046-04LSS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 4 Holes - Left, Stainless Steel	1
	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), Right, Stainless Steel	
11-046-03RSS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 3 Holes - Right, Stainless Steel	1
11-046-04RSS	2.4mm AV-Wiselock Dorsal Distal Radius Plate, Intermediate Column, (2 Head Holes), 4 Holes - Right, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), Left, Stainless Steel	
11-047-03LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 3 Holes - Left, Stainless Steel	1
11-047-05LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 5 Holes - Left, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), Right, Stainless Steel	
11-047-03RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 3 Holes - Right, Stainless Steel	1
11-047-05RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (2 Head Holes), 5 Holes - Right, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), Left, Stainless Steel	
11-048-03LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes - Left, Stainless Steel	1
11-048-05LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes - Left, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), Right, Stainless Steel	
11-048-03RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes - Right, Stainless Steel	1
11-048-05RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes - Right, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), Left, Stainless Steel	
11-049-03LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 3 Holes - Left, Stainless Steel	1
11-049-05LSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 5 Holes - Left, Stainless Steel	1
	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), Right, Stainless Steel	
11-049-03RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 3 Holes - Right, Stainless Steel	1
11-049-05RSS	2.4mm AV-Wiselock "L" Dorsal Distal Radius Plate, Oblique (3 Head Holes), 5 Holes - Right, Stainless Steel	1
	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), Stainless Steel	
11-050-03SS	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), 3 Holes, Stainless Steel	1
11-050-05SS	2.4mm AV-Wiselock "T" Dorsal Distal Radius Plate, (3 Head Holes), 5 Holes, Stainless Steel	1





SCREWS COMPOSITION

CODE	Item Description	UNIT
	2.4mm Cortical Screw, Self Tapping, (Star Head), Stainless Steel	
1144.06	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 6mm, Stainless Steel	3
1144.08	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	3
1144.10	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Stainless Steel	3
1144.12	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Stainless Steel	3
1144.14	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Stainless Steel	3
1144.16	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 16mm, Stainless Steel	3
1144.18	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 18mm, Stainless Steel	3
1144.20	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 20mm, Stainless Steel	3
1144.22	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 22mm, Stainless Steel	3
1144.24	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 24mm, Stainless Steel	3
1144.26	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 26mm, Stainless Steel	3
1144.28	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 28mm, Stainless Steel	3
1144.30	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 30mm, Stainless Steel	3
1144.32	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 32mm, Stainless Steel	3
1144.34	2.4mm Cortical Screw, Self-Tapping, (Star Head), Length 34mm, Stainless Steel	3
	2.4mm Variable Angle Screws, Self Tapping, (Star Head), Stainless Steel	
549.006	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 6mm, Stainless Steel	6
549.008	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	6
549.010	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 10mm, Stainless Steel	6
549.012	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 12mm, Stainless Steel	6
549.014	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 14mm, Stainless Steel	6
549.016	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 16mm, Stainless Steel	6
549.018	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 18mm, Stainless Steel	6
549.020	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 20mm, Stainless Steel	6
549.022	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 22mm, Stainless Steel	6
549.024	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 24mm, Stainless Steel	6
549.026	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 26mm, Stainless Steel	6
549.028	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 28mm, Stainless Steel	6
549.030	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 30mm, Stainless Steel	6
549.032	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 32mm, Stainless Steel	6
549.034	2.4mm Variable Angle Screw, Self-Tapping, (Star Head), Length 34mm, Stainless Steel	6
	1.5mm Cortical Screws, Self-Tapping, (Star Head), Stainless Steel	
1281.08	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	2
1281.10	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Stainless Steel	2
1281.12	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Stainless Steel	2
1281.14	1.5mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Stainless Steel	2





SCREWS COMPOSITION

CODE	Item Description	UNIT
	2.4mm Wise-Lock Screws, Self-Tapping, (Star Head), Stainless Steel	
1145.06	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	6
1145.08	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	6
1145.10	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 10mm, Stainless Steel	6
1145.12	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 12mm, Stainless Steel	6
1145.14	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 14mm, Stainless Steel	6
1145.16	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 16mm, Stainless Steel	6
1145.18	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 18mm, Stainless Steel	6
1145.20	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 20mm, Stainless Steel	6
1145.22	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 22mm, Stainless Steel	6
1145.24	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 24mm, Stainless Steel	6
1145.26	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 26mm, Stainless Steel	6
1145.28	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 28mm, Stainless Steel	6
1145.30	2.4mm Wise-Lock Screw, Self-Tapping, (Star Head), Length 30mm, Stainless Steel	6
	1.8mm Buttress Screws, (Star Head), (Star Head), Stainless Steel	
1103.112	1.8mm Buttress Screw, (Star Head), Length 12mm, Stainless Steel	3
1103.114	1.8mm Buttress Screw, (Star Head), Length 14mm, Stainless Steel	3
1103.116	1.8mm Buttress Screw, (Star Head), Length 16mm, Stainless Steel	3
1103.118	1.8mm Buttress Screw, (Star Head), Length 18mm, Stainless Steel	3
1103.120	1.8mm Buttress Screw, (Star Head), Length 20mm, Stainless Steel	3
1103.122	1.8mm Buttress Screw, (Star Head), Length 22mm, Stainless Steel	3
1103.124	1.8mm Buttress Screw, (Star Head), Length 24mm, Stainless Steel	3
	2.7mm Cortical Screws, Self-Tapping, (Star Head), Stainless Steel	
1242.06	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 6mm, Stainless Steel	3
1242.08	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 8mm, Stainless Steel	3
1242.10	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 10mm, Stainless Steel	3
1242.12	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 12mm, Stainless Steel	3
1242.14	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 14mm, Stainless Steel	3
1242.16	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 16mm, Stainless Steel	3
1242.18	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 18mm, Stainless Steel	3
1242.20	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 20mm, Stainless Steel	3
1242.22	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 22mm, Stainless Steel	3
1242.24	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 24mm, Stainless Steel	3
1242.26	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 26mm, Stainless Steel	3
1242.28	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 28mm, Stainless Steel	3
1242.30	2.7mm Cortical Screw, Self-Tapping, (Star Head), Length 30mm, Stainless Steel	3

INSTRUMENTS



1312-01 Depth Gauge measuring up to 60mm



1312-02 T-Handle With Quick Coupling



1312-03-15 Retractor, 15mm



1312-04-1.8 Drill Bit with Quick Coupling End, Ø1.8mm x Length 140mm



1312-05-2.0 Drill Bit with Quick Coupling End, Ø2.0mm x Length 140mm



1312-06-2.4 Drill Bit with Quick Coupling End, Ø2.4mm x Length 140mm








Drill Bit with Quick Coupling End, Ø2.7mm x Length 140mm



1312-08-1.2

Guide Wire, Ø1.2mm x Length 150mm

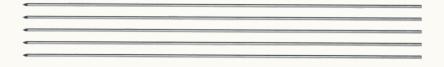

1312-09-1.2

Guide Wire With Threaded Tip, Ø1.2mm x Thread Length 5mm x Length 150mm



1312-10-2.4

Tap, HA 2.4mm



1312-11-2.7

Tap, HA 2.7mm



1312-13-1.8

Threaded Drill Sleeve, 01.8/2.4mm (Double Start Thread)





1312-15

Self-Centering Double Drill Guide Ø1.8/2.4mm



1312-16

Self-Centering Double Drill Guide Ø2.0/2.7mm



1312-17

Depth Gauge with Protector measuring up to 50mm



1312-18

Star Screwdriver Shaft, T8



1312-21

Torque Limiting Attachment, 0.8Nm



7-019-03

Handle for Torque Limiting Attachment





Retractor, 6.5mm



1312-23

Double Periosteal Elevator, Round



1312-24

Double Periosteal Elevator, Flat



1312-25

Reduction Forcep with Points, Small



1312-26

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
Reduction Forcep with Points, Large



1312-27

Reduction Forcep with Points, Serrated





1312-28

Bending Forcep



1312-30

Universal Variable Angle Drill Guide, Ø1.8mm



1312-20-2.7

Screw Holding Sleeve for Ø2.7mm Cortical Screw



1312-20-2.4

Screw Holding Sleeve for Ø2.4mm Cortical Screw



1312-30-2.4

Screw Holding Sleeve for Ø2.4mm Wise-lock Screw



1312-22

Quick Coupling Handle





1312-29

Sharp Hook



1312-31

Screwdriver Star Head



1312-35

Instrument trays for 2.4mm AV-Wiselock Distal Radius Instrument




1312-34


Container for 2.4mm AV-Wiselock Distal Radius Instrument





1312-000: 2.4mm AV-Wiselock Distal Radius Instrument Set

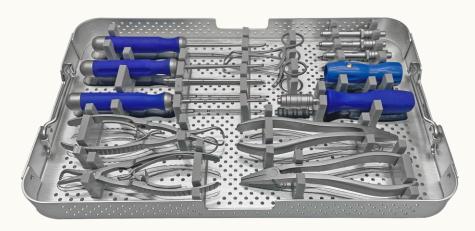

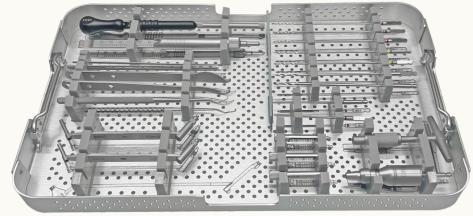



11-051

Implant Box with cover for 2.4mm AV-Wiselock Distal Radius Dorsal & Extra-Articular System





INDIA ALIXEIN MEDICAL PVILITO

PLOT NO. 168-169-170, PHASE-4, KUNDLI INDUSTRIAL AREA, HSIIDC, SECTOR-57, SONEPAT-131028, HARYANA TEL: +91 99106 43638 FAX: +91 86077 70197 EMAIL: INFO@AUXEIN.COM

MÉXICO ALIXEIN MÉXICO, S A DE CV

TEPIC 39 INT 801, COLONIA ROMA SUR, ALCADÍA CUAUHTÉMOC, CDMX MÉXICO, C.P 06760 TEL: +521 55 7261 0318 EMAIL: INFO@AUXEIN.MX

USA AUXFIN INC

1500 NW 89TH COURT, SUITE 107-108 DORAL,FLORIDA 33172 TEL. +1 305 395 6062 E FAX:+1 305 395 62 62 EMAIL: USOFFICE@AUXEIN.COM