


AUXILOCK® AC JOINT KIT

AC JOINT REPAIR

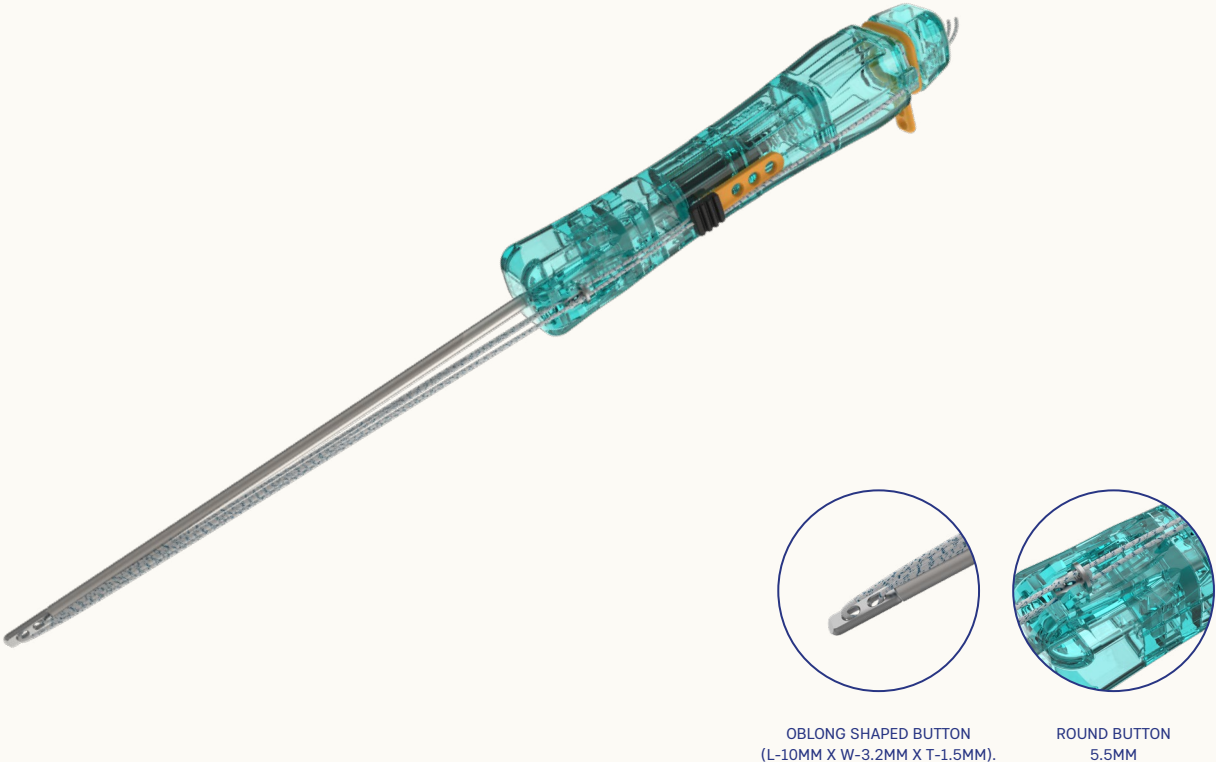

AUXILOCK® AC Joint Kit can be used for Acromioclavicular Joint (AC Joint) separation repair. The kit comprises of adjustable loop with round shaped button and oblong shaped button. A tunnel is made through the clavicle to coracoid using 1.5mm K-Wire and 3.5mm Cannulated Drill Bit.

The kit is passed through a 3.5mm drilling tunnel from clavicle to coracoid, providing fixation with a round button on the clavicle and an oblong-shaped button at the inferior side of the coracoid. It is ideal for Rockwood type III, IV & V classification. The AC Joint Kit provides a double locking mechanism which eliminates the need for knot tying and also provides very strong compression on the clavicles.

ORDERING INFORMATION

CODE	PRODUCT DESCRIPTION
6-041-02	AUXILOCK® AC Joint Kit

RELATED INSTRUMENTATION (INCLUDE IN KIT)

CODE	PRODUCT DESCRIPTION
7-147-01	Cannulated Drill Bit with Quick Coupling End, Ø3.5mm x Length 175mm
7-147-02	Ø1.5mm K-Wire