


AUXILOCK® MENI-FIX ALL INSIDE

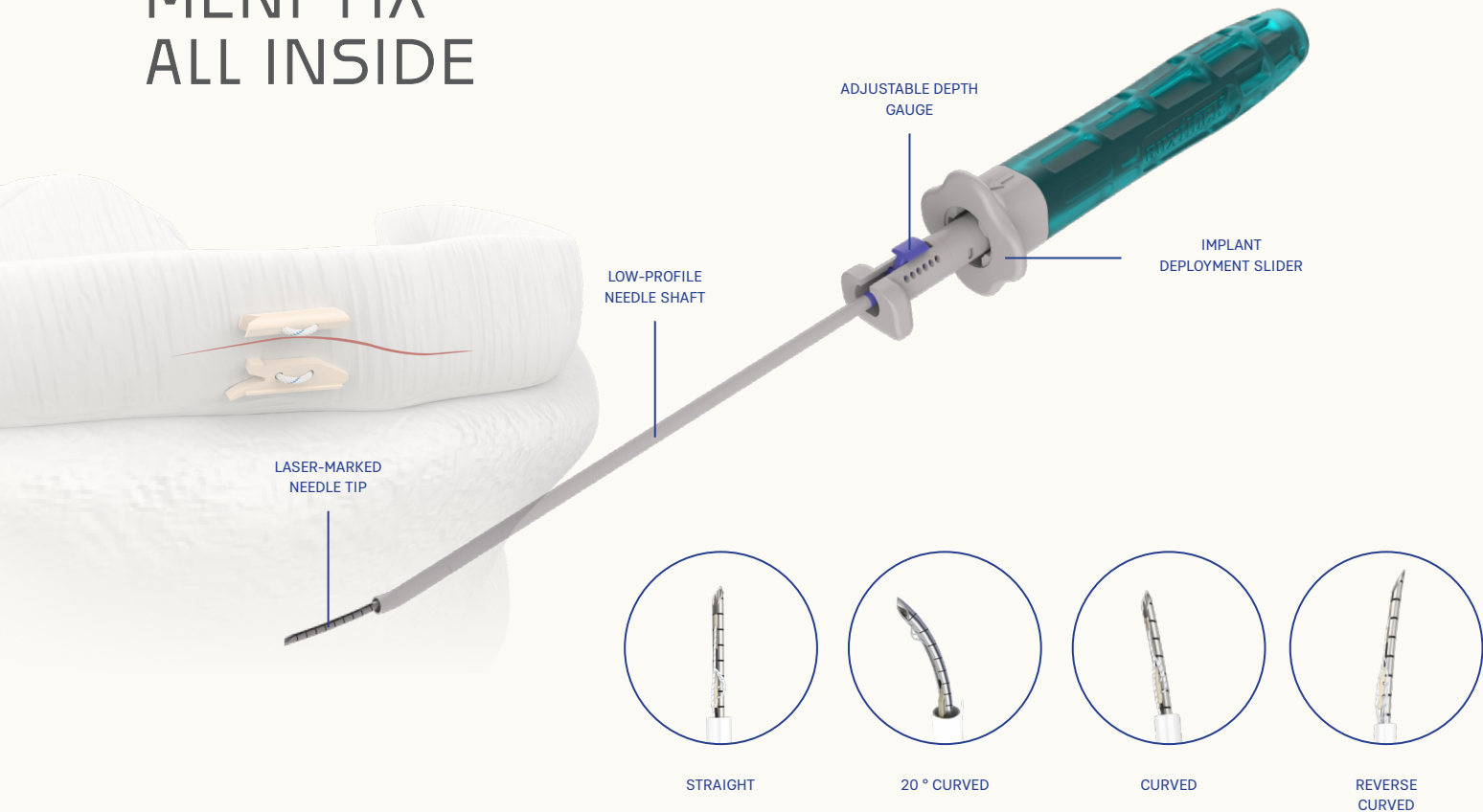

MENI-FIX All-Inside Meniscal Repair System includes two 0.9mm PEEK anchors with a pretied, self-sliding knot comprised of #2-0, UHMWPE Biobraid Suture.

The delivery needles are available in Straight, 15°Curved, 20° Curved and 15° Reverse Curved designs. The Curved and Reverse Curved designs allow the surgeon to rotate the needle tip away from the neurovascular structures when penetrating the meniscus, further reducing the risk of neurovascular injury. The Curved delivery needle is optimally shaped to allow vertical mattress sutures to be inserted on either the femoral or tibial surfaces of the meniscus.

The Reverse Curved delivery needle is most useful for repairing tears on the tibial surface and more anterior located tears. The built-in, adjustable depth penetration limiter is adjustable from 08mm to 18mm from the tip of the needle. Use of the meniscal depth probe in conjunction with the adjustable depth penetration limiter allows controlled delivery of the implants.

ORDERING INFORMATION

CODE	PRODUCT DESCRIPTION
6-048-01	AUXILOCK® MENI-FIX All-Inside Meniscal Repair System - Straight Needle
6-048-02	AUXILOCK® MENI-FIX All-Inside Meniscal Repair System - Curve Needle
6-048-03	AUXILOCK® MENI-FIX All-Inside Meniscal Repair System - Reverse Curve Needle
6-048-04	AUXILOCK® MENI-FIX All-Inside Meniscal Repair System - 20 ° Curve Needle

RELATED INSTRUMENTATION

CODE	PRODUCT DESCRIPTION
7-156-08	Slotted Cannula
7-156-14	Knot Pusher with Suture Cutter for Meniscal Repair