

Auxein


AJAX ADVANCE NAILING



NEXT-LEVEL CARE

ABOUT US

Auxein Medical is an integrated, research based, orthopaedic implants & instruments manufacturing company, producing a wide range of quality, affordable generic implants, trusted by healthcare professionals and patients across geographies. It is the Company's constant endeavour to provide a wide basket of generic and our innovator products that exceed the highest expectations of customers in term of quality and safety. The company has world-class manufacturing unit established in India and serves customers in over 75 countries worldwide.

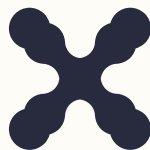

IMPACTING HEALTHCARE GLOBALLY

Our Reach & Influence

500+

Employees Worldwide

75+

Operations in Countries

150+

Distributors Worldwide

20^{M+}

Patients Cured

200+

FDA Approved Products

250+

Machines Worldwide

3000+

Innovative Orthopedic
Products & Solutions


Auxin is dedicated to advancing medical technology through innovative orthopedic solutions, including **Trauma, Arthroscopy, and Reconstructive Procedures**, with a commitment to improving patient outcomes worldwide. Our approach integrates advanced research, precision engineering, and high-quality manufacturing to develop reliable implants and surgical instruments that address the evolving needs of modern orthopedic practice.

OUR ACHIEVEMENTS



OVER THE YEARS




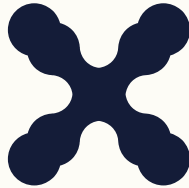


At Auxein, it is our goal to provide the best in class quality and value to every individual. Consistency is a virtue we pursue with a passion. From the procurement of raw materials to the manufacturing of our range of 1200+ orthopaedic implant products, we employ world-class systems and technology at par with the highest standards of globally recognized evaluation systems. The Ajax Advance Nailing System is part of our advanced orthopedic portfolio designed to support versatile fixation techniques in trauma and reconstructive procedures. Developed with input from global experts, the system provides reliable solutions for fixation of for intramedullary fixation of femoral fractures, helping achieve stable fixation and accurate anatomical alignment.



INDEX

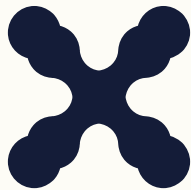


Introduction	11
AJAX Advance Nail	12-14
Screws	15-17
Implant Box	18-24
Instruments for Short Distal Targeting Device	25-38
Instruments for Distal Targeting Device	39-54
Cephalic Screws	55-61



AJAX ADVANCE NAILING SYSTEM

Ajax Advance Nailing System

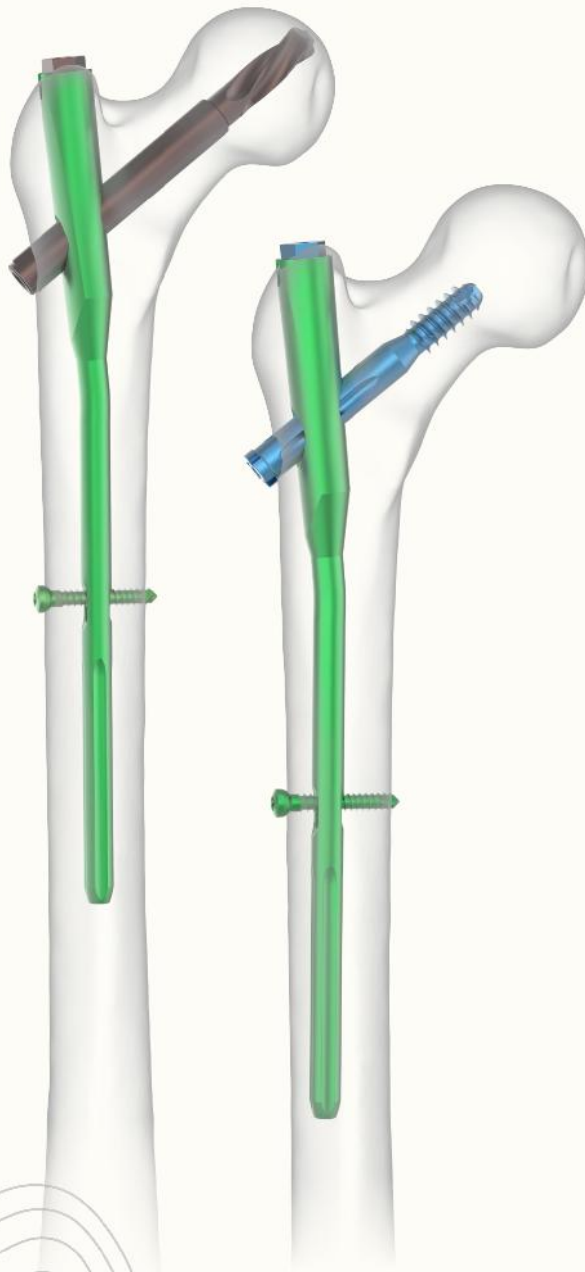

130° AJAX Advance Nail, Short, Titanium

Short

STERILE



Dia	Length	Sterile	Non-Sterile
9mm	170mm	TI-6085S-09-170-S	TI-6085S-09-170
9mm	200mm	TI-6085S-09-200-S	TI-6085S-09-200
9mm	240mm	TI-6085S-09-240-S	TI-6085S-09-240
10mm	170mm	TI-6085S-10-170-S	TI-6085S-10-170
10mm	200mm	TI-6085S-10-200-S	TI-6085S-10-200
10mm	240mm	TI-6085S-10-240-S	TI-6085S-10-240
11mm	170mm	TI-6085S-11-170-S	TI-6085S-11-170
11mm	200mm	TI-6085S-11-200-S	TI-6085S-11-200
11mm	240mm	TI-6085S-11-240-S	TI-6085S-11-240
12mm	170mm	TI-6085S-12-170-S	TI-6085S-12-170
12mm	200mm	TI-6085S-12-200-S	TI-6085S-12-200
12mm	240mm	TI-6085S-12-240-S	TI-6085S-12-240



130° AJAX Advance Nail, Long, Left

STERILE



Dia	Length	Sterile	Non-Sterile
9mm	300mm	TI-6085L-09-300L-S	TI-6085L-09-300L
9mm	320mm	TI-6085L-09-320L-S	TI-6085L-09-320L
9mm	340mm	TI-6085L-09-340L-S	TI-6085L-09-340L
9mm	360mm	TI-6085L-09-360L-S	TI-6085L-09-360L
9mm	380mm	TI-6085L-09-380L-S	TI-6085L-09-380L
9mm	400mm	TI-6085L-09-400L-S	TI-6085L-09-400L
9mm	420mm	TI-6085L-09-420L-S	TI-6085L-09-420L
10mm	300mm	TI-6085L-10-300L-S	TI-6085L-10-300L
10mm	320mm	TI-6085L-10-320L-S	TI-6085L-10-320L
10mm	340mm	TI-6085L-10-340L-S	TI-6085L-10-340L
10mm	360mm	TI-6085L-10-360L-S	TI-6085L-10-360L
10mm	380mm	TI-6085L-10-380L-S	TI-6085L-10-380L
10mm	400mm	TI-6085L-10-400L-S	TI-6085L-10-400L
10mm	420mm	TI-6085L-10-420L-S	TI-6085L-10-420L
11mm	300mm	TI-6085L-11-300L-S	TI-6085L-11-300L
11mm	320mm	TI-6085L-11-320L-S	TI-6085L-11-320L
11mm	340mm	TI-6085L-11-340L-S	TI-6085L-11-340L
11mm	360mm	TI-6085L-11-360L-S	TI-6085L-11-360L
11mm	380mm	TI-6085L-11-380L-S	TI-6085L-11-380L
11mm	400mm	TI-6085L-11-400L-S	TI-6085L-11-400L
11mm	420mm	TI-6085L-11-420L-S	TI-6085L-11-420L
12mm	300mm	TI-6085L-12-300L-S	TI-6085L-12-300L
12mm	320mm	TI-6085L-12-320L-S	TI-6085L-12-320L
12mm	340mm	TI-6085L-12-340L-S	TI-6085L-12-340L
12mm	360mm	TI-6085L-12-360L-S	TI-6085L-12-360L
12mm	380mm	TI-6085L-12-380L-S	TI-6085L-12-380L
12mm	400mm	TI-6085L-12-400L-S	TI-6085L-12-400L
12mm	420mm	TI-6085L-12-420L-S	TI-6085L-12-420L

Long



Left

130° AJAX Advance Nail, Long, Right

STERILE



Dia	Length	Sterile	Non-Sterile
9mm	300mm	TI-6085L-09-300R-S	TI-6085L-09-300R
9mm	320mm	TI-6085L-09-320R-S	TI-6085L-09-320R
9mm	340mm	TI-6085L-09-340R-S	TI-6085L-09-340R
9mm	360mm	TI-6085L-09-360R-S	TI-6085L-09-360R
9mm	380mm	TI-6085L-09-380R-S	TI-6085L-09-380R
9mm	400mm	TI-6085L-09-400R-S	TI-6085L-09-400R
9mm	420mm	TI-6085L-09-420R-S	TI-6085L-09-420R
10mm	300mm	TI-6085L-10-300R-S	TI-6085L-10-300R
10mm	320mm	TI-6085L-10-320R-S	TI-6085L-10-320R
10mm	340mm	TI-6085L-10-340R-S	TI-6085L-10-340R
10mm	360mm	TI-6085L-10-360R-S	TI-6085L-10-360R
10mm	380mm	TI-6085L-10-380R-S	TI-6085L-10-380R
10mm	400mm	TI-6085L-10-400R-S	TI-6085L-10-400R
10mm	420mm	TI-6085L-10-420R-S	TI-6085L-10-420R
11mm	300mm	TI-6085L-11-300R-S	TI-6085L-11-300R
11mm	320mm	TI-6085L-11-320R-S	TI-6085L-11-320R
11mm	340mm	TI-6085L-11-340R-S	TI-6085L-11-340R
11mm	360mm	TI-6085L-11-360R-S	TI-6085L-11-360R
11mm	380mm	TI-6085L-11-380R-S	TI-6085L-11-380R
11mm	400mm	TI-6085L-11-400R-S	TI-6085L-11-400R
11mm	420mm	TI-6085L-11-420R-S	TI-6085L-11-420R
12mm	300mm	TI-6085L-12-300R-S	TI-6085L-12-300R
12mm	320mm	TI-6085L-12-320R-S	TI-6085L-12-320R
12mm	340mm	TI-6085L-12-340R-S	TI-6085L-12-340R
12mm	360mm	TI-6085L-12-360R-S	TI-6085L-12-360R
12mm	380mm	TI-6085L-12-380R-S	TI-6085L-12-380R
12mm	400mm	TI-6085L-12-400R-S	TI-6085L-12-400R
12mm	420mm	TI-6085L-12-420R-S	TI-6085L-12-420R

Long



Right



SCREWS




4.9mm Locking Bolt for AJAX Nail, Self-Tapping

STERILE



Length	Sterile	Non-Sterile
26mm	TI-6085-01-026-S	TI-6085-01-026
28mm	TI-6085-01-028-S	TI-6085-01-028
30mm	TI-6085-01-030-S	TI-6085-01-030
32mm	TI-6085-01-032-S	TI-6085-01-032
34mm	TI-6085-01-034-S	TI-6085-01-034
36mm	TI-6085-01-036-S	TI-6085-01-036
38mm	TI-6085-01-038-S	TI-6085-01-038
40mm	TI-6085-01-040-S	TI-6085-01-040
42mm	TI-6085-01-042-S	TI-6085-01-042
44mm	TI-6085-01-044-S	TI-6085-01-044
46mm	TI-6085-01-046-S	TI-6085-01-046
48mm	TI-6085-01-048-S	TI-6085-01-048
50mm	TI-6085-01-050-S	TI-6085-01-050
52mm	TI-6085-01-052-S	TI-6085-01-052
54mm	TI-6085-01-054-S	TI-6085-01-054
56mm	TI-6085-01-056-S	TI-6085-01-056
58mm	TI-6085-01-058-S	TI-6085-01-058
60mm	TI-6085-01-060-S	TI-6085-01-060
62mm	TI-6085-01-062-S	TI-6085-01-062
64mm	TI-6085-01-064-S	TI-6085-01-064
66mm	TI-6085-01-066-S	TI-6085-01-066
68mm	TI-6085-01-068-S	TI-6085-01-068
70mm	TI-6085-01-070-S	TI-6085-01-070
72mm	TI-6085-01-072-S	TI-6085-01-072
74mm	TI-6085-01-074-S	TI-6085-01-074
76mm	TI-6085-01-076-S	TI-6085-01-076
78mm	TI-6085-01-078-S	TI-6085-01-078
80mm	TI-6085-01-080-S	TI-6085-01-080
85mm	TI-6085-01-085-S	TI-6085-01-085
90mm	TI-6085-01-090-S	TI-6085-01-090
95mm	TI-6085-01-095-S	TI-6085-01-095
100mm	TI-6085-01-100-S	TI-6085-01-100

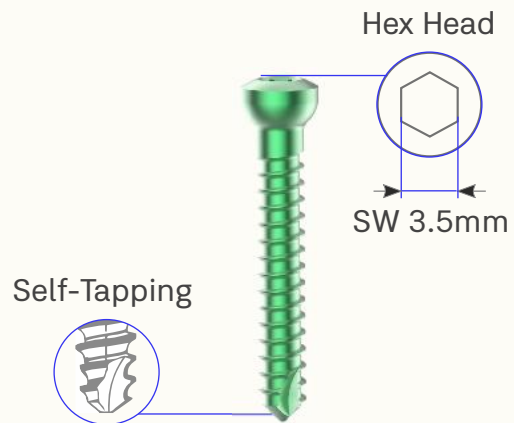

AJAX Blade, Titanium

STERILE



Length	Sterile	Non-Sterile
75mm	TI-6085-02-075-S	TI-6085-02-075
80mm	TI-6085-02-080-S	TI-6085-02-080
85mm	TI-6085-02-085-S	TI-6085-02-085
90mm	TI-6085-02-090-S	TI-6085-02-090
95mm	TI-6085-02-095-S	TI-6085-02-095
100mm	TI-6085-02-100-S	TI-6085-02-100
105mm	TI-6085-02-105-S	TI-6085-02-105
110mm	TI-6085-02-110-S	TI-6085-02-110
115mm	TI-6085-02-115-S	TI-6085-02-115
120mm	TI-6085-02-120-S	TI-6085-02-120



AJAX Nail End Cap (For AJAX Blade), Titanium

STERILE



Length	Sterile	Non-Sterile
0mm	TI-6085-03-000-S	TI-6085-03-000
5mm	TI-6085-03-005-S	TI-6085-03-005
10mm	TI-6085-03-010-S	TI-6085-03-010
15mm	TI-6085-03-015-S	TI-6085-03-015





IMPLANTS BOX




7-164	Implant Box for AJAX Nail with Blade (Left)
-------	---






Tray 1

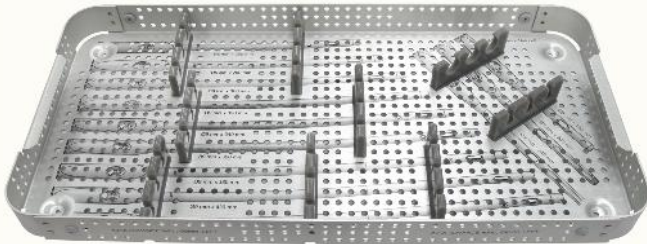

Dia	Type	Item	Length	Unit
Ø9mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø9mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement


Tray 2

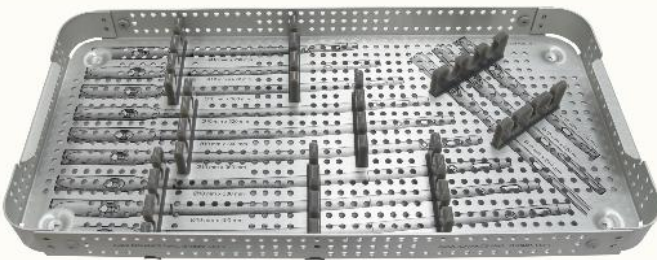

Dia	Type	Item	Length	Unit
Ø10mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø10mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement


Tray 3

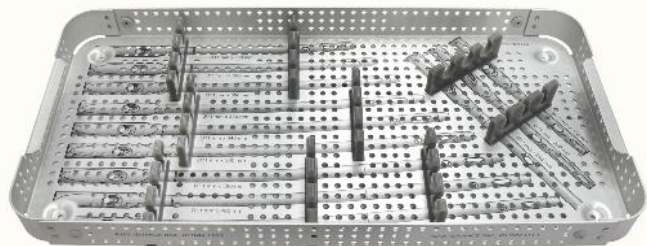

Dia	Type	Item	Length	Unit
Ø11mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø11mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement

Tray 4

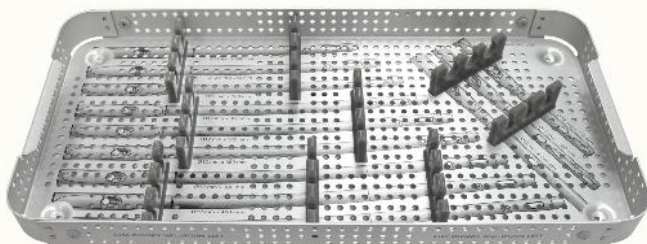

Dia	Type	Item	Length	Unit
Ø12mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø12mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement



Ø4.9mm Locking Bolt for AJAX Nail, Self-Tapping

Dia	Length	Unit
Ø4.9mm	26mm to 100mm	3pcs

Stainless Steel or Titanium Implants can be placed as per requirement



AJAX Blade

Length	Unit
75mm to 120mm	1pc

AJAX Nail End Cap (For Ajax Blade)

Length	Unit
0mm to 15mm	2pcs


Stainless Steel or Titanium Implants can be placed as per requirement

1-015-01R	Implant Box for AJAX Nail with Blade (Right)
------------------	--






Tray 1

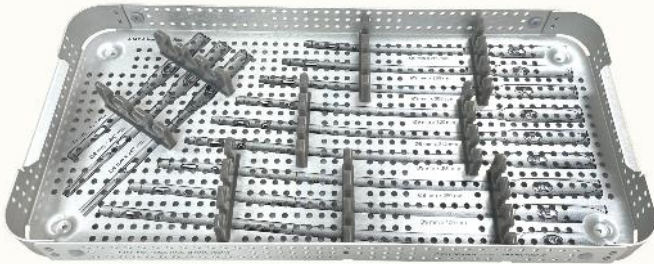

Dia	Type	Item	Length	Unit
Ø9mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø9mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement


Tray 2

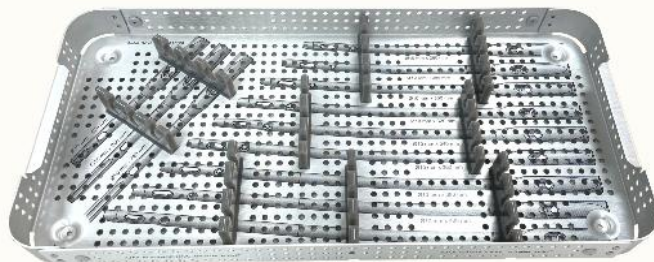

Dia	Type	Item	Length	Unit
Ø10mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø10mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement


Tray 3

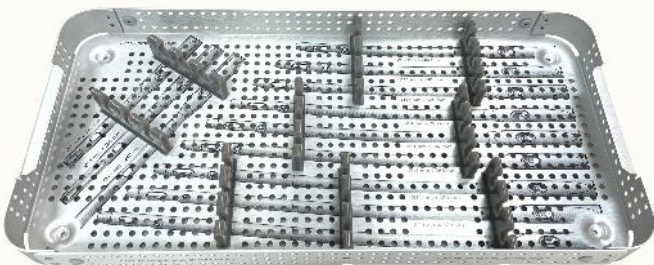

Dia	Type	Item	Length	Unit
Ø11mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø11mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement

Tray 4

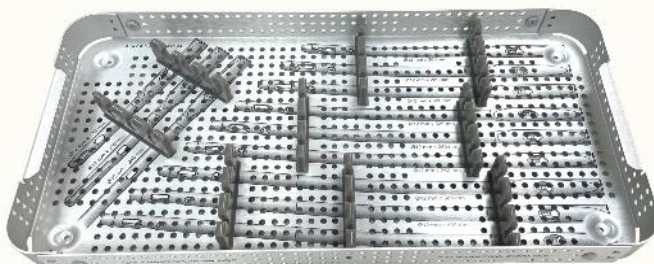

Dia	Type	Item	Length	Unit
Ø12mm	Short	130° AJAX Advance Nail	170mm to 240mm	1pc
Ø12mm	Long	130° AJAX Advance Nail	300mm to 420mm	1pc

Stainless Steel or Titanium Implants can be placed as per requirement



Ø4.9mm Locking Bolt for AJAX Nail, Self-Tapping

Dia	Length	Unit
Ø4.9mm	26mm to 100mm	3pcs

Stainless Steel or Titanium Implants can be placed as per requirement



AJAX Blade

Length	Unit
75mm to 120mm	1pc

AJAX Nail End Cap (For Ajax Blade)

Length	Unit
0mm to 15mm	2pcs

Stainless Steel or Titanium Implants can be placed as per requirement



INSTRUMENTS

Short Distal Targeting Device



AJAX-001

Radiographic Ruler for AJAX Advance Nail



AJAX-002

T-Handle Quick Coupling (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)



AJAX-003

Entry Reamer $\varnothing 17\text{mm}$ for AJAX Advance Nail



AJAX-004

Cannulated Awl for AJAX Advance Nail



AJAX-005

Tissue Protector for AJAX Advance Nail



AJAX-006

Nail Holding Bolt for AJAX Advance Nail



AJAX-007

Key for AJAX Blade



AJAX-008

Slotted Hammer for AJAX Advance Nail



AJAX-009

Distal Targeting Device for Short AJAX Advance Nail



AJAX-009-01

Bolt for Distal Targeting Device for Short AJAX Advance Nail



AJAX-010

Guide Wire with Threaded Tip $\varnothing 3.2\text{mm}$ x Thread Length 10mm X Length 400mm



AJAX-011

Guide Wire $\varnothing 3.2\text{mm}$ x Length 400mm



AJAX-012

Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Advance Nail



AJAX-013

Reamer Ø10.5mm for AJAX Blade



AJAX-014

Extraction Screw for AJAX Blade



AJAX-015

Impactor for AJAX Blade



AJAX-016

Hammer Guide for Slotted Hammer M8




AJAX-017

Combination Wrench 11mm for AJAX Advance Nail



AJAX-018-130

130° Proximal Targeting Device for AJAX Advance Nail

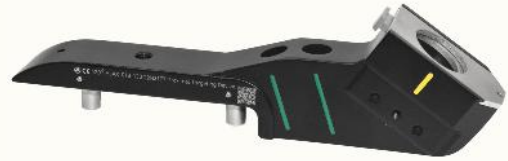

AJAX-018-130-01

Bolt for 130° Proximal Targeting Device for AJAX Advance Nail



AJAX-019

Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm x Length 300mm for AJAX Advance Nail



AJAX-020

Stopper for Ø4.0mm Drill Bit - AJAX Advance Nail



AJAX-021

Allen Key, Hex 3mm for AJAX Advance Nail



AJAX-022

Screwdriver, Hex 3.5mm for AJAX Advance Nail



AJAX-024

Cleaning Stylet Ø2.8mm for AJAX Advance Nail



AJAX-025

Guide Wire Introducer for AJAX Advance Nail



AJAX-026

Entry Reamer Protection Sleeve 20/17mm for AJAX Advance Nail



AJAX-027

Entry Reamer Guide Wire Sleeve 17/3.2mm for AJAX Advance Nail



AJAX-028

Aiming Jig For Anti Rotation Wire



AJAX-029

Flexible Reaming Shaft Jacob End for AJAX Advance Nail



Reamer Head

Code	Dia
AJAX-030	Ø8.5mm
AJAX-031	Ø9.0mm
AJAX-032	Ø9.5mm
AJAX-033	Ø10.0mm
AJAX-034	Ø10.5mm
AJAX-035	Ø11.0mm
AJAX-036	Ø11.5mm
AJAX-037	Ø12.0mm
7-026-22	Ø12.5mm
7-026-23	Ø13.5mm



AJAX-038

Quick Coupling Handle (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)



AJAX-039

Compression Instrument for AJAX Blade



AJAX-040

Insertion Handle for AJAX Advance Nail



AJAX-041

Protection Sleeve for AJAX Advance Nail



AJAX-042

Cannulated Trocar Sleeve Ø10.9/3.2mm for AJAX Blade



AJAX-043

Trocar Ø3.2mm for AJAX Blade



AJAX-044

Insertion Handle Connector for AJAX Advance Nail



AJAX-045

Spherical Head Screwdriver, Hex 4.0mm for AJAX Advance Nail



AJAX-046

Aiming Jig Drill Sleeve for Anti-Rotation Wire



AJAX-047

Protection Sleeve Ø11/8mm for AJAX Advance Nail



AJAX-048

Drill Sleeve Ø8/4mm for AJAX Advance Nail



AJAX-049

Trocar Ø4.0mm for AJAX Advance Nail



AJAX-050

Direct Measuring Device for Guide Wire - AJAX Advance Nail



AJAX-051

Pin Wrench Ø4.0mm for AJAX Advance Nail



AJAX-053

Measuring Device for Locking Bolt upto 100mm - AJAX Advance Nail



AJAX-054

Reduction Instrument for AJAX Advance Nail



AJAX-055

Driver for Nail Holding Bolt - AJAX Advance Nail




7-026-15

Guide Wire Without Ball Tip Ø2.5mm x Length 1000mm - Nitinol, for AJAX Advance Nail



7-026-18

Guide Wire With Ball Tip Ø2.5/3.65mm x Length 1000mm - Nitinol, for AJAX Advance Nail

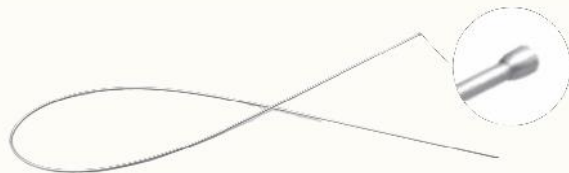

7-026-19

Curved Awl- Ø9.5-Ø5.6mm for AJAX Advanced Nail



7-026-20

T-Handle Screwdriver With Wrench Rod, Hex 4mm - AJAX Nail End Cap



7-026-21

7-026-21 Long Flexible Screwdriver With Flexible Holder, Hex 4.0mm - AJAX Nail End Cap



AJAX-057

Instrument Trays for AJAX Advance Nailing Instrument Set (Short Targeting Device)



AJAX-056

Container with PPSU Lid for AJAX Advance Nailing Instrument Set (Short Targeting Device)



AJAX-000 : AJAX Advance Nailing Instrument Set (Short Targeting device)





ORDER INFORMATION

AJAX-000	AJAX Advance Nailing Instrument Set (Short Targeting device)
----------	--

Code	Set Consisting Of	Unit
AJAX-001	Radiographic Ruler for AJAX Advance Nail	1
AJAX-002	T-Handle Quick Coupling (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)	1
AJAX-003	Entry Reamer Ø17mm for AJAX Advance Nail	1
AJAX-004	Cannulated Awl for AJAX Advance Nail	1
AJAX-005	Tissue Protector for AJAX Advance Nail	1
AJAX-006	Nail Holding Bolt for AJAX Advance Nail	2
AJAX-007	Key for AJAX Blade	1
AJAX-008	Slotted Hammer for AJAX Advance Nail	1
AJAX-009	Distal Targeting Device for Short AJAX Advance Nail	1
AJAX-009-01	Bolt for Distal Targeting Device for Short AJAX Advance Nail	1
AJAX-010	Guide Wire with Threaded Tip Ø3.2mm x Thread Length 10mm X Length 400mm	3
AJAX-011	Guide Wire Ø3.2mm x Length 400mm	3
AJAX-012	Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Advance Nail	1
AJAX-013	Reamer Ø10.5mm for AJAX Blade	1
AJAX-014	Extraction Screw for AJAX Blade	1
AJAX-015	Impactor for AJAX Blade	1
AJAX-016	Hammer Guide for Slotted Hammer M8	1
AJAX-017	Combination Wrench 11mm for AJAX Advance Nail	1
AJAX-018-130	130° Proximal Targeting Device for AJAX Advance Nail	1
AJAX-018-130-01	Bolt for 130° Proximal Targeting Device for AJAX Advance Nail	1
AJAX-019	Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm x Length 300mm for AJAX Advance Nail	2
AJAX-020	Stopper for Ø4.0mm Drill Bit - AJAX Advance Nail	1
AJAX-021	Allen Key, Hex 3mm for AJAX Advance Nail	1
AJAX-022	Screwdriver, Hex 3.5mm for AJAX Advance Nail	1
AJAX-024	Cleaning Stylet Ø2.8mm for AJAX Advance Nail	1
AJAX-025	Guide Wire Introducer for AJAX Advance Nail	1
AJAX-026	Entry Reamer Protection Sleeve 20/17mm for AJAX Advance Nail	1
AJAX-027	Entry Reamer Guide Wire Sleeve 17/3.2mm for AJAX Advance Nail	1
AJAX-028	Aiming Jig For Anti Rotation Wire	1
AJAX-029	Flexible Reaming Shaft Jacob End for AJAX Advance Nail	2
AJAX-030	Reamer Head Ø8.5mm for AJAX Advance Nail	1
AJAX-031	Reamer Head Ø9.0mm for AJAX Advance Nail	1
AJAX-032	Reamer Head Ø9.5mm for AJAX Advance Nail	1
AJAX-033	Reamer Head Ø10.0mm for AJAX Advance Nail	1
AJAX-034	Reamer Head Ø10.5mm for AJAX Advance Nail	1



ORDER INFORMATION

AJAX-000	AJAX Advance Nailing Instrument Set (Short Targeting device)
----------	--

Code	Set Consisting Of	Unit
AJAX-035	Reamer Head Ø11.0mm for AJAX Advance Nail	1
AJAX-036	Reamer Head Ø11.5mm for AJAX Advance Nail	1
AJAX-037	Reamer Head Ø12.0mm for AJAX Advance Nail	1
7-026-22	Reamer Head Ø12.5mm for AJAX Advance Nail	1
7-026-23	Reamer Head Ø13.0mm for AJAX Advance Nail	1
AJAX-038	Quick Coupling Handle (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)	1
AJAX-039	Compression Instrument for AJAX Blade	1
AJAX-040	Insertion Handle for AJAX Advance Nail	1
AJAX-041	Protection Sleeve for AJAX Advance Nail	1
AJAX-042	Cannulated Trocar Sleeve Ø10.9/3.2mm for AJAX Blade	1
AJAX-043	Trocar Ø3.2mm for AJAX Blade	1
AJAX-044	Insertion Handle Connector for AJAX Advance Nail	1
AJAX-045	Spherical Head Screwdriver, Hex 4.0mm for AJAX Advance Nail	1
AJAX-046	Aiming Jig Drill Sleeve for Anti-Rotation Wire	2
AJAX-047	Protection Sleeve Ø11/8mm for AJAX Advance Nail	2
AJAX-048	Drill Sleeve Ø8/4mm for AJAX Advance Nail	2
AJAX-049	Trocar Ø4.0mm for AJAX Advance Nail	2
AJAX-050	Direct Measuring Device for Guide Wire - AJAX Advance Nail	1
AJAX-051	Pin Wrench Ø4.0mm for AJAX Advance Nail	1
AJAX-053	Measuring Device for Locking Bolt upto 100mm - AJAX Advance Nail	1
AJAX-054	Reduction Instrument for AJAX Advance Nail	1
AJAX-055	Driver for Nail Holding Bolt - AJAX Advance Nail	1
7-026-15	Guide Wire Without Ball Tip Ø2.5mm x Length 1000mm - Nitinol, for AJAX Advance Nail	1
7-026-18	Guide Wire With Ball Tip Ø2.5/3.65mm x Length 1000mm - Nitinol, for AJAX Advance Nail	1
AJAX-057	Instrument Trays for AJAX Advance Nailing Instrument Set (Short Targeting Device)	2
7-026-20	T-Handle Screwdriver With Wrench Rod, Hex 4mm - AJAX Nail End Cap	1
7-026-21	Long Flexible Screwdriver With Flexible Holder, Hex 4.0mm - AJAX Nail End Cap	1
7-026-19	Curved Awl- Ø9.5-Ø5.6mm for AJAX Advanced Nail	1
AJAX-056	Container with PPSU Lid for AJAX Advance Nailing Instrument Set (Short Targeting Device)	1



INSTRUMENTS

Distal Targeting Device



AJAX-001

Radiographic Ruler for AJAX Advance Nail



AJAX-002

T-Handle Quick Coupling (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)



AJAX-003

Entry Reamer \varnothing 17mm for AJAX Advance Nail



AJAX-004

Cannulated Awl for AJAX Advance Nail



AJAX-005

Tissue Protector for AJAX Advance Nail



AJAX-006

Nail Holding Bolt for AJAX Advance Nail



AJAX-007

Key for AJAX Blade



AJAX-008

Slotted Hammer for AJAX Advance Nail



AJAX-009

Distal Targeting Device for Short AJAX Advance Nail



AJAX-009-01

Bolt for Distal Targeting Device for Short AJAX Advance Nail



AJAX-010

Guide Wire with Threaded Tip $\varnothing 3.2\text{mm}$ x Thread Length 10mm X Length 400mm



AJAX-011

Guide Wire $\varnothing 3.2\text{mm}$ x Length 400mm



AJAX-012

Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Advance Nail



AJAX-013

Reamer Ø10.5mm for AJAX Blade



AJAX-014

Extraction Screw for AJAX Blade



AJAX-015

Impactor for AJAX Blade



AJAX-016

Hammer Guide for Slotted Hammer M8



AJAX-017

Combination Wrench 11mm for AJAX Advance Nail



AJAX-018-130

130° Proximal Targeting Device for AJAX Advance Nail



AJAX-018-130-01

Bolt for 130° Proximal Targeting Device for AJAX Advance Nail



AJAX-019

Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm x Length 300mm for AJAX Advance Nail



AJAX-020

Stopper for Ø4.0mm Drill Bit - AJAX Advance Nail



AJAX-021

Allen Key, Hex 3mm for AJAX Advance Nail



AJAX-022

Screwdriver, Hex 3.5mm for AJAX Advance Nail



AJAX-024

Cleaning Stylet Ø2.8mm for AJAX Advance Nail



AJAX-025

Guide Wire Introducer for AJAX Advance Nail




AJAX-026

Entry Reamer Protection Sleeve 20/17mm for AJAX Advance Nail




AJAX-027

Entry Reamer Guide Wire Sleeve 17/3.2mm for AJAX Advance Nail

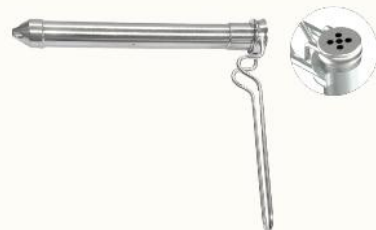

AJAX-028

Aiming Jig For Anti Rotation Wire

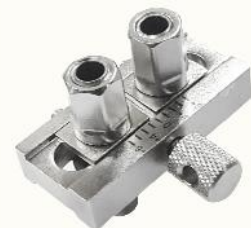

AJAX-029

Flexible Reaming Shaft Jacob End for AJAX Advance Nail



Reamer Head

Code	Dia
AJAX-030	Ø8.5mm
AJAX-031	Ø9.0mm
AJAX-032	Ø9.5mm
AJAX-033	Ø10.0mm
AJAX-034	Ø10.5mm
AJAX-035	Ø11.0mm
AJAX-036	Ø11.5mm
AJAX-037	Ø12.0mm
7-026-22	Ø12.5mm
7-026-23	Ø13.5mm



AJAX-038

Quick Coupling Handle (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)



AJAX-039

Compression Instrument for AJAX Blade



AJAX-040

Insertion Handle for AJAX Advance Nail



AJAX-041

Protection Sleeve for AJAX Advance Nail



AJAX-042

Cannulated Trocar Sleeve Ø10.9/3.2mm for AJAX Blade



AJAX-043

Trocar Ø3.2mm for AJAX Blade



AJAX-044

Insertion Handle Connector for AJAX Advance Nail



AJAX-045

Spherical Head Screwdriver, Hex 4.0mm for AJAX Advance Nail



AJAX-046

Aiming Jig Drill Sleeve for Anti-Rotation Wire



AJAX-047

Protection Sleeve Ø11/8mm for AJAX Advance Nail




AJAX-048

Drill Sleeve Ø8/4mm for AJAX Advance Nail



AJAX-049

Trocar Ø4.0mm for AJAX Advance Nail

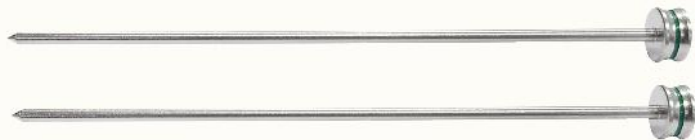

AJAX-050

Direct Measuring Device for Guide Wire - AJAX Advance Nail



AJAX-051

Pin Wrench Ø4.0mm for AJAX Advance Nail



AJAX-053

Measuring Device for Locking Bolt upto 100mm - AJAX Advance Nail



AJAX-054

Reduction Instrument for AJAX Advance Nail



AJAX-055

Driver for Nail Holding Bolt - AJAX Advance Nail



7-026-01

Guide Block for AJAX Advance Nail - Right



7-026-02

Guide Block for AJAX Advance Nail - Left



7-026-03

Locking Bolt for Insertion Handle and Distal Guide Bar - AJAX Advance Nail



7-026-04

Locking Bolt for Distal Targeting Device - AJAX Advance Nail



7-026-05

Locking Bolt for Distal Aiming Bar- AJAX Advance Nail



7-026-06

Positioning Rod for AJAX Advance Nail



7-026-07

Ø5.2mm T-Handle Drill Bit with Flat End for AJAX Advance Nail



7-026-08

Drill Bit Plain Shank/Jacob Chuck End Ø5.2mm x Length 250mm for Long AJAX Advance Nail



7-026-09

Distal Guide Bar for AJAX Advance Nail



7-026-10

Distal Targeting Device for Long AJAX Advance Nail



7-026-11

Distal Aiming Bar for Long AJAX Advance Nail



7-026-12

Trocar Ø5.1mm for AJAX Advance Nail



7-026-13

Positioning Rod Drill Sleeve for AJAX Advance Nail



7-026-14

Allen Key, Hex 5mm for AJAX Advance Nail



7-026-15

Guide Wire Without Ball Tip Ø2.5mm x Length 1000mm - Nitinol, for AJAX Advance Nail



7-026-18

Guide Wire With Ball Tip Ø2.5/3.65mm x Length 1000mm - Nitinol, for AJAX Advance Nail



7-026-19

Curved Awl- Ø9.5-Ø5.6mm for AJAX Advanced Nail



7-026-20

T-Handle Screwdriver With Wrench Rod, Hex 4mm - AJAX Nail End Cap



7-026-21

Long Flexible Screwdriver With Flexible Holder, Hex 4.0mm - AJAX Nail End Cap




7-026-16

Instrument Trays for AJAX Advance Nailing Instrument Set (Distal Targeting Device)



7-026-17

Container with PPSU Lid for AJAX Advance Nailing Instrument set (Distal Targeting Device)

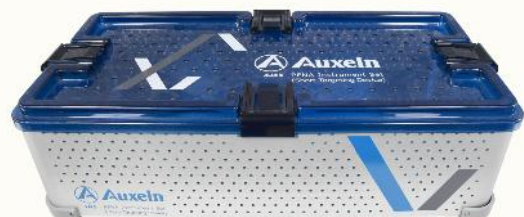

7-026 : AJAX Advance Nailing Instrument Set (Distal Targeting Device)





ORDER INFORMATION

7-026

AJAX Advance Nailing Instrument Set (Distal Targeting Device)

Code	Set Consisting Of	Unit
AJAX-001	Radiographic Ruler for AJAX Advance Nail	1
AJAX-002	T-Handle Quick Coupling (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)	1
AJAX-003	Entry Reamer Ø17mm for AJAX Advance Nail	1
AJAX-004	Cannulated Awl for AJAX Advance Nail	1
AJAX-005	Tissue Protector for AJAX Advance Nail	1
AJAX-006	Nail Holding Bolt for AJAX Advance Nail	2
AJAX-007	Key for AJAX Blade	1
AJAX-008	Slotted Hammer for AJAX Advance Nail	1
AJAX-009	Distal Targeting Device for Short AJAX Advance Nail	1
AJAX-009-01	Bolt for Distal Targeting Device for Short AJAX Advance Nail	1
AJAX-010	Guide Wire with Threaded Tip Ø3.2mm x Thread Length 10mm X Length 400mm	3
AJAX-011	Guide Wire Ø3.2mm x Length 400mm	3
AJAX-012	Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Advance Nail	1
AJAX-013	Reamer Ø10.5mm for AJAX Blade	1
AJAX-014	Extraction Screw for AJAX Blade	1
AJAX-015	Impactor for AJAX Blade	1
AJAX-016	Hammer Guide for Slotted Hammer M8	1
AJAX-017	Combination Wrench 11mm for AJAX Advance Nail	1
AJAX-018-130	130° Proximal Targeting Device for AJAX Advance Nail	1
AJAX-018-130-01	Bolt for 130° Proximal Targeting Device for AJAX Advance Nail	1
AJAX-019	Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm x Length 300mm for AJAX Advance Nail	2
AJAX-020	Stopper for Ø4.0mm Drill Bit - AJAX Advance Nail	1
AJAX-021	Allen Key, Hex 3mm for AJAX Advance Nail	1
AJAX-022	Screwdriver, Hex 3.5mm for AJAX Advance Nail	1
AJAX-024	Cleaning Stylet Ø2.8mm for AJAX Advance Nail	1
AJAX-025	Guide Wire Introducer for AJAX Advance Nail	1
AJAX-026	Entry Reamer Protection Sleeve 20/17mm for AJAX Advance Nail	1
AJAX-027	Entry Reamer Guide Wire Sleeve 17/3.2mm for AJAX Advance Nail	1
AJAX-028	Aiming Jig For Anti Rotation Wire	1
AJAX-029	Flexible Reaming Shaft Jacob End for AJAX Advance Nail	2
AJAX-030	Reamer Head Ø8.5mm for AJAX Advance Nail	1
AJAX-031	Reamer Head Ø9.0mm for AJAX Advance Nail	1
AJAX-032	Reamer Head Ø9.5mm for AJAX Advance Nail	1
AJAX-033	Reamer Head Ø10.0mm for AJAX Advance Nail	1
AJAX-034	Reamer Head Ø10.5mm for AJAX Advance Nail	1
AJAX-035	Reamer Head Ø11.0mm for AJAX Advance Nail	1
AJAX-036	Reamer Head Ø11.5mm for AJAX Advance Nail	1
AJAX-037	Reamer Head Ø12.0mm for AJAX Advance Nail	1
7-026-22	Reamer Head Ø12.5mm for AJAX Advance Nail	1
7-026-23	Reamer Head Ø13.0mm for AJAX Advance Nail	1



ORDER INFORMATION

7-026	AJAX Advance Nailing Instrument Set (Distal Targeting Device)
-------	--

Code	Set Consisting Of	Unit
AJAX-038	Quick Coupling Handle (For Flexible Shaft/Entry Reamer/Proximal Cortex Cannulated Drill/ Reamer For AJAX Blade)	1
AJAX-039	Compression Instrument for AJAX Blade	1
AJAX-040	Insertion Handle for AJAX Advance Nail	1
AJAX-041	Protection Sleeve for AJAX Advance Nail	1
AJAX-042	Cannulated Trocar Sleeve Ø10.9/3.2mm for AJAX Blade	1
AJAX-043	Trocar Ø3.2mm for AJAX Blade	1
AJAX-044	Insertion Handle Connector for AJAX Advance Nail	1
AJAX-045	Spherical Head Screwdriver, Hex 4.0mm for AJAX Advance Nail	1
AJAX-046	Aiming Jig Drill Sleeve for Anti-Rotation Wire	2
AJAX-047	Protection Sleeve Ø11/8mm for AJAX Advance Nail	2
AJAX-048	Drill Sleeve Ø8/4mm for AJAX Advance Nail	2
AJAX-049	Trocar Ø4.0mm for AJAX Advance Nail	2
AJAX-050	Direct Measuring Device for Guide Wire - AJAX Advance Nail	1
AJAX-051	Pin Wrench Ø4.0mm for AJAX Advance Nail	1
AJAX-053	Measuring Device for Locking Bolt upto 100mm - AJAX Advance Nail	1
AJAX-054	Reduction Instrument for AJAX Advance Nail	1
AJAX-055	Driver for Nail Holding Bolt - AJAX Advance Nail	1
7-026-01	Guide Block for AJAX Advance Nail - Right	1
7-026-02	Guide Block for AJAX Advance Nail - Left	1
7-026-03	Locking Bolt for Insertion Handle and Distal Guide Bar - AJAX Advance Nail	1
7-026-04	Locking Bolt for Distal Targeting Device - AJAX Advance Nail	1
7-026-05	Locking Bolt for Distal Aiming Bar- AJAX Advance Nail	1
7-026-06	Positioning Rod for AJAX Advance Nail	1
7-026-07	Ø5.2mm T-Handle Drill Bit with Flat End for AJAX Advance Nail	1
7-026-08	Drill Bit Plain Shank/Jacob Chuck End Ø5.2mm x Length 250mm for Long AJAX Advance Nail	1
7-026-09	Distal Guide Bar for AJAX Advance Nail	1
7-026-10	Distal Targeting Device for Long AJAX Advance Nail	1
7-026-11	Distal Aiming Bar for Long AJAX Advance Nail	1
7-026-12	Trocar Ø5.1mm for AJAX Advance Nail	1
7-026-13	Positioning Rod Drill Sleeve for AJAX Advance Nail	1
7-026-14	Allen Key, Hex 5mm for AJAX Advance Nail	1
7-026-18	Guide Wire With Ball Tip Ø2.5/3.65mm x Length 1000mm - Nitinol, for AJAX Advance Nail	1
7-026-15	Guide Wire Without Ball Tip Ø2.5mm x Length 1000mm - Nitinol, for AJAX Advance Nail	1
7-026-19	Curved Awl- Ø9.5-Ø5.6mm for AJAX Advanced Nail	1
7-026-20	T-Handle Screwdriver With Wrench Rod, Hex 4mm - AJAX Nail End Cap	1
7-026-21	Long Flexible Screwdriver With Flexible Holder, Hex 4.0mm - AJAX Nail End Cap	1
7-026-16	Instrument Trays for AJAX Advance Nailing Instrument Set (Distal Targeting Device)	4
7-026-17	Container with PPSU Lid for AJAX Advance Nailing Instrument set (Distal Targeting Device)	1



CEPHALIC SCREW SYSTEM





AJAX Nail End Cap (for Cephalic Screw)

STERILE



Length	Sterile	Non-Sterile
0mm	TI-6085-04-000-S	TI-6085-04-000
5mm	TI-6085-04-005-S	TI-6085-04-005
10mm	TI-6085-04-010-S	TI-6085-04-010
15mm	TI-6085-04-015-S	TI-6085-04-015



Inner Screw for AJAX Cephalic Screw

STERILE



Sterile	Non-Sterile
TI-6085-05-S	TI-6085-05



AJAX Cephalic Screw

STERILE



Length	Sterile	Non-Sterile
70mm	TI-6085-06-070-S	TI-6085-06-070
75mm	TI-6085-06-075-S	TI-6085-06-075
80mm	TI-6085-06-080-S	TI-6085-06-080
85mm	TI-6085-06-085-S	TI-6085-06-085
90mm	TI-6085-06-090-S	TI-6085-06-090
95mm	TI-6085-06-095-S	TI-6085-06-095
100mm	TI-6085-06-100-S	TI-6085-06-100
105mm	TI-6085-06-105-S	TI-6085-06-105
110mm	TI-6085-06-110-S	TI-6085-06-110
115mm	TI-6085-06-115-S	TI-6085-06-115
120mm	TI-6085-06-120-S	TI-6085-06-120



End Cap for AJAX Cephalic Screw

STERILE



Length	Sterile	Non-Sterile
0mm	TI-6085-06-00-S	TI-6085-06-00






1-015-02L Implant Box for AJAX Nail with Cephalic Screw (Left)

1-015-02R Implant Box for AJAX Nail with Cephalic Screw (Right)



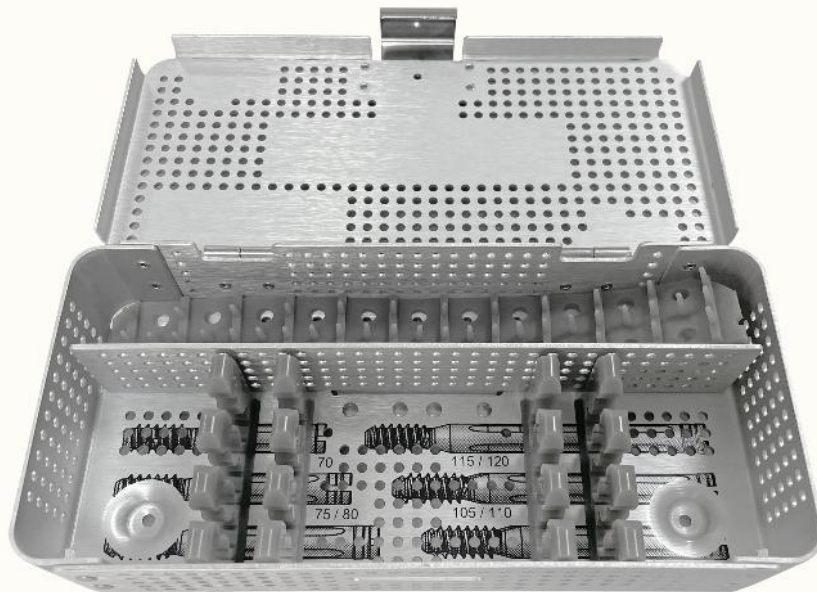

AJAX Nail End Cap (for Cephalic Screw)

Unit	Length
2pcs	0mm to 15mm

Inner Screw for AJAX Cephalic Screw

Unit
5pcs

AJAX Cephalic Screw

Unit	Length
1pc	70mm to 120mm

End Cap for AJAX Cephalic Screw

Unit
5pcs

Stainless Steel or Titanium Implants can be placed as per requirement

7-005-01

Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Cephalic Screw



7-005-02

Cephalic Screw Inserter for AJAX Cephalic Screw



7-005-03

Reamer Ø10.5mm for AJAX Cephalic Screw



7-005-04

Guide Wire Ø3.2mm x Length 400mm for AJAX Cephalic Screw



7-005-05

Protection Sleeve for AJAX Cephalic Screw



7-005-06

Cannulated Trocar Sleeve for AJAX Cephalic Screw



7-005-07

Trocar Ø3.2mm For Ajax Cephalic Screw



7-005-08

Caddy For Ajax Cephalic Screw End Cap



7-005-10

Inner Screwdriver, Hex 4.0mm




7-005-11

Inner Screw Holder, Hex 4.0mm




7-005-09

Container For Ajax Cephalic Screw Instrument Set

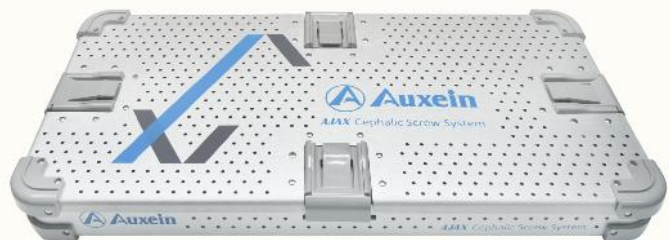

ORDER INFORMATION

7-005 : AJAX Cephalic Screw Instrument Set

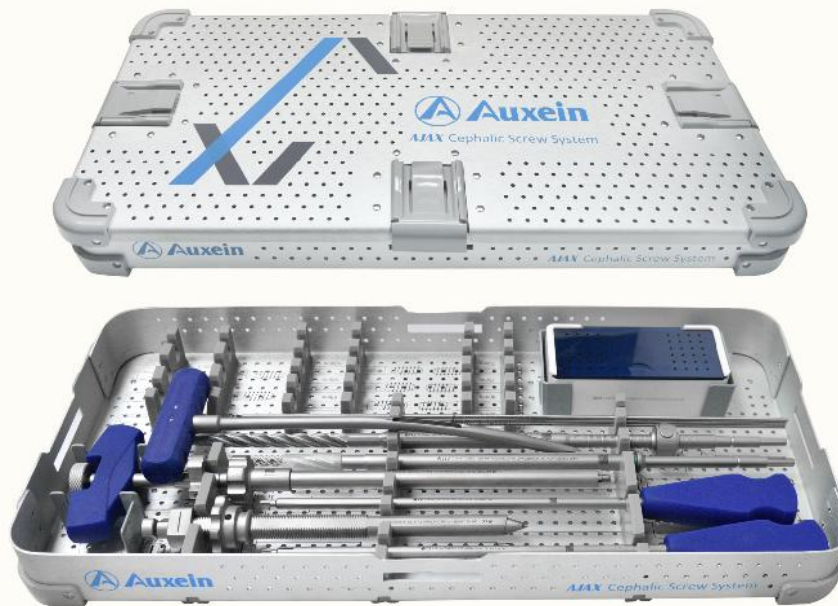

Code	Set Consisting Of	Unit
7-005-01	Proximal Cortex Cannulated Drill Ø10.5mm for AJAX Cephalic Screw	1
7-005-02	Cephalic Screw Inserter for AJAX Cephalic Screw	1
7-005-03	Reamer Ø10.5mm for AJAX Cephalic Screw	1
7-005-04	Guide Wire Ø3.2mm x Length 400mm for AJAX Cephalic Screw	3
7-005-05	Protection Sleeve for AJAX Cephalic Screw	1
7-005-06	Cannulated Trocar Sleeve for AJAX Cephalic Screw	1
7-005-07	Trocar Ø3.2mm for AJAX Cephalic Screw	1
7-005-08	Caddy for AJAX Cephalic Screw End Cap	1
7-005-10	Inner Screwdriver, Hex 4.0mm	1
7-005-11	Inner Screw Holder, Hex 4.0mm	1
7-005-09	Container for AJAX Cephalic Screw Instrument Set	1

AJAX ADVANCE NAILING SYSTEM

The content of this catalogue is intended for healthcare professionals only. All product information, illustrations, descriptions, and specifications provided here are for general informational purposes and are subject to change without notice. Auxein Medical Pvt. Ltd. reserves the right to improve or discontinue products at any time without prior notification.

This document does not constitute medical advice. Surgeons and medical practitioners must exercise their own professional judgment when using any medical devices or instruments, including those provided by Auxein.

INTENDED USE

The implants and instruments in this catalogue are intended for use by qualified orthopedic trauma surgeons trained in fracture fixation and intramedullary nailing procedures. Improper selection of the implant, incorrect surgical technique, misuse, or insufficient surgical experience may result in implant failure or other complications.

REGULATORY COMPLIANCE CERTIFIED

CE Approved (EUMDR-2017/745)

USFDA 510(k)

ISO 13485:2016

CDSCO

MDSAP

INTELLECTUAL PROPERTY

All trademarks, trade names, brand names, product names, and logos mentioned in this catalogue are the property of Auxein Medical Pvt. Ltd. Unauthorized use or reproduction is strictly prohibited.

V1-2026



SCAN TO KNOW MORE

INDIA
AUXEIN MEDICAL PVT.LTD.

PLOT NO. 168-169-170, PHASE-4,
KUNDLI INDUSTRIAL AREA,
HSIIDC, SECTOR-57,
SONEPAT-131028, HARYANA
TEL: +91 99106 43638
FAX: +91 86077 70197
EMAIL: INFO@AUXEIN.COM

MÉXICO
AUXEIN MÉXICO, S.A DE CV

TEPIC 39 INT 801, COLONIA
ROMA SUR,
ALCADÍA CUAUHTÉMOC, CDMX
MÉXICO, C.P 06760
TEL: +521 55 7261 0318
EMAIL: INFO@AUXEIN.MX

USA
AUXEIN INC.

1500 NW 89TH COURT,
SUITE 107-108
DORAL, FLORIDA 33172
TEL. +1 305 395 6062
E FAX: +1 305 395 62 62
EMAIL: USOFFICE@AUXEIN.COM

DUBAI UAE
AUXEIN MEDICAL

TRADING FZCO OUD METHA
OFFICES BUILDING B BLOCK,
1ST FLOOR, OFFICE NO.26,
NEAR HEALTHCARE CITY,
PO BOX-25572
TEL. +971 438 71333
EMAIL: INFO@AUXEIN.COM

WWW.AUXEIN.COM