

# Auxein

## RETROGRADE FEMUR NAILING SYSTEM




NEXT-LEVEL CARE



## ABOUT US

Auxein Medical is an integrated, research based, orthopaedic implants & instruments manufacturing company, producing a wide range of quality, affordable generic implants, trusted by healthcare professionals and patients across geographies. It is the Company's constant endeavour to provide a wide basket of generic and our innovator products that exceed the highest expectations of customers in term of quality and safety. The company has world-class manufacturing unit established in India and serves customers in over 75 countries worldwide.

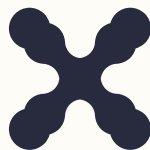

# IMPACTING HEALTHCARE GLOBALLY

Our Reach & Influence

**500+**

Employees Worldwide

**75+**

Operations in Countries

**150+**

Distributors Worldwide

**20**<sup>M+</sup>

Patients Cured

**200+**

FDA Approved Products

**250+**

Machines Worldwide

**3000+**

Innovative Orthopedic  
Products & Solutions

Auxein is dedicated to advancing medical technology through innovative orthopedic solutions, including **Trauma, Arthroscopy, and Reconstructive Procedures**, with a commitment to improving patient outcomes worldwide. Our approach integrates advanced research, precision engineering, and high-quality manufacturing to develop reliable implants and surgical instruments that address the evolving needs of modern orthopedic practice.

# OUR ACHIEVEMENTS



OVER THE YEARS



HNS

HNS

POWER ON POWER OFF

EMERGENCY STOP

HANDLE JOG


RESET

OVERRIDES

CYCLE START FEED HOLD

Machine status display showing various parameters and a 'Main Screen' section with a table of data.


Main Screen	
Feed Rate	1000
Spindle Speed	1000
Tool Wear	0.0
Chuck Load	0.000
Feed Axis	100.000
Axis Feed	100.000

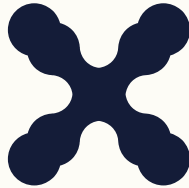

At Auxein, it is our goal to provide the best in class quality and value to every individual. Consistency is a virtue we pursue with a passion. From the procurement of raw materials to the manufacturing of our range of 1200+ orthopaedic implant products, we employ world-class systems and technology at par with the highest standards of globally recognized evaluation systems. The Retrograde Femur Nailing System is part of our orthopedic trauma portfolio designed for the intramedullary fixation of femoral fractures. Developed with input from global experts, the system provides reliable stabilization to help achieve stable fixation and accurate anatomical alignment during fracture management.



# INDEX

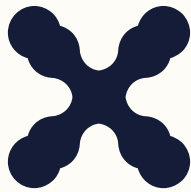

Implants .....	11-15
Implants box & Tray .....	16-19
Instruments .....	20-31
Order Information .....	32-33



# **RETROGRADE FEMUR NAILING SYSTEM**



# Retrograde Femur Nailing System

The Retrograde Femur Nailing System is designed for the intramedullary fixation of distal and diaphyseal femoral fractures using a retrograde surgical approach. Developed with input from global experts, the system incorporates anatomically contoured cannulated nails with multiple locking options to support stable fixation and maintain proper anatomical alignment of the femur. The system includes femoral nails, locking screws, end caps, and accessory components to provide versatility for the management of a wide range of femoral fracture patterns.

## Key Features

- **Anatomically Contoured Nail Design** to match the natural curvature of the femur.
- **Cannulated Nail Construction** allowing guided insertion over a guide wire for precise placement.
- **Multiple Locking Options** compatible with **5.0 mm locking screws and low-profile locking screws**.
- **Distal Femur Washer Plate** enabling additional fixation using **3.5 mm variable angle locking screws**.
- **Wide Size Range** with nail diameters **9.5 mm, 10 mm, 11 mm, and 12 mm** and lengths **180 mm to 400 mm**.

## Indications

The **Retrograde Femur Nailing System** is indicated for the treatment of:

- Distal femoral fractures
- Femoral shaft fractures
- Metaphyseal and diaphyseal femoral fractures
- Periprosthetic fractures above total knee arthroplasty (where anatomy allows)
- Non-unions or malunions of the femoral shaft
- Pathological fractures of the femur





# Retrograde Femur Nail



Dia	Length	Stainless Steel	Titanium
Ø9.5mm	180mm	1458-9.5-180	TI-1458-9.5-180
Ø9.5mm	200mm	1458-9.5-200	TI-1458-9.5-200
Ø9.5mm	300mm	1458-9.5-300	TI-1458-9.5-300
Ø9.5mm	320mm	1458-9.5-320	TI-1458-9.5-320
Ø9.5mm	340mm	1458-9.5-340	TI-1458-9.5-340
Ø9.5mm	360mm	1458-9.5-360	TI-1458-9.5-360
Ø9.5mm	380mm	1458-9.5-380	TI-1458-9.5-380
Ø9.5mm	400mm	1458-9.5-400	TI-1458-9.5-400
Ø10mm	180mm	1458-10-180	TI-1458-10-180
Ø10mm	200mm	1458-10-200	TI-1458-10-200
Ø10mm	300mm	1458-10-300	TI-1458-10-300
Ø10mm	320mm	1458-10-320	TI-1458-10-320
Ø10mm	340mm	1458-10-340	TI-1458-10-340
Ø10mm	360mm	1458-10-360	TI-1458-10-360
Ø10mm	380mm	1458-10-380	TI-1458-10-380
Ø10mm	400mm	1458-10-400	TI-1458-10-400
Ø11mm	180mm	1458-11-180	TI-1458-11-180
Ø11mm	200mm	1458-11-200	TI-1458-11-200
Ø11mm	300mm	1458-11-300	TI-1458-11-300
Ø11mm	320mm	1458-11-320	TI-1458-11-320
Ø11mm	340mm	1458-11-340	TI-1458-11-340
Ø11mm	360mm	1458-11-360	TI-1458-11-360
Ø11mm	380mm	1458-11-380	TI-1458-11-380
Ø11mm	400mm	1458-11-400	TI-1458-11-400
Ø12mm	180mm	1458-12-180	TI-1458-12-180
Ø12mm	200mm	1458-12-200	TI-1458-12-200
Ø12mm	300mm	1458-12-300	TI-1458-12-300
Ø12mm	320mm	1458-12-320	TI-1458-12-320
Ø12mm	340mm	1458-12-340	TI-1458-12-340
Ø12mm	360mm	1458-12-360	TI-1458-12-360
Ø12mm	380mm	1458-12-380	TI-1458-12-380
Ø12mm	400mm	1458-12-400	TI-1458-12-400





## Retrograde Femur Nail

**STERILE**

Dia	Length	Stainless Steel	Titanium
Ø9.5mm	180mm	1458-9.5-180-S	TI-1458-9.5-180-S
Ø9.5mm	200mm	1458-9.5-200-S	TI-1458-9.5-200-S
Ø9.5mm	300mm	1458-9.5-300-S	TI-1458-9.5-300-S
Ø9.5mm	320mm	1458-9.5-320-S	TI-1458-9.5-320-S
Ø9.5mm	340mm	1458-9.5-340-S	TI-1458-9.5-340-S
Ø9.5mm	360mm	1458-9.5-360-S	TI-1458-9.5-360-S
Ø9.5mm	380mm	1458-9.5-380-S	TI-1458-9.5-380-S
Ø9.5mm	400mm	1458-9.5-400-S	TI-1458-9.5-400-S
Ø10mm	180mm	1458-10-180-S	TI-1458-10-180-S
Ø10mm	200mm	1458-10-200-S	TI-1458-10-200-S
Ø10mm	300mm	1458-10-300-S	TI-1458-10-300-S
Ø10mm	320mm	1458-10-320-S	TI-1458-10-320-S
Ø10mm	340mm	1458-10-340-S	TI-1458-10-340-S
Ø10mm	360mm	1458-10-360-S	TI-1458-10-360-S
Ø10mm	380mm	1458-10-380-S	TI-1458-10-380-S
Ø10mm	400mm	1458-10-400-S	TI-1458-10-400-S
Ø11mm	180mm	1458-11-180-S	TI-1458-11-180-S
Ø11mm	200mm	1458-11-200-S	TI-1458-11-200-S
Ø11mm	300mm	1458-11-300-S	TI-1458-11-300-S
Ø11mm	320mm	1458-11-320-S	TI-1458-11-320-S
Ø11mm	340mm	1458-11-340-S	TI-1458-11-340-S
Ø11mm	360mm	1458-11-360-S	TI-1458-11-360-S
Ø11mm	380mm	1458-11-380-S	TI-1458-11-380-S
Ø11mm	400mm	1458-11-400-S	TI-1458-11-400-S
Ø12mm	180mm	1458-12-180-S	TI-1458-12-180-S
Ø12mm	200mm	1458-12-200-S	TI-1458-12-200-S
Ø12mm	300mm	1458-12-300-S	TI-1458-12-300-S
Ø12mm	320mm	1458-12-320-S	TI-1458-12-320-S
Ø12mm	340mm	1458-12-340-S	TI-1458-12-340-S
Ø12mm	360mm	1458-12-360-S	TI-1458-12-360-S
Ø12mm	380mm	1458-12-380-S	TI-1458-12-380-S
Ø12mm	400mm	1458-12-400-S	TI-1458-12-400-S





# 5.0mm Locking Screw for Retrograde Femur Nail



**STERILE**

Dia	Length	Titanium	Stainless Steel
Ø5.0	32mm	TI-1458-032	1458-032
Ø5.0	34mm	TI-1458-034	1458-034
Ø5.0	36mm	TI-1458-036	1458-036
Ø5.0	38mm	TI-1458-038	1458-038
Ø5.0	40mm	TI-1458-040	1458-040
Ø5.0	42mm	TI-1458-042	1458-042
Ø5.0	44mm	TI-1458-044	1458-044
Ø5.0	46mm	TI-1458-046	1458-046
Ø5.0	48mm	TI-1458-048	1458-048
Ø5.0	50mm	TI-1458-050	1458-050
Ø5.0	52mm	TI-1458-052	1458-052
Ø5.0	54mm	TI-1458-054	1458-054
Ø5.0	56mm	TI-1458-056	1458-056
Ø5.0	58mm	TI-1458-058	1458-058
Ø5.0	60mm	TI-1458-060	1458-060
Ø5.0	62mm	TI-1458-062	1458-062
Ø5.0	64mm	TI-1458-064	1458-064
Ø5.0	66mm	TI-1458-066	1458-066
Ø5.0	68mm	TI-1458-068	1458-068
Ø5.0	70mm	TI-1458-070	1458-070
Ø5.0	72mm	TI-1458-072	1458-072
Ø5.0	74mm	TI-1458-074	1458-074
Ø5.0	76mm	TI-1458-076	1458-076
Ø5.0	78mm	TI-1458-078	1458-078
Ø5.0	80mm	TI-1458-080	1458-080

Dia	Length	Titanium	Stainless Steel
Ø5.0	32mm	TI-1458-032-S	1458-032-S
Ø5.0	34mm	TI-1458-034-S	1458-034-S
Ø5.0	36mm	TI-1458-036-S	1458-036-S
Ø5.0	38mm	TI-1458-038-S	1458-038-S
Ø5.0	40mm	TI-1458-040-S	1458-040-S
Ø5.0	42mm	TI-1458-042-S	1458-042-S
Ø5.0	44mm	TI-1458-044-S	1458-044-S
Ø5.0	46mm	TI-1458-046-S	1458-046-S
Ø5.0	48mm	TI-1458-048-S	1458-048-S
Ø5.0	50mm	TI-1458-050-S	1458-050-S
Ø5.0	52mm	TI-1458-052-S	1458-052-S
Ø5.0	54mm	TI-1458-054-S	1458-054-S
Ø5.0	56mm	TI-1458-056-S	1458-056-S
Ø5.0	58mm	TI-1458-058-S	1458-058-S
Ø5.0	60mm	TI-1458-060-S	1458-060-S
Ø5.0	62mm	TI-1458-062-S	1458-062-S
Ø5.0	64mm	TI-1458-064-S	1458-064-S
Ø5.0	66mm	TI-1458-066-S	1458-066-S
Ø5.0	68mm	TI-1458-068-S	1458-068-S
Ø5.0	70mm	TI-1458-070-S	1458-070-S
Ø5.0	72mm	TI-1458-072-S	1458-072-S
Ø5.0	74mm	TI-1458-074-S	1458-074-S
Ø5.0	76mm	TI-1458-076-S	1458-076-S
Ø5.0	78mm	TI-1458-078-S	1458-078-S
Ø5.0	80mm	TI-1458-080-S	1458-080-S



# End Cap For Retrograde Femur Nail



**STERILE**

Stainless Steel	Titanium
1458-01	TI-1458-01

Stainless Steel	Titanium
1458-01-S	TI-1458-01-S




# IMPLANT BOX



**1-019** Implant Box for Retrograde Femur Nail




Tray 1

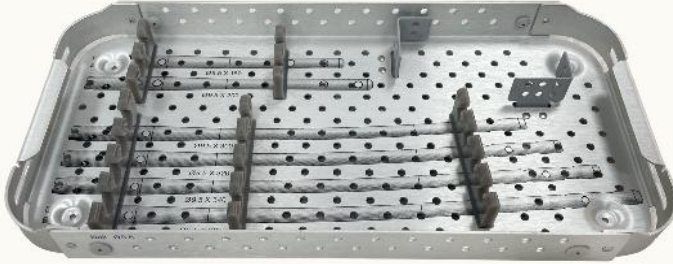

Dia	Length	Qty
Ø9.5mm	180mm to 360mm	1pcs

*Stainless Steel or Titanium Implants can be placed as per requirement*

Tray 2

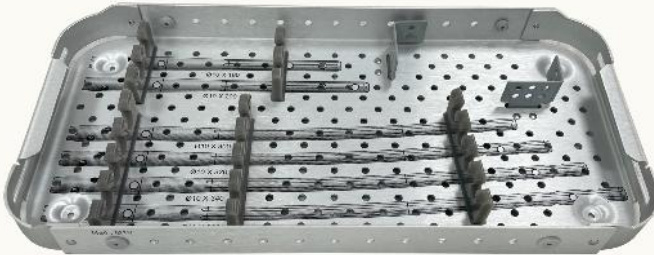

Dia	Length	Qty
Ø10mm	180mm to 360mm	1pcs

*Stainless Steel or Titanium Implants can be placed as per requirement*


Tray 3



Dia	Length	Qty
Ø11mm	180mm to 360mm	1pcs


*Stainless Steel or Titanium Implants can be placed as per requirement*

Tray 4

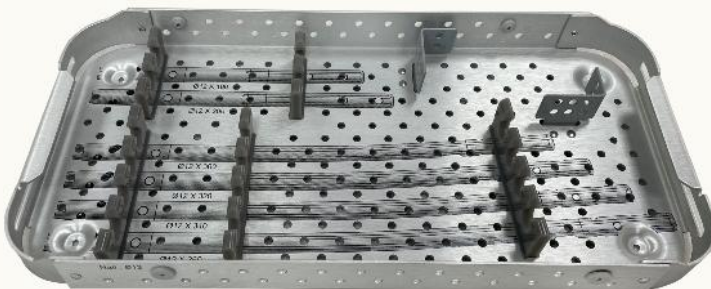

Dia	Length	Qty
Ø12mm	180mm to 360mm	1pcs

*Stainless Steel or Titanium Implants can be placed as per requirement*

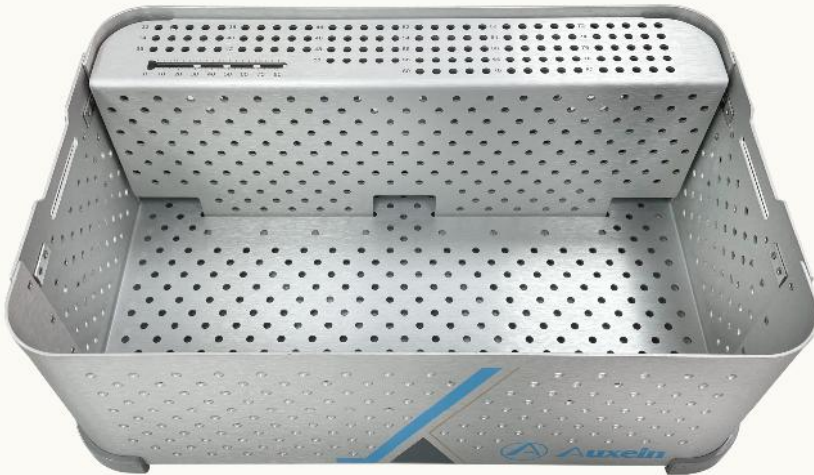

**Ø5.0mm Locking Screw For Retrograde Femur Nail**

Dia	Length	Qty
Ø5.0mm	32mm to 80mm	5pcs

*Stainless Steel or Titanium Implants can be placed as per requirement*

*Stainless Steel or Titanium Implants can be placed as per requirement*



**End Cap For Retrograde Femur Nail**

Qty
2pcs

*Stainless Steel or Titanium Implants can be placed as per requirement*



# INSTRUMENTS



1458-IN-1

Guide Wire With Threaded Tip  $\text{\O}3.2\text{mm}$  X Thread Length 15mm X Length 400mm

1458-IN-2

Tissue Protector For Retrograde Nail



1458-IN-3

Depth Gauge Measuring Upto 120mm For Retrograde Nail



1458-IN-4

Protection Sleeve 17/14mm For Entry Reamer/Wire Guide For Retrograde Nail



1458-IN-5

Entry Wire Guide  $\text{\O}14\text{mm}$  For 3.2mm Guide Wire - Retrograde Nail

1458-IN-6

Entry Reamer  $\text{\O}13.5\text{mm}$  For Retrograde Nail

1458-IN-7

Cannulated Awl  $\varnothing$ 13.5mm For Retrograde Nail



1458-IN-8

Insertion Handle For Retrograde Nail



1458-IN-9

Distal Aiming Device For Long Retrograde Nail



1458-IN-10

Proximal/Distal Aiming Attachment For Retrograde Nail



1458-IN-11

Proximal Aiming Arm For Retrograde Nail



1458-IN-12

Distal Arm Attachment For Distal Aiming Device - Retrograde Nail



1458-IN-13

Proximal/ Distal Short Nail Aiming Device For Retrograde Nail



1458-IN-14

Distal Aiming Device Bolt For Retrograde Nail



1458-IN-15

Stabilizing Rod Guide Ø10/8.0mm For Retrograde Nail



1458-IN-16

Stabilizing Rod With Quick Coupling End For Retrograde Nail



1458-IN-17

Stabilizing Spacer For Retrograde Nail



1458-IN-18

Distal Arm For Retrograde Nail



1458-IN-19

Locking Bolt For Distal Arm For Retrograde Nail



1458-IN-20

Threaded Lateral Stabilizing Rods Guide For Retrograde Femur Nail



1458-IN-21

Threaded Lateral Stabilizing Rod With Quick Coupling End Ø9.5mm,  
For Retrograde Nail

1458-IN-22

Threaded Lateral Stabilizing Rod With Quick Coupling End Ø10mm,  
For Retrograde Nail

1458-IN-23

Threaded Lateral Stabilizing Rod With Quick Coupling End Ø11mm,  
For Retrograde Nail

1458-IN-24

Threaded Lateral Stabilizing Rod With Quick Coupling End Ø12mm,  
For Retrograde Nail



1458-IN-25

Stabilizing Spacer For Threaded Rod, For Retrograde Nail



1458-IN-26

Nail Holding Bolt For Retrograde Nail



1458-IN-27

Allen Key, Hex 5.0mm For Retrograde Nail



1458-IN-28

Impactor For Guide Handle For Retrograde Nail



1458-IN-29

Drill For Stabilizing Rod With Quick Coupling End For Retrograde Nail



1458-IN-30

Hammer Guide For Retrograde Nail



1458-IN-31

Extraction Screw For Retrograde Nail



1458-IN-32

Trocar Nail Ø3.2mm For Retrograde Nail



1458-IN-33

Position Rod Ø3.8mm X Length 260mm, For Retrograde Nail



Reamer Head

Dia	Code
Ø9mm	1458-IN-34
Ø9.5mm	1458-IN-35
Ø10mm	1458-IN-36
Ø10.5mm	1458-IN-37
Ø11mm	1458-IN-38
Ø11.5mm	1458-IN-39
Ø12mm	1458-IN-40
Ø12.5mm	1458-IN-41
Ø13mm	1458-IN-42



1458-IN-43

Handle With Quick Coupling For Retrograde Nail




1458-IN-44

Reducer For Retrograde Nail



1458-IN-45

Guide Wire With Olive Ø2.5mm X Length 1000mm X Olive Ø3.8mm

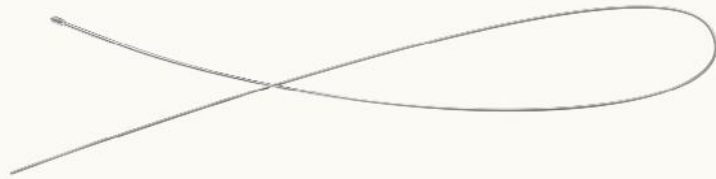

1458-IN-46

Flexible Reamer Shaft With Jacob Coupling, For Retrograde Nail



1458-IN-47

Outer Sleeve 11/8mm For Retrograde Nail



1458-IN-48

Cannulated Sleeve For Drill Bit 8/4mm For Retrograde Nail



1458-IN-49

Long Trocar Ø4.0mm X Length 205mm For Retrograde Nail



1458-IN-50

Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm X Length 320mm, For Retrograde Nail



1458-IN-51

Stopper For Drill Bit Ø4.0mm, For Retrograde Nail



1458-IN-52

Locating Pin Ø4.0mm X Length 450mm, For Retrograde Nail



1458-IN-53

Reamer For Lateral Cortex With Quick Coupling End For Retrograde Nail



1458-IN-54

Hexagonal Wrench, Hex 4.5mm, For Retrograde Nail



1458-IN-60

Threaded Trocar For Hexagonal Wrench Ø2.2mm X Thread 4mm X Length 267mm, For Retrograde Nail



1458-IN-55

Combined Hammer For Retrograde Nail



1458-IN-56

Combined Wrench 11mm For Retrograde Nail



1458-IN-57

L Handle Pin Wrench Ø4.5mm For Impactor - Retrograde Nail



1458-IN-62

Radiographic Ruler For Retrograde Nail




1458-IN-63

Wire Holder For Retrograde Nail



1458-IN-64

Guide Wire Without Olive Ø2.5mm X Length 1000mm

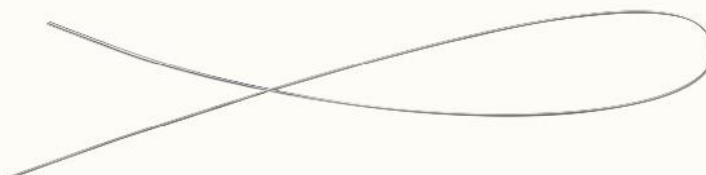



1458-IN-61

Instrument Trays For Retrograde Nailing Instrument Set





1458-IN-59

Container For Retrograde Nailing Instrument Set





**INS-1458 : Retrograde Nail Instruments Set**

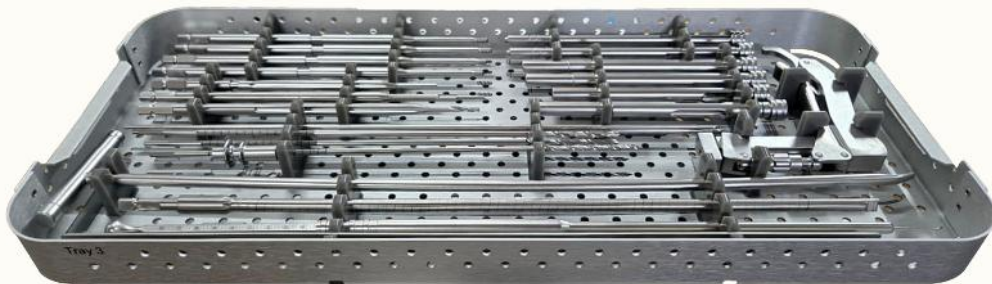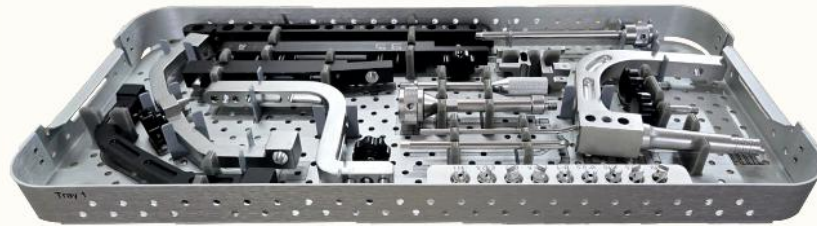



# ORDER INFORMATION

**INS-1458**
**Retrograde Nail Instrument Set**

Code	Set Consisting Of	Unit
1458-IN-1	Guide Wire with Threaded Tip Ø3.2mm x Thread Length 15mm x Length 400mm	2
1458-IN-2	Tissue Protector for Retrograde Nail	1
1458-IN-3	Depth Gauge measuring upto 120mm for Retrograde Nail	1
1458-IN-4	Protection Sleeve 17/14mm For Entry Reamer/Wire Guide for Retrograde Nail	1
1458-IN-5	Entry Wire Guide Ø14mm for 3.2mm Guide Wire - Retrograde Nail	1
1458-IN-6	Entry Reamer Ø13.5mm for Retrograde Nail	1
1458-IN-7	Cannulated Awl Ø13.5mm for Retrograde Nail	1
1458-IN-8	Insertion Handle for Retrograde Nail	1
1458-IN-9	Distal Aiming Device for Long Retrograde Nail	1
1458-IN-10	Proximal/Distal Aiming Attachment for Retrograde Nail	1
1458-IN-11	Proximal Aiming Arm for Retrograde Nail	1
1458-IN-12	Distal Arm Attachment for Distal Aiming Device - Retrograde Nail	1
1458-IN-13	Proximal/ Distal Short Nail Aiming Device for Retrograde Nail	1
1458-IN-14	Distal Aiming Device Bolt for Retrograde Nail	1
1458-IN-15	Stabilizing Rod Guide Ø10/8.0mm for Retrograde Nail	1
1458-IN-16	Stabilizing Rod With Quick Coupling End for Retrograde Nail	1
1458-IN-17	Stabilizing Spacer for Retrograde Nail	1
1458-IN-18	Distal Arm for Retrograde Nail	1
1458-IN-19	Locking Bolt For Distal Arm for Retrograde Nail	1
1458-IN-20	Threaded Lateral Stabilizing Rods Guide for Retrograde Femur Nail	1
1458-IN-21	Threaded Lateral Stabilizing Rod with Quick Coupling End Ø9.5mm	1
1458-IN-22	Threaded Lateral Stabilizing Rod with Quick Coupling End Ø10mm	1
1458-IN-23	Threaded Lateral Stabilizing Rod with Quick Coupling End Ø11mm	1
1458-IN-24	Threaded Lateral Stabilizing Rod with Quick Coupling End Ø12mm	1
1458-IN-25	Stabilizing Spacer for Threaded Rod, for Retrograde Nail	1
1458-IN-26	Nail Holding Bolt for Retrograde Nail	1
1458-IN-27	Allen key, Hex 5.0mm for Retrograde Nail	1
1458-IN-28	Impactor For Guide Handle for Retrograde Nail	1
1458-IN-29	Drill For Stabilizing Rod with Quick Coupling End for Retrograde Nail	1
1458-IN-30	Hammer Guide for Retrograde Nail	1
1458-IN-31	Extraction Screw for Retrograde Nail	1
1458-IN-32	Trocar Nail Ø3.2mm for Retrograde Nail	1
1458-IN-33	Position Rod Ø3.8mm x Length 260mm, for Retrograde Nail	1
1458-IN-34	Reamer Head Ø9.0mm for Retrograde Nail	1
1458-IN-35	Reamer Head Ø9.5mm for Retrograde Nail	1
1458-IN-36	Reamer Head Ø10.0mm for Retrograde Nail	1



# ORDER INFORMATION

**INS-1458**
**Retrograde Nail Instrument Set**

Code	Set Consisting Of	Unit
1458-IN-37	Reamer Head Ø10.5mm for Retrograde Nail	1
1458-IN-38	Reamer Head Ø11.0mm for Retrograde Nail	1
1458-IN-39	Reamer Head Ø11.5mm for Retrograde Nail	1
1458-IN-40	Reamer Head Ø12.0mm for Retrograde Nail	1
1458-IN-41	Reamer Head Ø12.5mm for Retrograde Nail	1
1458-IN-42	Reamer Head Ø13.0mm for Retrograde Nail	1
1458-IN-43	Handle with Quick Coupling for Retrograde Nail	1
1458-IN-44	Reducer for Retrograde Nail	1
1458-IN-45	Guide Wire with Olive Ø2.5mm x Length 1000mm x Olive Ø3.8mm	1
1458-IN-46	Flexible Reamer Shaft with Jacob Coupling, for Retrograde Nail	1
1458-IN-47	Outer Sleeve 11/8mm for Retrograde Nail	2
1458-IN-48	Cannulated Sleeve for Drill Bit 8/4mm for Retrograde Nail	2
1458-IN-49	Long Trocar Ø4.0mm x Length 205mm for Retrograde Nail	1
1458-IN-50	Drill Bit Plain Shank/Jacob Chuck End Ø4.0mm x Length 320mm	3
1458-IN-51	Stopper for Drill Bit Ø4.0mm, for Retrograde Nail	1
1458-IN-52	Locating Pin Ø4.0mm x Length 320mm, for Retrograde Nail	2
1458-IN-53	Reamer For Lateral Cortex with Quick Coupling End for Retrograde Nail	1
1458-IN-54	Hexagonal Wrench, Hex 4.5mm, for Retrograde Nail	2
1458-IN-60	Threaded Trocar For Hexagonal Wrench Ø2.2mm x Thread 4mm x Length 267mm	2
1458-IN-55	Combined Hammer for Retrograde Nail	1
1458-IN-56	Combined Wrench 11mm for Retrograde Nail	1
1458-IN-57	L Handle Pin Wrench Ø4.5mm for Impactor - Retrograde Nail	1
1458-IN-62	Radiographic Ruler for Retrograde Nail	1
1458-IN-63	Wire Holder for Retrograde Nail	1
1458-IN-64	Guide Wire Without Olive Ø2.5mm x Length 1000mm	1
1458-IN-61	Instrument Trays for Retrograde Nailing Instrument Set	3
1458-IN-59	Container For Retrograde Nailing Instrument Set	1

## **RETROGRADE FEMUR NAILING SYSTEM**

The content of this catalogue is intended for healthcare professionals only. All product information, illustrations, descriptions, and specifications provided here are for general informational purposes and are subject to change without notice. Auxein Medical Pvt. Ltd. reserves the right to improve or discontinue products at any time without prior notification.

This document does not constitute medical advice. Surgeons and medical practitioners must exercise their own professional judgment when using any medical devices or instruments, including those provided by Auxein.

### **INTENDED USE**

The implants and instruments in this catalogue are intended for use by qualified orthopedic trauma surgeons trained in fracture fixation and intramedullary nailing procedures. Improper selection of the implant, incorrect surgical technique, misuse, or insufficient surgical experience may result in implant failure or other complications.

### **REGULATORY COMPLIANCE CERTIFIED**

CE Approved (EUMDR-2017/745)

USFDA 510(k)

ISO 13485:2016

CDSCO

MDSAP

### **INTELLECTUAL PROPERTY**

All trademarks, trade names, brand names, product names, and logos mentioned in this catalogue are the property of Auxein Medical Pvt. Ltd. Unauthorized use or reproduction is strictly prohibited.





SCAN TO KNOW MORE

**INDIA**  
**AUXEIN MEDICAL PVT.LTD.**

PLOT NO. 168-169-170, PHASE-4,  
KUNDLI INDUSTRIAL AREA,  
HSIIDC, SECTOR-57,  
SONEPAT-131028, HARYANA  
TEL: +91 99106 43638  
FAX: +91 86077 70197  
EMAIL: INFO@AUXEIN.COM

**MÉXICO**  
**AUXEIN MÉXICO, S.A DE CV**

TEPIC 39 INT 801, COLONIA  
ROMA SUR,  
ALCADÍA CUAUHTÉMOC, CDMX  
MÉXICO, C.P 06760  
TEL: +521 55 7261 0318  
EMAIL: INFO@AUXEIN.MX

**USA**  
**AUXEIN INC.**

1500 NW 89TH COURT,  
SUITE 107-108  
DORAL, FLORIDA 33172  
TEL. +1 305 395 6062  
E FAX: +1 305 395 62 62  
EMAIL: USOFFICE@AUXEIN.COM

**DUBAI UAE**  
**AUXEIN MEDICAL**

TRADING FZCO OUD METHA  
OFFICES BUILDING B BLOCK,  
1ST FLOOR, OFFICE NO.26,  
NEAR HEALTHCARE CITY,  
PO BOX-25572  
TEL. +971 438 71333  
EMAIL: INFO@AUXEIN.COM

[WWW.AUXEIN.COM](http://WWW.AUXEIN.COM)