



WISELOCK

3.5MM WISE-LOCK CALCANEAL PLATE

3.5mm Wise-lock Small Fragment

Indications:

- Extra-articular fractures of the calcaneus.
- Intra-articular calcaneal fractures (including joint depression and tongue-type patterns).
- Severely comminuted calcaneal fractures.
- Calcaneal osteotomies requiring stable fixation.
- Malunions of the calcaneus requiring corrective osteotomy and fixation.
- Reconstruction of calcaneal deformities.



3.5mm Wise-Lock Calcaneal Plate


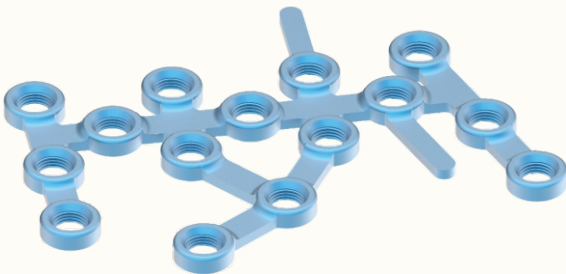


Plate Features:

- Specialized, pre-contoured design for optimal fixation of calcaneal and ankle fractures with anatomical alignment.
- Compatible with 3.5 mm locking & non-locking screws, providing versatile and stable fixation.
- Low-profile, biocompatible Stainless Steel/Titanium construction minimizes soft tissue irritation and supports open/MIS approaches.

Post Operative Lateral right foot X ray
with WISE-LOCK Calcaneal Plate

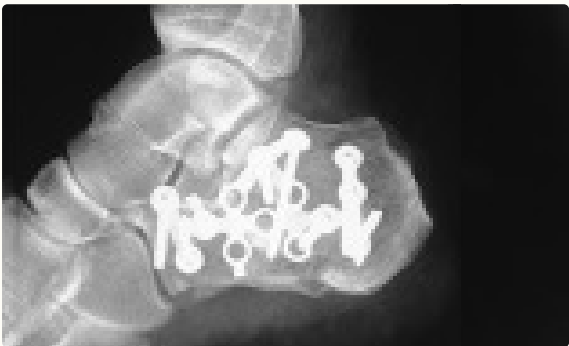

Direction	Titanium	Stainless Steel
Left	TI-797.069L	797.069L
Left	TI-797.076L	797.076L
Right	TI-797.069R	797.069R
Right	TI-797.076R	797.076R

STERILE

Direction	Titanium	Stainless Steel
Left	TI-797.069L-S	797.069L-S
Left	TI-797.076L-S	797.076L-S
Right	TI-797.069R-S	797.069R-S
Right	TI-797.076R-S	797.076R-S



3.5mm Wise-Lock Calcaneal Plate Type-I



Direction	Holes	Titanium	Stainless Steel
Left	3	10-007-03LTI	10-007-03LSS
Right	3	10-007-03RTI	10-007-03RSS

STERILE

Direction	Holes	Titanium	Stainless Steel
Left	3	10-007-03LTI-S	10-007-03LSS-S
Right	3	10-007-03RTI-S	10-007-03RSS-S

3.5mm Wise-Lock Calcaneal Plate-II



Direction	Holes		Titanium	Stainless Steel
Left	3	Small	10-008-03LTI	10-008-03LSS
Right	3	Small	10-008-03RTI	10-008-03RSS
Left	3	Medium	10-009-03LTI	10-009-03LSS
Right	3	Medium	10-009-03RTI	10-009-03RSS

STERILE

Direction	Holes		Titanium	Stainless Steel
Left	3	Small	10-008-03LTI-S	10-008-03LSS-S
Right	3	Small	10-008-03RTI-S	10-008-03RSS-S
Left	3	Medium	10-009-03LTI-S	10-009-03LSS-S
Right	3	Medium	10-009-03RTI-S	10-009-03RSS-S



3.5mm Wise-Lock Calcaneal Plate Type-III



Plate Features:

- Enables fixation using a minimally invasive approach through a sinus tarsi incision.
- The low-profile plate with rounded edges reduces soft tissue irritation.
- Designed with a low profile and rounded edges to minimize soft tissue irritation.
- Calcaneal plates accommodate 3.5 mm locking screws for secure fixation.

Post Operative Lateral right foot X ray
with WISE-LOCK Calcaneal Plate Type-III




Direction	Holes	Titanium	Stainless Steel
Left	6	10-010-06LTI	10-010-06LSS
Right	6	10-010-06RTI	10-010-06RSS
Left	7	10-010-07LTI	10-010-07LSS
Right	7	10-010-07RTI	10-010-07RSS



Direction	Holes	Titanium	Stainless Steel
Left	6	10-010-06LTI-S	10-010-06LSS-S
Right	6	10-010-06RTI-S	10-010-06RSS-S
Left	7	10-010-07LTI-S	10-010-07LSS-S
Right	7	10-010-07RTI-S	10-010-07RSS-S



3.5mm Wise-Lock Calcaneal Plate Type-IV

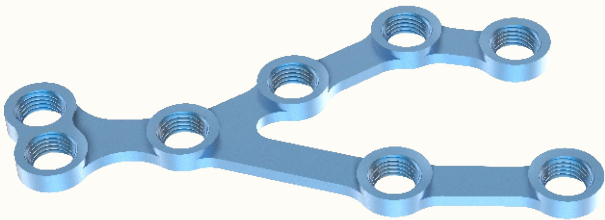


Direction	Holes		Titanium	Stainless Steel
Left	8	Small	10-011-08LTI	10-011-08LSS
Right	8	Small	10-011-08RTI	10-011-08RSS
Left	8	Medium	10-012-08LTI	10-012-08LSS
Right	8	Medium	10-012-08RTI	10-012-08RSS
Left	8	Large	10-013-08LTI	10-013-08LSS
Right	8	Large	10-013-08RTI	10-013-08RSS

STERILE

Direction	Holes		Titanium	Stainless Steel
Left	8	Small	10-011-08LTI-S	10-011-08LSS-S
Right	8	Small	10-011-08RTI-S	10-011-08RSS-S
Left	8	Medium	10-012-08LTI-S	10-012-08LSS-S
Right	8	Medium	10-012-08RTI-S	10-012-08RSS-S
Left	8	Large	10-013-08LTI-S	10-013-08LSS-S
Right	8	Large	10-013-08RTI-S	10-013-08RSS-S



3.5mm Wise-Lock Calcaneal Plate Type-V

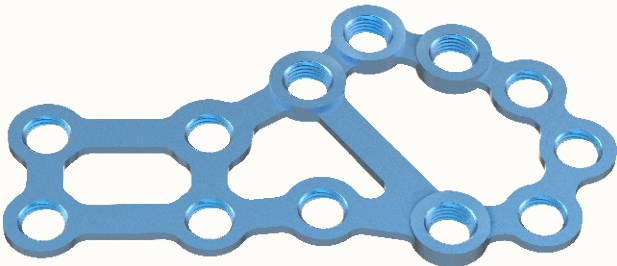

- Plate Features:**
- The frame design ensures uniform stress distribution across the plate.
 - Plate coverage of the calcaneus facilitates screw fixation in dense bone regions.



Post Operative Lateral right foot X ray with WISE-LOCK Calcaneal Plate Type-V




Direction	Holes		Titanium	Stainless Steel
Left	12	Small	10-014-12LTI	10-014-12LSS
Right	12	Small	10-014-12RTI	10-014-12RSS
Left	12	Large	10-015-12LTI	10-015-12LSS
Right	12	Large	10-015-12RTI	10-015-12RSS

STERILE

Direction	Holes		Titanium	Stainless Steel
Left	12	Small	10-014-12LTI-S	10-014-12LSS-S
Right	12	Small	10-014-12RTI-S	10-014-12RSS-S
Left	12	Large	10-015-12LTI-S	10-015-12LSS-S
Right	12	Large	10-015-12RTI-S	10-015-12RSS-S



2.7mm Wise-Lock X-plate

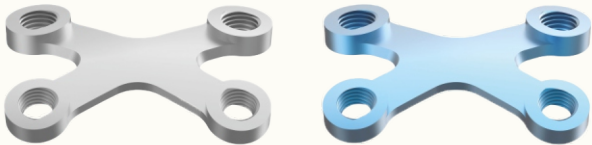


	Titanium	Stainless Steel
Small	TI-812.300S	812.300S
Medium	TI-812.300M	812.300M
Large	TI-812.300L	812.300L



	Titanium	Stainless Steel
Small	TI-812.300S-S	812.300S-S
Medium	TI-812.300M-S	812.300M-S
Large	TI-812.300L-S	812.300L-S


2.7mm Wise-Lock Straight Plate With Single Start Thread

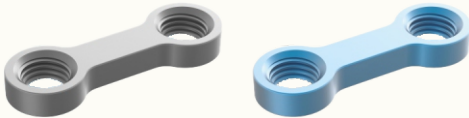

Holes	Titanium	Stainless Steel
2	10-061-02TI	10-061-02SS



Holes	Titanium	Stainless Steel
2	10-061-02TI-S	10-061-02SS-S

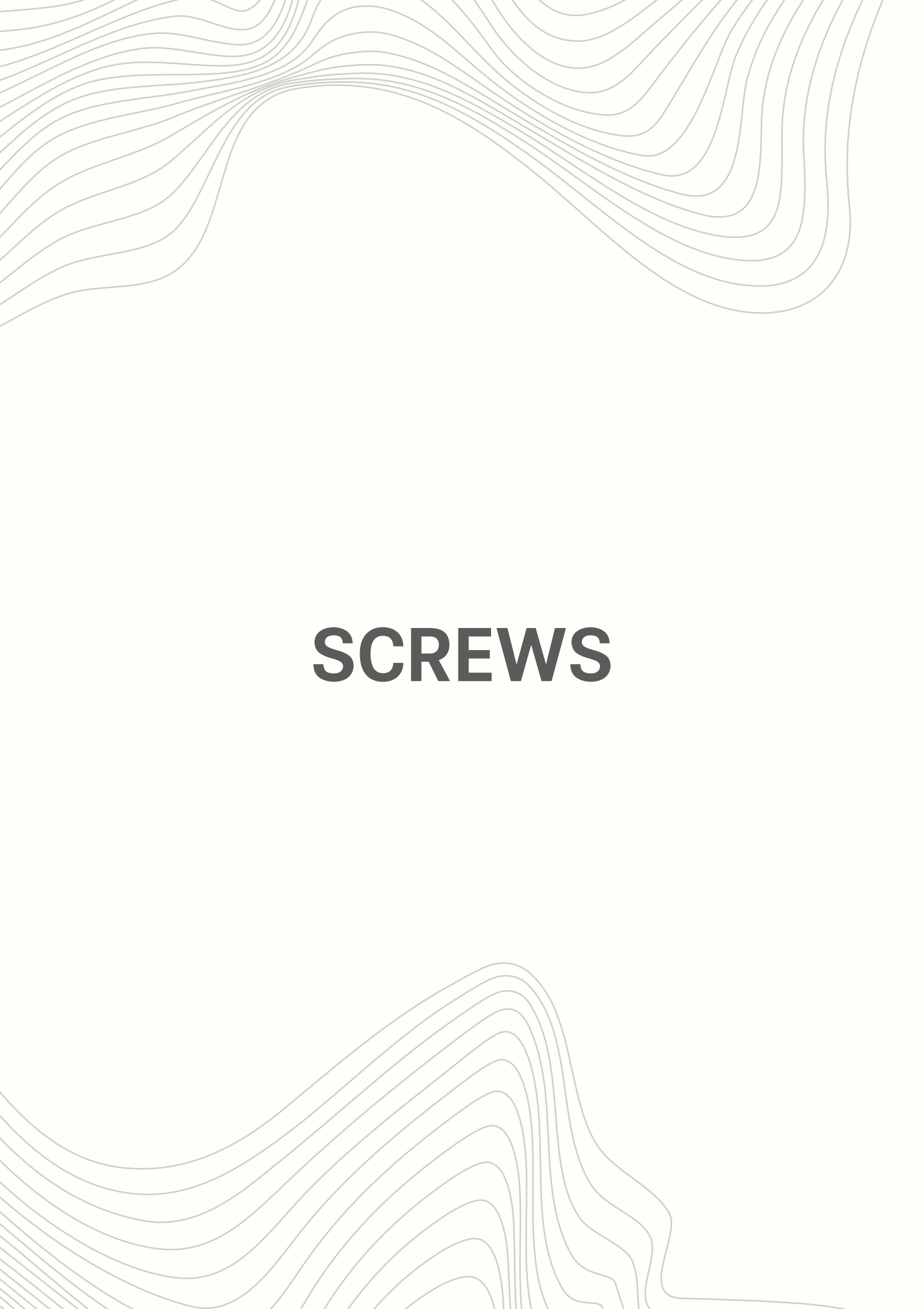The background of the page features a light gray, wavy, topographic-like pattern that flows from the top and bottom edges towards the center, framing the central text.

SCREWS



3.5mm Wise-Lock Screw, Self-Tapping (Hex Head)


STERILE


Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.010	117.010
Ø3.5	12	TI-117.012	117.012
Ø3.5	14	TI-117.014	117.014
Ø3.5	16	TI-117.016	117.016
Ø3.5	18	TI-117.018	117.018
Ø3.5	20	TI-117.020	117.020
Ø3.5	22	TI-117.022	117.022
Ø3.5	24	TI-117.024	117.024
Ø3.5	26	TI-117.026	117.026
Ø3.5	28	TI-117.028	117.028
Ø3.5	30	TI-117.030	117.030
* Ø3.5	32	TI-117.032	117.032
* Ø3.5	34	TI-117.034	117.034
Ø3.5	35	TI-117.035	117.035
* Ø3.5	36	TI-117.036	117.036
* Ø3.5	38	TI-117.038	117.038
Ø3.5	40	TI-117.040	117.040
* Ø3.5	42	TI-117.042	117.042
* Ø3.5	44	TI-117.044	117.044
Ø3.5	45	TI-117.045	117.045
* Ø3.5	46	TI-117.046	117.046
* Ø3.5	48	TI-117.048	117.048
Ø3.5	50	TI-117.050	117.050
* Ø3.5	52	TI-117.052	117.052
* Ø3.5	54	TI-117.054	117.054
Ø3.5	55	TI-117.055	117.055
* Ø3.5	56	TI-117.056	117.056
* Ø3.5	58	TI-117.058	117.058
Ø3.5	60	TI-117.060	117.060
Ø3.5	65	TI-117.065	117.065
Ø3.5	70	TI-117.070	117.070
Ø3.5	75	TI-117.075	117.075
Ø3.5	80	TI-117.080	117.080

Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.010-S	117.010-S
Ø3.5	12	TI-117.012-S	117.012-S
Ø3.5	14	TI-117.014-S	117.014-S
Ø3.5	16	TI-117.016-S	117.016-S
Ø3.5	18	TI-117.018-S	117.018-S
Ø3.5	20	TI-117.020-S	117.020-S
Ø3.5	22	TI-117.022-S	117.022-S
Ø3.5	24	TI-117.024-S	117.024-S
Ø3.5	26	TI-117.026-S	117.026-S
Ø3.5	28	TI-117.028-S	117.028-S
Ø3.5	30	TI-117.030-S	117.030-S
* Ø3.5	32	TI-117.032-S	117.032-S
* Ø3.5	34	TI-117.034-S	117.034-S
Ø3.5	35	TI-117.035-S	117.035-S
* Ø3.5	36	TI-117.036-S	117.036-S
* Ø3.5	38	TI-117.038-S	117.038-S
Ø3.5	40	TI-117.040-S	117.040-S
* Ø3.5	42	TI-117.042-S	117.042-S
* Ø3.5	44	TI-117.044-S	117.044-S
Ø3.5	45	TI-117.045-S	117.045-S
* Ø3.5	46	TI-117.046-S	117.046-S
* Ø3.5	48	TI-117.048-S	117.048-S
Ø3.5	50	TI-117.050-S	117.050-S
* Ø3.5	52	TI-117.052-S	117.052-S
* Ø3.5	54	TI-117.054-S	117.054-S
Ø3.5	55	TI-117.055-S	117.055-S
* Ø3.5	56	TI-117.056-S	117.056-S
* Ø3.5	58	TI-117.058-S	117.058-S
Ø3.5	60	TI-117.060-S	117.060-S
Ø3.5	65	TI-117.065-S	117.065-S
Ø3.5	70	TI-117.070-S	117.070-S
Ø3.5	75	TI-117.075-S	117.075-S
Ø3.5	80	TI-117.080-S	117.080-S



* Sizes not available in Screw Caddy

* Sizes not available in Screw Caddy



3.5mm Wise-Lock Screw, Self-Drilling (Hex Head)


STERILE



Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.210	117.210
Ø3.5	12	TI-117.212	117.212
Ø3.5	14	TI-117.214	117.214
Ø3.5	16	TI-117.216	117.216
Ø3.5	18	TI-117.218	117.218
Ø3.5	20	TI-117.220	117.220
Ø3.5	22	TI-117.222	117.222
Ø3.5	24	TI-117.224	117.224
Ø3.5	26	TI-117.226	117.226
Ø3.5	28	TI-117.228	117.228
Ø3.5	30	TI-117.230	117.230
* Ø3.5	32	TI-117.232	117.232
* Ø3.5	34	TI-117.234	117.234
Ø3.5	36	TI-117.236	117.236
* Ø3.5	38	TI-117.238	117.238
* Ø3.5	40	TI-117.240	117.240
Ø3.5	42	TI-117.242	117.242
* Ø3.5	44	TI-117.244	117.244
* Ø3.5	46	TI-117.246	117.246
Ø3.5	48	TI-117.248	117.248
* Ø3.5	50	TI-117.250	117.250
* Ø3.5	52	TI-117.252	117.252
Ø3.5	54	TI-117.254	117.254
* Ø3.5	56	TI-117.256	117.256
* Ø3.5	58	TI-117.258	117.258
Ø3.5	60	TI-117.260	117.260

* Sizes not available in Screw Caddy

Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.210-S	117.210-S
Ø3.5	12	TI-117.212-S	117.212-S
Ø3.5	14	TI-117.214-S	117.214-S
Ø3.5	16	TI-117.216-S	117.216-S
Ø3.5	18	TI-117.218-S	117.218-S
Ø3.5	20	TI-117.220-S	117.220-S
Ø3.5	22	TI-117.222-S	117.222-S
Ø3.5	24	TI-117.224-S	117.224-S
Ø3.5	26	TI-117.226-S	117.226-S
Ø3.5	28	TI-117.228-S	117.228-S
Ø3.5	30	TI-117.230-S	117.230-S
* Ø3.5	32	TI-117.232-S	117.232-S
* Ø3.5	34	TI-117.234-S	117.234-S
Ø3.5	36	TI-117.236-S	117.236-S
* Ø3.5	38	TI-117.238-S	117.238-S
* Ø3.5	40	TI-117.240-S	117.240-S
Ø3.5	42	TI-117.242-S	117.242-S
* Ø3.5	44	TI-117.244-S	117.244-S
* Ø3.5	46	TI-117.246-S	117.246-S
Ø3.5	48	TI-117.248-S	117.248-S
* Ø3.5	50	TI-117.250-S	117.250-S
* Ø3.5	52	TI-117.252-S	117.252-S
Ø3.5	54	TI-117.254-S	117.254-S
* Ø3.5	56	TI-117.256-S	117.256-S
* Ø3.5	58	TI-117.258-S	117.258-S
Ø3.5	60	TI-117.260-S	117.260-S

* Sizes not available in Screw Caddy

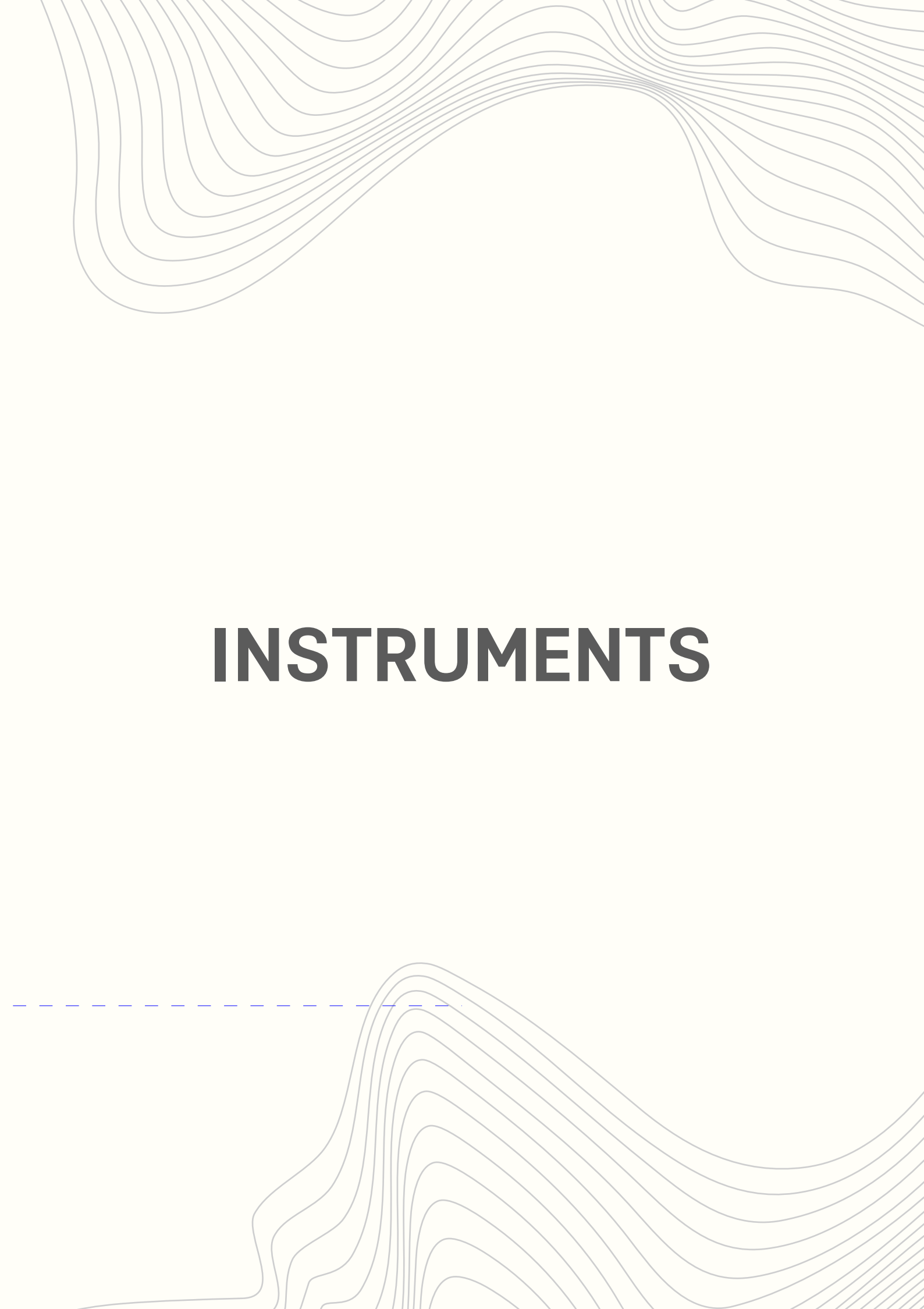

The background features a light beige color with abstract, wavy, topographic-style lines in a light gray color. These lines are concentrated in the top and bottom corners, creating a sense of depth and movement. A horizontal dashed blue line is positioned across the middle of the page, just below the main title.

INSTRUMENTS



3443-37

Depth Gauge Measuring Upto 80mm For Small Fragment



3445-2.5

T-handle Screwdriver, Hex 2.5mm, Self-retaining For Small Fragment




3992-035

Allen Key, Hex 3.0mm, For Small Fragment



2100-2.0-110

Drill Bit With Quick Coupling End, Ø2.0mm X Length 110mm, For Small Fragment

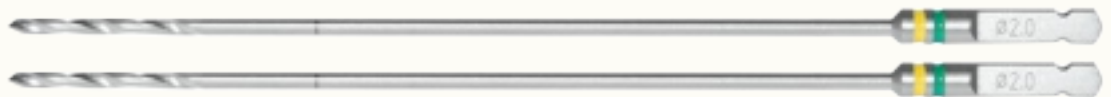


2100-2.5-112

Drill Bit With Quick Coupling End, Ø2.5mm X Length 112mm, For Small Fragment




2100-3.5-112

Drill Bit With Quick Coupling End, Ø3.5mm X Length 112mm, For Small Fragment

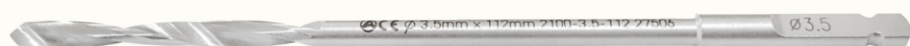

2103-2.8-165

Drill Bit Quick Coupling With Stopper, Ø2.8mm X Length 165mm,
For Small Fragment

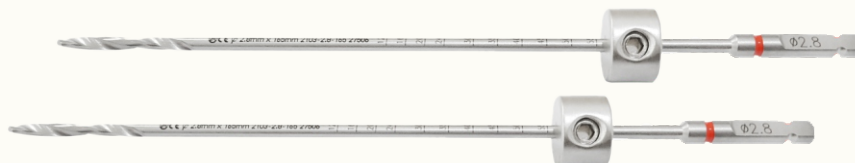

2104-27

Bone Tap Quick Coupling For Wise-lock Screws, Ø2.7mm, For Small Fragment




2104-02

Bone Tap Quick Coupling For Cortical Screws, Ø3.5mm, For Small Fragment



2104-04

Bone Tap, Quick Coupling For Cancellous Screws, Ø4.0mm, For Small Fragment

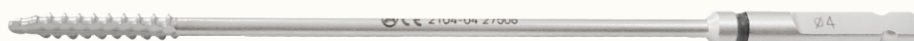


3400-01

Countersink For Ø3.5/4.0mm Screws, Quick Coupling, For Small Fragment



3408-01


Hexagonal Screwdriver Shaft - 2.5mm Tip, Quick Coupling, For Small Fragment

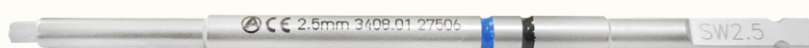



3408-03

Hexagonal Screwdriver Shaft - 2.0mm Tip, Quick Coupling, For Small Fragment

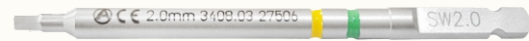

2186-2.5

Hss Drill Bit, Ø2.5mm, For Small Fragment



2106-1.2

Guide Sleeve For Ø1.2mm K. Wires, For Small Fragment



3443-05

Depth Gauge With Protector Measuring Upto 50mm For Small Fragment



3443-39

Trepine For Small Fragment



BT-SF-06

Bending Template, Small, For Small Fragment





BT-SF-08

Bending Template, Medium, For Small Fragment



BT-SF-10

Bending Template, Large, For Small Fragment



3402-000

T-handle With Quick Coupling




1472-054

Quick Coupling Shaft



TDG-2.7

Threaded Drill Sleeve For Drill Bit Ø2.0mm - Small Fragment

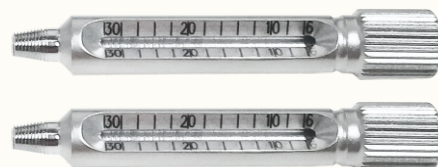

7-050-04

Threaded Drill Sleeve For Drill Bit Ø2.8 For Wise-lock Small Fragment For T-handle





3420-01

Drill Sleeve Insert Ø3.5/2.5mm For Small Fragment



1472-036

Drill Guide 2.0mm For Small Fragment



1472-044

Double Drill Guide Ø2.0/2.7mm For Small Fragment



1472-046

Self-centering Double Drill Guide, Ø2.5/3.5mm, For Small Fragment



3441-16

Drill Guide For Neutral And Loaded Position Ø3.5mm, For Small Fragment



1472-066

Hohmann Retractor, 6.5mm, For Small Fragment





1472-068

Hohmann Retractor, 8.5mm, For Small Fragment



2146-018

Hohmann Retractor, 15.5mm, For Small Fragment



2149-1012

Periosteal Elevator, Straight, 12mm, For Small Fragment



3406-02

Hexagonal Screwdriver - 2.0mm Tip For Small Fragment



3406-02S

Screw Holding Sleeve For 2.0mm Tip Screwdriver, For Small Fragment



3406-03

Hexagonal Screwdriver - 2.5mm Tip, For Small Fragment





3406-03S

Screw Holding Sleeve For 2.5mm Tip Screwdriver, For Small Fragment



3409-03L

Bending Iron, Left For Small Fragment



3409-03R

Bending Iron, Right For Small Fragment



2107-1180

Reduction Forcep, Pointed, Ratchet Lock, 180mm, For Small Fragment



2106-1160

Reduction Forcep, Serrated Jaws, Speed Lock, 160mm, For Small Fragment



2106-190

Self-centering Bone Holding Forcep, Speed Lock, 190mm, For Small Fragment





2150-1006

Periosteal Elevator With Silicon Handle, Curved, 6mm, For Small Fragment



TQ-2.0

Torque Limiting Attachment, 0.8nm, For Small Fragment



TQ-2.5

Torque Limiting Attachment, 1.5nm, For Small Fragment




1472-064

Torque Screwdriver Handle



GW-1.2-230

Guide Wire With Threaded Trocar Tip, Ø1.2mm X Thread Length 10MM X Length 230MM

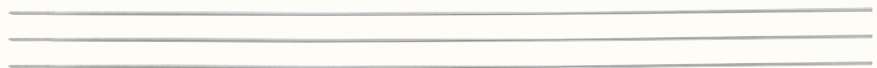

7-050-05

T- Handle For Threaded Drill Sleeve





3415-000

Screw Holding Forcep





ST-007NW

Screw Caddy For 3.5mm Wise-lock Small Fragment System



7-050-03


Instrument Trays For 3.5mm Wise-lock Small Fragment Instrument Set

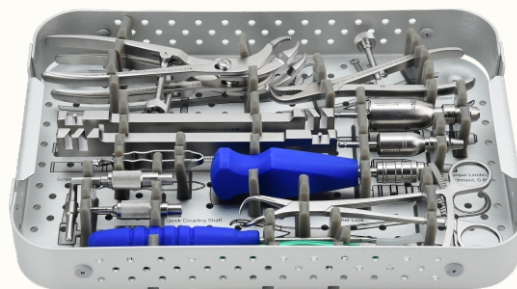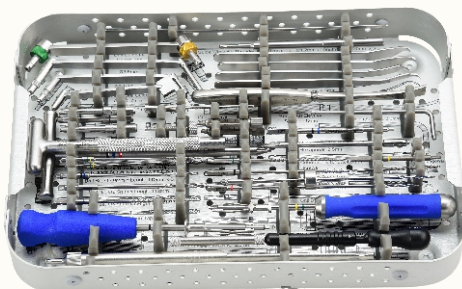

IC-2310-WL

Container For 3.5mm Wise-lock Small Fragment Instrument Set

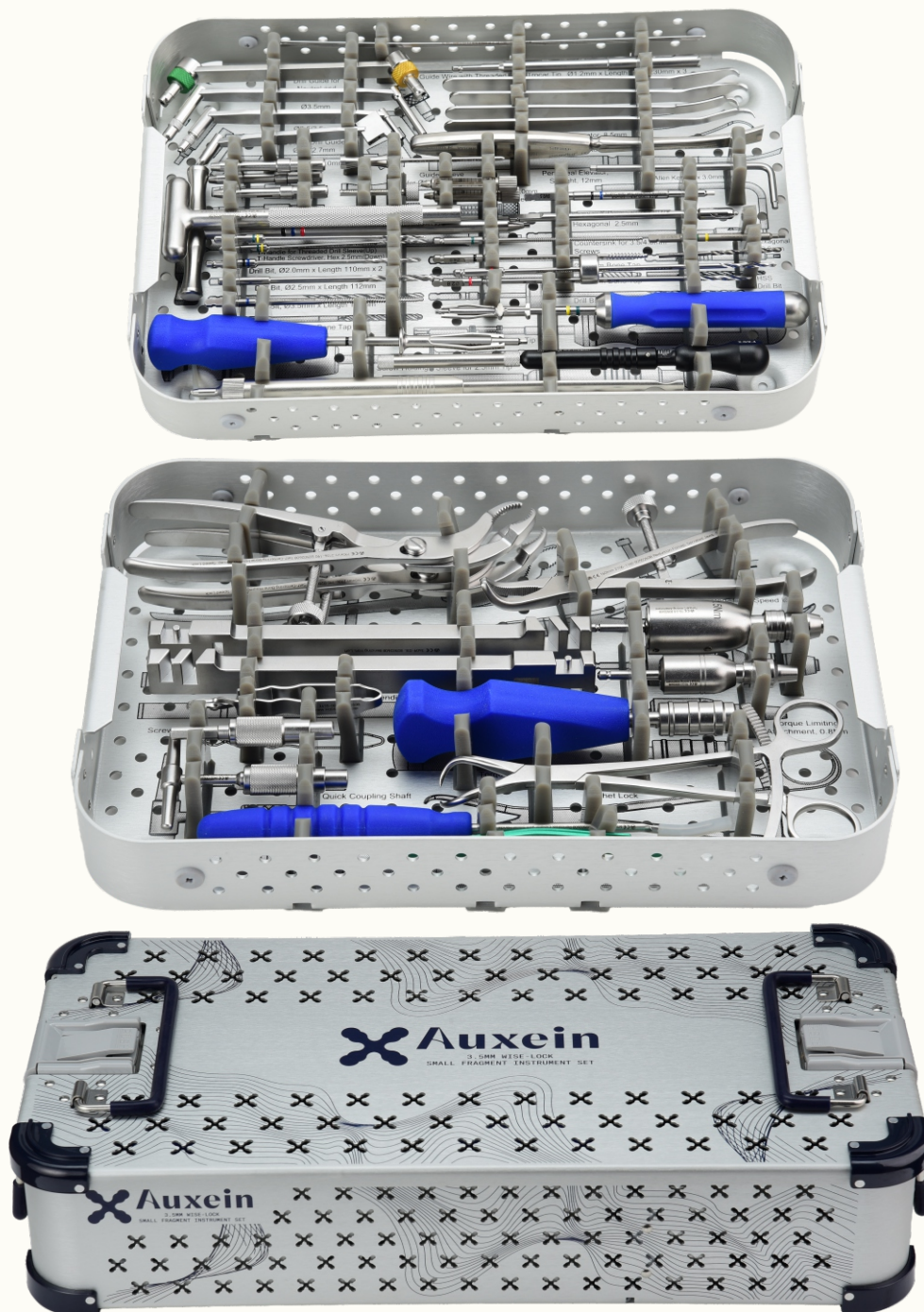

**2302-000 : Small Fragment Wise-Lock Instrument Set**



ORDER INFORMATION

2302-000	Small Fragment Wise-Lock Instrument Set
----------	---

Code	Set Consisting Of	Unit
3443-37	Depth Gauge measuring upto 80mm for Small Fragment	1
3445-2.5	T-Handle Screwdriver, Hex 2.5mm, Self-Retaining for Small Fragment	1
3992-035	Allen Key, Hex 3.0mm, for Small Fragment	1
2100-2.0-110	Drill Bit with Quick Coupling End, Ø2.0mm x Length 110mm, for Small Fragment	2
2100-2.5-112	Drill Bit with Quick Coupling End, Ø2.5mm x Length 112mm, for Small Fragment	1
2100-3.5-112	Drill Bit with Quick Coupling End, Ø3.5mm x Length 112mm, for Small Fragment	1
2103-2.8-165	Drill Bit Quick Coupling with Stopper, Ø2.8mm x Length 165mm, for Small Fragment	2
2104-27	Bone Tap Quick Coupling for Wise-Lock Screws, Ø2.7mm, for Small Fragment	1
2104-02	Bone Tap Quick Coupling for Cortical Screws, Ø3.5mm, for Small Fragment	1
2104-04	Bone Tap, Quick Coupling for Cancellous Screws, Ø4.0mm, for Small Fragment	1
3400-01	Countersink for Ø3.5/4.0mm Screws, Quick Coupling, for Small Fragment	1
3408-01	Hexagonal Screwdriver Shaft - 2.5mm Tip, Quick Coupling, for Small Fragment	1
3408-03	Hexagonal Screwdriver Shaft - 2.0mm Tip, Quick Coupling, for Small Fragment	1
2186-2.5	HSS Drill Bit, Ø2.5mm, for Small Fragment	1
2106-1.2	Guide Sleeve for Ø1.2mm K. Wires, , for Small Fragment	1
3443-05	Depth Gauge with Protector measuring upto 50mm for Small Fragment	1
3443-39	Trephine for Small Fragment	1
BT-SF-06	Bending Template, Small, for Small Fragment	1
BT-SF-08	Bending Template, Medium, for Small Fragment	1
BT-SF-10	Bending Template, Large, for Small Fragment	1
3402-000	T-Handle with Quick Coupling	1
1472-054	Quick Coupling Shaft	1
TDG-2.7	Threaded Drill Sleeve for Drill Bit Ø2.0mm - Small Fragment	2
7-050-04	Threaded Drill Sleeve for Drill Bit Ø2.8 for Wise-lock Small Fragment for T-Handle	3
3420-01	Drill Sleeve Insert Ø3.5/2.5mm for Small Fragment	1
1472-036	Drill Guide 2.0mm for Small Fragment	1
1472-044	Double Drill Guide Ø2.0/2.7mm for Small Fragment	1
1472-046	Self-Centering Double Drill Guide, Ø2.5/3.5mm, for Small Fragment	1
3441-16	Drill Guide for Neutral and Loaded Position Ø3.5mm, for Small Fragment	1
1472-066	Hohmann Retractor, 6.5mm, for Small Fragment	1
1472-068	Hohmann Retractor, 8.5mm, for Small Fragment	1
2146-018	Hohmann Retractor, 15.5mm, for Small Fragment	2
2149-1012	Periosteal Elevator, Straight, 12mm, for Small Fragment	1
3406-02	Hexagonal Screwdriver - 2.0mm Tip for Small Fragment	1
3406-02S	Screw Holding Sleeve for 2.0mm Tip Screwdriver, for Small Fragment	1
3406-03	Hexagonal Screwdriver - 2.5mm Tip, for Small Fragment	1
3406-03S	Screw Holding Sleeve for 2.5mm Tip Screwdriver, for Small Fragment	1
3409-03L	Bending Iron, Left for Small Fragment	1
3409-03R	Bending Iron, Right for Small Fragment	1
2107-1180	Reduction Forcep, Pointed, Ratchet Lock, 180mm, for Small Fragment	1
2106-1160	Reduction Forcep, Serrated Jaws, Speed Lock, 160mm, for Small Fragment	1



2302-000	Small Fragment Wise-Lock Instrument Set
----------	---

Code	Set Consisting Of	Unit
2106-190	Self-Centering Bone Holding Forcep, Speed Lock, 190mm, for Small Fragment	2
2150-1006	Periosteal Elevator with Silicon Handle, Curved, 6mm, for Small Fragment	1
TQ-2.0	Torque Limiting Attachment, 0.8Nm, for Small Fragment	1
TQ-2.5	Torque Limiting Attachment, 1.5Nm, for Small Fragment	1
1472-064	Torque Screwdriver Handle	1
GW-1.2-230	Guide Wire with Threaded Trocar Tip, Ø1.2mm x Thread Length 10mm x Length 230mm	3
7-050-05	T- Handle for Threaded Drill Sleeve	1
3415-000	Screw Holding Forcep	1
ST-007NW	Screw Caddy for 3.5mm Wise-Lock Small Fragment System	1
7-050-03	Instrument Trays for 3.5mm Wise-Lock Small Fragment Instrument Set	2
IC-2310-WL	Container for 3.5mm Wise-Lock Small Fragment Instrument Set	1