



WISELOCK™

3.5mm Wise-Lock Distal Tibia Plates

3.5mm Wise-lock Small Fragment

Introduction:

An anatomically precontoured plate designed for fixation of fractures of the distal tibia via a medial approach. The design accommodates the subcutaneous medial surface and supports angular stable fixation while preserving the soft-tissue envelope.

Indications:

- Extra-articular distal tibia fractures (AO/OTA 43-A).
- Simple intra-articular distal tibia fractures with minimal articular comminution (selected 43-B).
- Metaphyseal distal tibia fractures with or without diaphyseal extension.
- Distal tibia fractures in osteoporotic bone requiring angular stability.
- Fractures requiring minimally invasive plate osteosynthesis (MIPO) technique.
- Nonunions and malunions of the distal tibia.






3.5mm Wise-Lock Medial Distal Tibia Plate



Left



Right

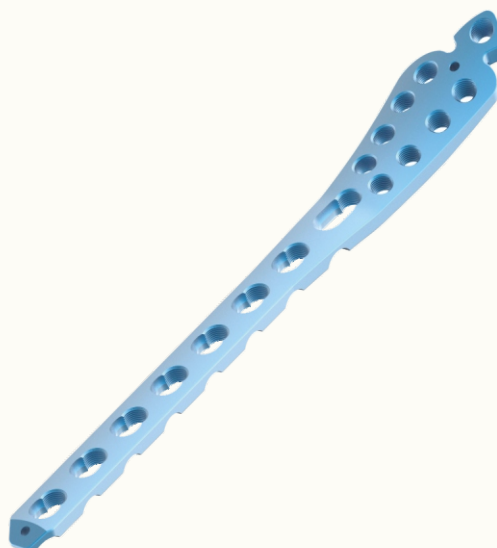Round
Locking HoleCompression and
Locking Hole**STERILE**

Holes	Direction	Titanium	Stainless Steel
4	Left	TI-746.104L	746.104L
4	Right	TI-746.104R	746.104R
6	Left	TI-746.106L	746.106L
6	Right	TI-746.106R	746.106R
8	Left	TI-746.108L	746.108L
8	Right	TI-746.108R	746.108R
10	Left	TI-746.110L	746.110L
10	Right	TI-746.110R	746.110R
12	Left	TI-746.112L	746.112L
12	Right	TI-746.112R	746.112R
14	Left	TI-746.114L	746.114L
14	Right	TI-746.114R	746.114R

Holes	Direction	Titanium	Stainless Steel
4	Left	TI-746.104L-S	746.104L-S
4	Right	TI-746.104R-S	746.104R-S
6	Left	TI-746.106L-S	746.106L-S
6	Right	TI-746.106R-S	746.106R-S
8	Left	TI-746.108L-S	746.108L-S
8	Right	TI-746.108R-S	746.108R-S
10	Left	TI-746.110L-S	746.110L-S
10	Right	TI-746.110R-S	746.110R-S
12	Left	TI-746.112L-S	746.112L-S
12	Right	TI-746.112R-S	746.112R-S
14	Left	TI-746.114L-S	746.114L-S
14	Right	TI-746.114R-S	746.114R-S




3.5mm Wise-Lock Anterolateral Distal Tibia Plate



Left



Right

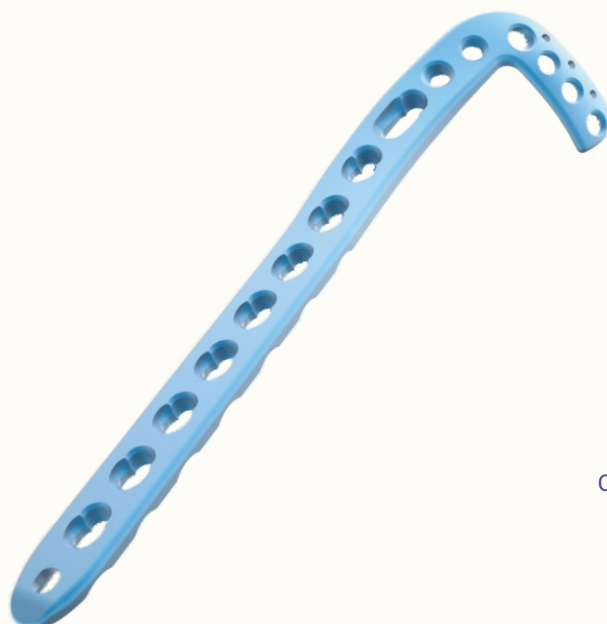Round
Locking HoleCompression and
Locking Hole**STERILE**


Holes	Direction	Titanium	Stainless Steel
5	Left	TI-WS1.05	WS1.05
5	Right	TI-WS2.05	WS2.05
7	Left	TI-WS1.07	WS1.07
7	Right	TI-WS2.07	WS2.07
9	Left	TI-WS1.09	WS1.09
9	Right	TI-WS2.09	WS2.09
11	Left	TI-WS1.11	WS1.11
11	Right	TI-WS2.11	WS2.11
13	Left	TI-WS1.13	WS1.13
13	Right	TI-WS2.13	WS2.13
15	Left	TI-WS1.15	WS1.15
15	Right	TI-WS2.15	WS2.15
17	Left	TI-WS1.17	WS1.17
17	Right	TI-WS2.17	WS2.17
19	Left	TI-WS1.19	WS1.19
19	Right	TI-WS2.19	WS2.19
21	Left	TI-WS1.21	WS1.21
21	Right	TI-WS2.21	WS2.21

Holes	Direction	Titanium	Stainless Steel
5	Left	TI-WS1.05-S	WS1.05-S
5	Right	TI-WS2.05-S	WS2.05-S
7	Left	TI-WS1.07-S	WS1.07-S
7	Right	TI-WS2.07-S	WS2.07-S
9	Left	TI-WS1.09-S	WS1.09-S
9	Right	TI-WS2.09-S	WS2.09-S
11	Left	TI-WS1.11-S	WS1.11-S
11	Right	TI-WS2.11-S	WS2.11-S
13	Left	TI-WS1.13-S	WS1.13-S
13	Right	TI-WS2.13-S	WS2.13-S
15	Left	TI-WS1.15-S	WS1.15-S
15	Right	TI-WS2.15-S	WS2.15-S
17	Left	TI-WS1.17-S	WS1.17-S
17	Right	TI-WS2.17-S	WS2.17-S
19	Left	TI-WS1.19-S	WS1.19-S
19	Right	TI-WS2.19-S	WS2.19-S
21	Left	TI-WS1.21-S	WS1.21-S
21	Right	TI-WS2.21-S	WS2.21-S



3.5mm Wise-Lock Medial Distal Tibia Plate, Without Tab

- Left
- Right

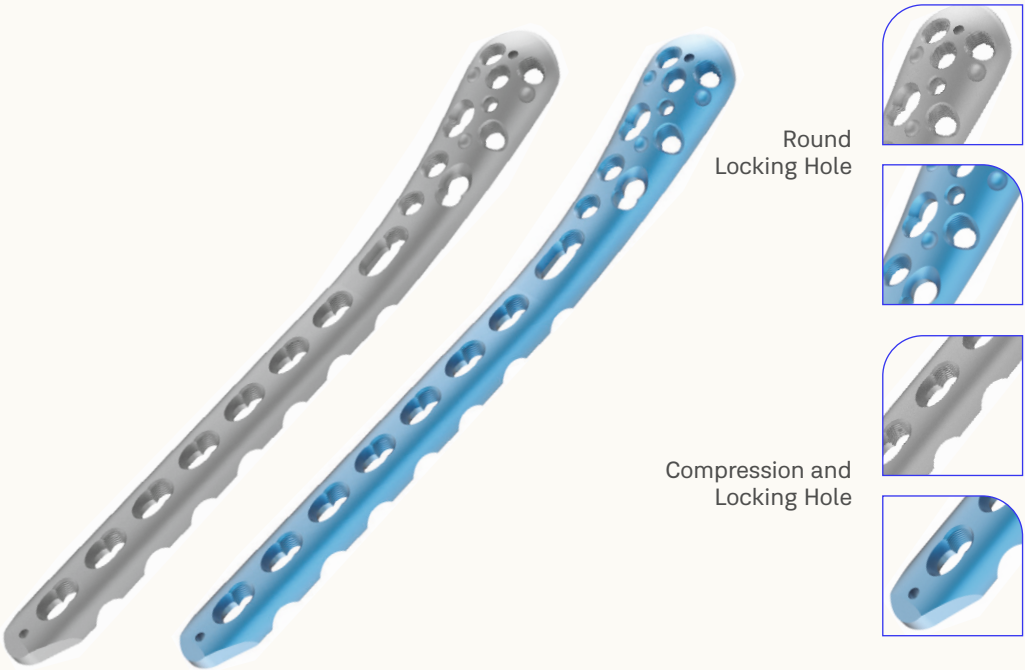

Direction	Holes	Stainless Steel	Titanium
Left	4	WS3.04	TI-WS3.04
Left	6	WS3.06	TI-WS3.06
Left	8	WS3.08	TI-WS3.08
Left	10	WS3.10	TI-WS3.10
Left	12	WS3.12	TI-WS3.12
Left	14	WS3.14	TI-WS3.14
Right	4	WS4.04	TI-WS4.04
Right	6	WS4.06	TI-WS4.06
Right	8	WS4.08	TI-WS4.08
Right	10	WS4.10	TI-WS4.10
Right	12	WS4.12	TI-WS4.12
Right	14	WS4.14	TI-WS4.14

STERILE

Direction	Holes	Stainless Steel	Titanium
Left	4	WS3.04-S	TI-WS3.04-S
Left	6	WS3.06-S	TI-WS3.06-S
Left	8	WS3.08-S	TI-WS3.08-S
Left	10	WS3.10-S	TI-WS3.10-S
Left	12	WS3.12-S	TI-WS3.12-S
Left	14	WS3.14 -S	TI-WS3.14-S
Right	4	WS4.04-S	TI-WS4.04-S
Right	6	WS4.06-S	TI-WS4.06-S
Right	8	WS4.08-S	TI-WS4.08-S
Right	10	WS4.10-S	TI-WS4.10-S
Right	12	WS4.12-S	TI-WS4.12-S
Right	14	WS4.14-S	TI-WS4.14-S





3.5mm Wise-Lock Low Profile Medial Distal Tibia Plate, Without Tab



Left

Right

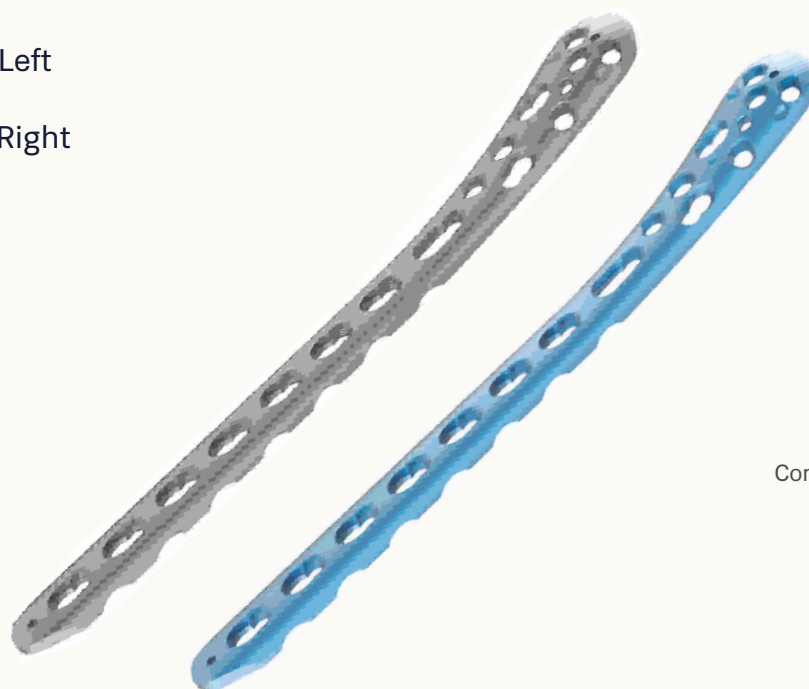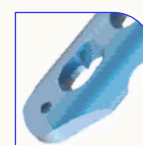Round
Locking HoleCompression and
Locking Hole**STERILE**

Direction	Holes	Stainless Steel	Titanium
Left	4	10-002-04LSS	10-002-04LTI
Left	6	10-002-06LSS	10-002-06LTI
Left	8	10-002-08LSS	10-002-08LTI
Left	10	10-002-10LSS	10-002-10LTI
Left	12	10-002-12LSS	10-002-12LTI
Left	14	10-002-14LSS	10-002-14LTI
Right	4	10-002-04RSS	10-002-04RTI
Right	6	10-002-06RSS	10-002-06RTI
Right	8	10-002-08RSS	10-002-08RTI
Right	10	10-002-10RSS	10-002-10RTI
Right	12	10-002-12RSS	10-002-12RTI
Right	14	10-002-14RSS	10-002-14RTI

Direction	Holes	Stainless Steel	Titanium
Left	4	10-002-04LSS-S	10-002-04LTI-S
Left	6	10-002-06LSS-S	10-002-06LTI-S
Left	8	10-002-08LSS-S	10-002-08LTI-S
Left	10	10-002-10LSS-S	10-002-10LTI-S
Left	12	10-002-12LSS-S	10-002-12LTI-S
Left	14	10-002-14LSS-S	10-002-14LTI-S
Right	4	10-002-04RSS-S	10-002-04RTI-S
Right	6	10-002-06RSS-S	10-002-06RTI-S
Right	8	10-002-08RSS-S	10-002-08RTI-S
Right	10	10-002-10RSS-S	10-002-10RTI-S
Right	12	10-002-12RSS-S	10-002-12RTI-S
Right	14	10-002-14RSS-S	10-002-14RTI-S




3.5mm Wise-lock Metaphyseal Plate For Medial Distal Tibia




Left



Right

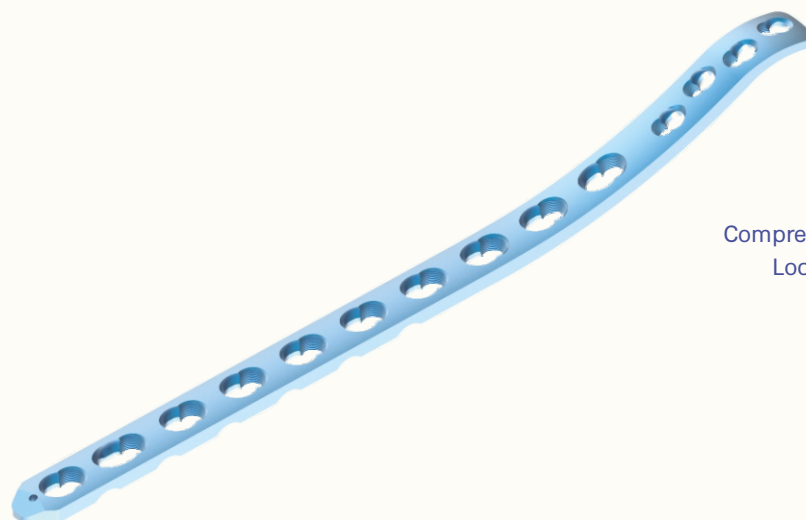

Compression and
Locking Hole

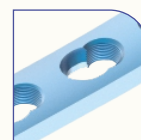

STERILE

Holes	Direction	Titanium	Stainless Steel
4+4	Left	TI-741.104L	741.104L
4+4	Right	TI-741.104R	741.104R
4+5	Left	TI-741.105L	741.105L
4+5	Right	TI-741.105R	741.105R
4+6	Left	TI-741.106L	741.106L
4+6	Right	TI-741.106R	741.106R
4+7	Left	TI-741.107L	741.107L
4+7	Right	TI-741.107R	741.107R
4+8	Left	TI-741.108L	741.108L
4+8	Right	TI-741.108R	741.108R
4+9	Left	TI-741.109L	741.109L
4+9	Right	TI-741.109R	741.109R
4+10	Left	TI-741.110L	741.110L
4+10	Right	TI-741.110R	741.110R
4+12	Left	TI-741.112L	741.112L
4+12	Right	TI-741.112R	741.112R
4+14	Left	TI-741.114L	741.114L
4+14	Right	TI-741.114R	741.114R
4+16	Left	TI-741.116L	741.116L
4+16	Right	TI-741.116R	741.116R
4+18	Left	TI-741.118L	741.118L
4+18	Right	TI-741.118R	741.118R
4+20	Left	TI-741.120L	741.120L
4+20	Right	TI-741.120R	741.120R

Holes	Direction	Titanium	Stainless Steel
4+4	Left	TI-741.104L-S	741.104L-S
4+4	Right	TI-741.104R-S	741.104R-S
4+5	Left	TI-741.105L-S	741.105L-S
4+5	Right	TI-741.105R-S	741.105R-S
4+6	Left	TI-741.106L-S	741.106L-S
4+6	Right	TI-741.106R-S	741.106R-S
4+7	Left	TI-741.107L-S	741.107L-S
4+7	Right	TI-741.107R-S	741.107R-S
4+8	Left	TI-741.108L-S	741.108L-S
4+8	Right	TI-741.108R-S	741.108R-S
4+9	Left	TI-741.109L-S	741.109L-S
4+9	Right	TI-741.109R-S	741.109R-S
4+10	Left	TI-741.110L-S	741.110L-S
4+10	Right	TI-741.110R-S	741.110R-S
4+12	Left	TI-741.112L-S	741.112L-S
4+12	Right	TI-741.112R-S	741.112R-S
4+14	Left	TI-741.114L-S	741.114L-S
4+14	Right	TI-741.114R-S	741.114R-S
4+16	Left	TI-741.116L-S	741.116L-S
4+16	Right	TI-741.116R-S	741.116R-S
4+18	Left	TI-741.118L-S	741.118L-S
4+18	Right	TI-741.118R-S	741.118R-S
4+20	Left	TI-741.120L-S	741.120L-S
4+20	Right	TI-741.120R-S	741.120R-S





3.5mm Wise-Lock Z-Type Distal Medial Tibia Plate



Left



Right

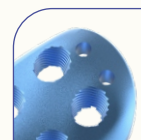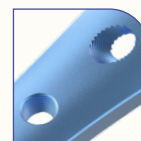Round
Locking Hole**STERILE**


Holes	Direction	Titanium	Stainless Steel
6	Left	10-059-006LTI	10-059-006LSS
6	Right	10-059-006RTI	10-059-006RSS
8	Left	10-059-008LTI	10-059-008LSS
8	Right	10-059-008RTI	10-059-008RSS
10	Left	10-059-010LTI	10-059-010LSS
10	Right	10-059-010RTI	10-059-010RSS
14	Left	10-059-014LTI	10-059-014LSS
14	Right	10-059-014RTI	10-059-014RSS
18	Left	10-059-018LTI	10-059-018LSS
18	Right	10-059-018RTI	10-059-018RSS

Holes	Direction	Titanium	Stainless Steel
6	Left	10-059-006LTI-S	10-059-006LSS-S
6	Right	10-059-006RTI-S	10-059-006RSS-S
8	Left	10-059-008LTI-S	10-059-008LSS-S
8	Right	10-059-008RTI-S	10-059-008RSS-S
10	Left	10-059-010LTI-S	10-059-010LSS-S
10	Right	10-059-010RTI-S	10-059-010RSS-S
14	Left	10-059-014LTI-S	10-059-014LSS-S
14	Right	10-059-014RTI-S	10-059-014RSS-S
18	Left	10-059-018LTI-S	10-059-018LSS-S
18	Right	10-059-018RTI-S	10-059-018RSS-S



3.5/4.5mm Wise-Lock Metaphyseal Plate



- Left
- Right

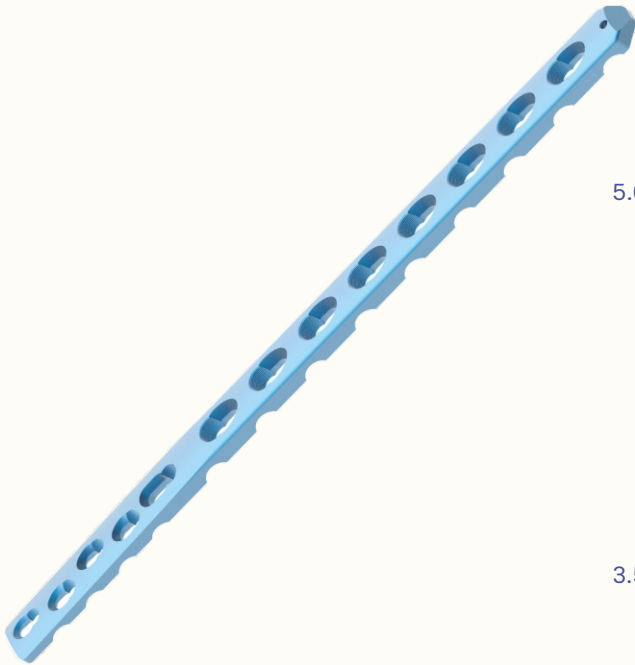

5.0mm Compression
and Locking Hole



3.5mm Compression
and Locking Hole

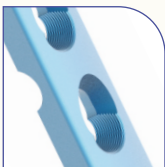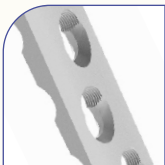

STERILE


Holes	Titanium	Stainless Steel
3/5	TI-740.103	740.103
4/5	TI-740.104	740.104
5/5	TI-740.105	740.105
6/5	TI-740.106	740.106
7/5	TI-740.107	740.107
8/5	TI-740.108	740.108
9/5	TI-740.109	740.109
11/5	TI-740.111	740.111
13/5	TI-740.113	740.113
15/5	TI-740.115	740.115

Holes	Titanium	Stainless Steel
3/5	TI-740.103-S	740.103-S
4/5	TI-740.104-S	740.104-S
5/5	TI-740.105-S	740.105-S
6/5	TI-740.106-S	740.106-S
7/5	TI-740.107-S	740.107-S
8/5	TI-740.108-S	740.108-S
9/5	TI-740.109-S	740.109-S
11/5	TI-740.111-S	740.111-S
13/5	TI-740.113-S	740.113-S
15/5	TI-740.115-S	740.115-S




3.5mm Wise-Lock Metaphyseal Plate

- Left
- Right

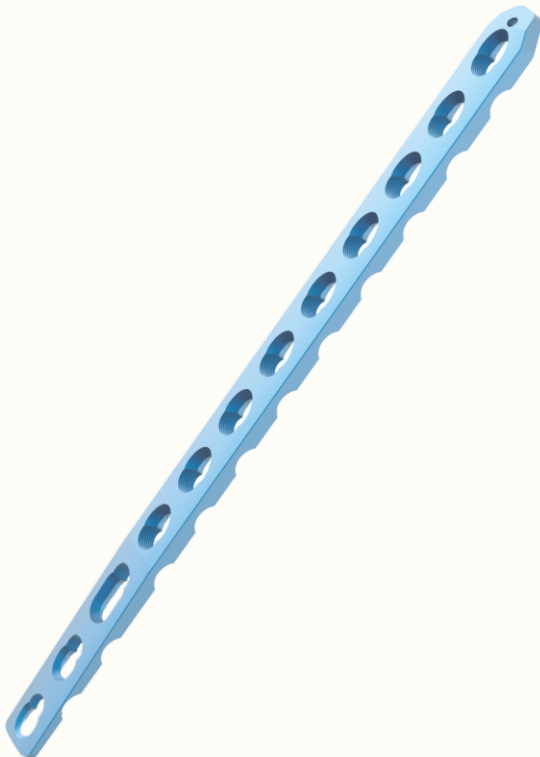

Compression
and Locking Hole

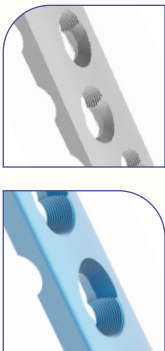

Holes	Titanium	Stainless Steel
6	TI-739.106	739.106
7	TI-739.107	739.107
8	TI-739.108	739.108
9	TI-739.109	739.109
10	TI-739.110	739.110
11	TI-739.111	739.111
12	TI-739.112	739.112
14	TI-739.114	739.114
16	TI-739.116	739.116
18	TI-739.118	739.118

STERILE

Holes	Titanium	Stainless Steel
6	TI-739.106-S	739.106-S
7	TI-739.107-S	739.107-S
8	TI-739.108-S	739.108-S
9	TI-739.109-S	739.109-S
10	TI-739.110-S	739.110-S
11	TI-739.111-S	739.111-S
12	TI-739.112-S	739.112-S
14	TI-739.114-S	739.114-S
16	TI-739.116-S	739.116-S
18	TI-739.118-S	739.118-S



3.5mm Wise-Lock Z-Type Distal Lateral Fibular Plate

Left

Right



Holes	Direction	Titanium	Stainless Steel
4	Left	10-064-004LTI	10-064-004LSS
4	Right	10-064-004RTI	10-064-004RSS
6	Left	10-064-006LTI	10-064-006LSS
6	Right	10-064-006RTI	10-064-006RSS
8	Left	10-064-008LTI	10-064-008LSS
8	Right	10-064-008RTI	10-064-008RSS
10	Left	10-064-010LTI	10-064-010LSS
10	Right	10-064-010RTI	10-064-010RSS

STERILE

Holes	Direction	Titanium	Stainless Steel
4	Left	10-064-004LTI-S	10-064-004LSS
4	Right	10-064-004RTI-S	10-064-004RSS
6	Left	10-064-006LTI-S	10-064-006LSS
6	Right	10-064-006RTI-S	10-064-006RSS
8	Left	10-064-008LTI-S	10-064-008LSS
8	Right	10-064-008RTI-S	10-064-008RSS
10	Left	10-064-010LTI-S	10-064-010LSS
10	Right	10-064-010RTI-S	10-064-010RSS




3.5mm Wise-Lock Lateral Distal Fibula Plate



Left




Right

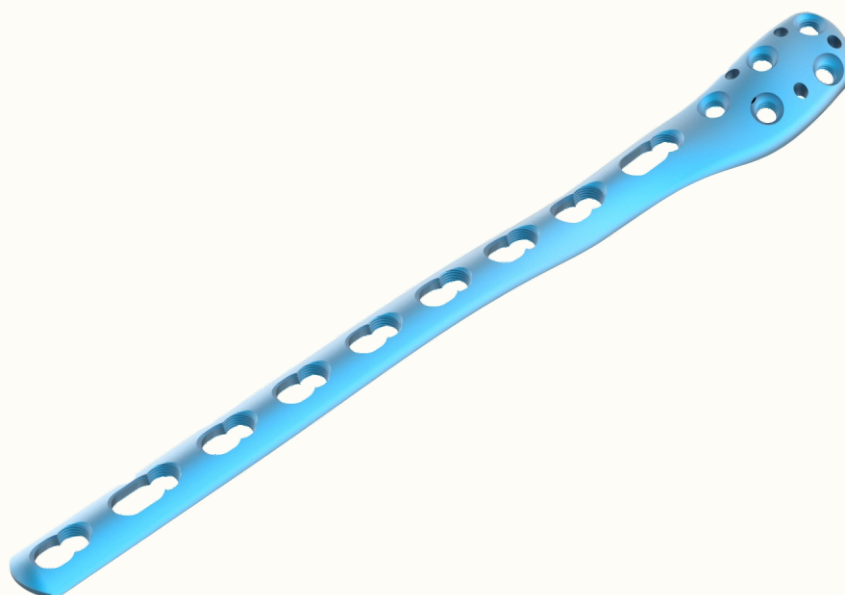

Holes	Direction	Titanium	Stainless Steel
3	Left	TI-584.003L	584.003L
3	Right	TI-584.003R	584.003R
4	Left	TI-584.004L	584.004L
4	Right	TI-584.004R	584.004R
5	Left	TI-584.005L	584.005L
5	Right	TI-584.005R	584.005R
6	Left	TI-584.006L	584.006L
6	Right	TI-584.006R	584.006R
7	Left	TI-584.007L	584.007L
7	Right	TI-584.007R	584.007R
9	Left	TI-584.009L	584.009L
9	Right	TI-584.009R	584.009R
11	Left	TI-584.011L	584.011L
11	Right	TI-584.011R	584.011R

STERILE

Holes	Direction	Titanium	Stainless Steel
3	Left	TI-584.003L-S	584.003L-S
3	Right	TI-584.003R-S	584.003R-S
4	Left	TI-584.004L-S	584.004L-S
4	Right	TI-584.004R-S	584.004R-S
5	Left	TI-584.005L-S	584.005L-S
5	Right	TI-584.005R-S	584.005R-S
6	Left	TI-584.006L-S	584.006L-S
6	Right	TI-584.006R-S	584.006R-S
7	Left	TI-584.007L-S	584.007L-S
7	Right	TI-584.007R-S	584.007R-S
9	Left	TI-584.009L-S	584.009L-S
9	Right	TI-584.009R-S	584.009R-S
11	Left	TI-584.011L-S	584.011L-S
11	Right	TI-584.011R-S	584.011R-S

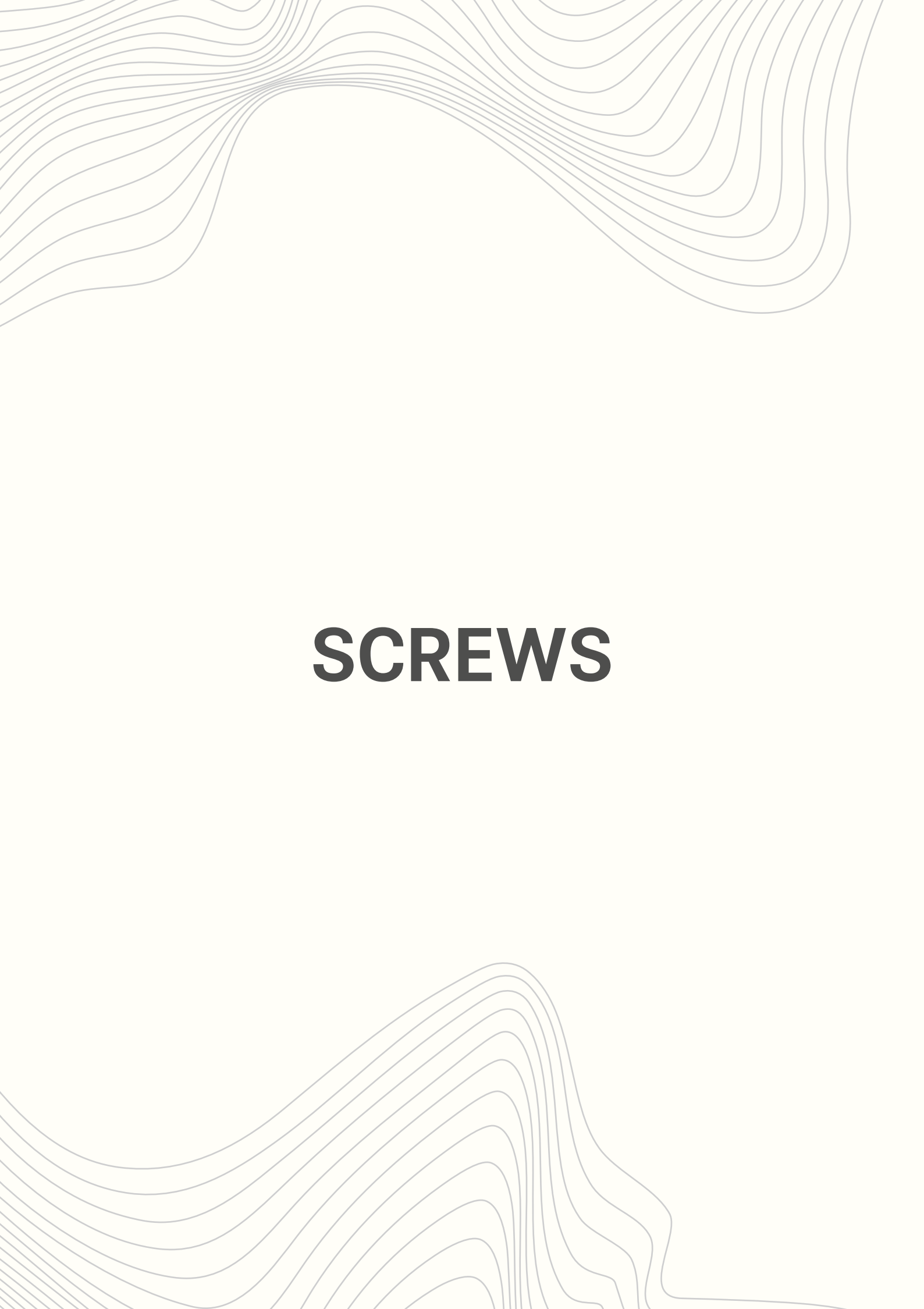The background of the page features a series of thin, light gray wavy lines that flow from the top left towards the right, creating a sense of movement and depth. These lines are more densely packed in some areas, creating a subtle gradient effect. The overall aesthetic is clean and modern.

SCREWS




3.5mm Wise-Lock Screw, Self-Tapping (Hex Head)


STERILE


Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.010	117.010
Ø3.5	12	TI-117.012	117.012
Ø3.5	14	TI-117.014	117.014
Ø3.5	16	TI-117.016	117.016
Ø3.5	18	TI-117.018	117.018
Ø3.5	20	TI-117.020	117.020
Ø3.5	22	TI-117.022	117.022
Ø3.5	24	TI-117.024	117.024
Ø3.5	26	TI-117.026	117.026
Ø3.5	28	TI-117.028	117.028
Ø3.5	30	TI-117.030	117.030
*	Ø3.5	32	TI-117.032
*	Ø3.5	34	TI-117.034
	Ø3.5	35	TI-117.035
*	Ø3.5	36	TI-117.036
*	Ø3.5	38	TI-117.038
	Ø3.5	40	TI-117.040
*	Ø3.5	42	TI-117.042
*	Ø3.5	44	TI-117.044
	Ø3.5	45	TI-117.045
*	Ø3.5	46	TI-117.046
*	Ø3.5	48	TI-117.048
	Ø3.5	50	TI-117.050
*	Ø3.5	52	TI-117.052
*	Ø3.5	54	TI-117.054
	Ø3.5	55	TI-117.055
*	Ø3.5	56	TI-117.056
*	Ø3.5	58	TI-117.058
	Ø3.5	60	TI-117.060
	Ø3.5	65	TI-117.065
	Ø3.5	70	TI-117.070
	Ø3.5	75	TI-117.075
	Ø3.5	80	TI-117.080

Dia	Length	Titanium	Stainless Steel
Ø3.5	10	TI-117.010-S	117.010-S
Ø3.5	12	TI-117.012-S	117.012-S
Ø3.5	14	TI-117.014-S	117.014-S
Ø3.5	16	TI-117.016-S	117.016-S
Ø3.5	18	TI-117.018-S	117.018-S
Ø3.5	20	TI-117.020-S	117.020-S
Ø3.5	22	TI-117.022-S	117.022-S
Ø3.5	24	TI-117.024-S	117.024-S
Ø3.5	26	TI-117.026-S	117.026-S
Ø3.5	28	TI-117.028-S	117.028-S
Ø3.5	30	TI-117.030-S	117.030-S
*	Ø3.5	32	TI-117.032-S
*	Ø3.5	34	TI-117.034-S
	Ø3.5	35	TI-117.035-S
*	Ø3.5	36	TI-117.036-S
*	Ø3.5	38	TI-117.038-S
	Ø3.5	40	TI-117.040-S
*	Ø3.5	42	TI-117.042-S
*	Ø3.5	44	TI-117.044-S
	Ø3.5	45	TI-117.045-S
*	Ø3.5	46	TI-117.046-S
*	Ø3.5	48	TI-117.048-S
	Ø3.5	50	TI-117.050-S
*	Ø3.5	52	TI-117.052-S
*	Ø3.5	54	TI-117.054-S
	Ø3.5	55	TI-117.055-S
*	Ø3.5	56	TI-117.056-S
*	Ø3.5	58	TI-117.058-S
	Ø3.5	60	TI-117.060-S
	Ø3.5	65	TI-117.065-S
	Ø3.5	70	TI-117.070-S
	Ø3.5	75	TI-117.075-S
	Ø3.5	80	TI-117.080-S

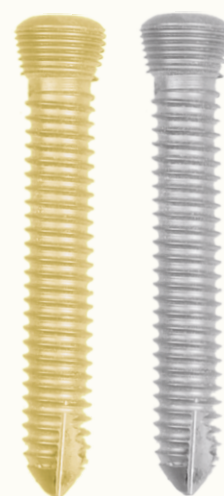

* Sizes not available in Screw Caddy

* Sizes not available in Screw Caddy




3.5mm Cortical Screw, Self-Tapping, (Hex Head)


STERILE

Dia	Length	Titanium	Stainless Steel
Ø3.5	10mm	TI-104.210	104.210
Ø3.5	12mm	TI-104.212	104.212
Ø3.5	14mm	TI-104.214	104.214
Ø3.5	16mm	TI-104.216	104.216
Ø3.5	18mm	TI-104.218	104.218
Ø3.5	20mm	TI-104.220	104.220
Ø3.5	22mm	TI-104.222	104.222
Ø3.5	24mm	TI-104.224	104.224
Ø3.5	26mm	TI-104.226	104.226
Ø3.5	28mm	TI-104.228	104.228
Ø3.5	30mm	TI-104.230	104.230
Ø3.5	32mm	TI-104.232	104.232
Ø3.5	34mm	TI-104.234	104.234
Ø3.5	36mm	TI-104.236	104.236
Ø3.5	38mm	TI-104.238	104.238
Ø3.5	40mm	TI-104.240	104.240
Ø3.5	42mm	TI-104.242	104.242
Ø3.5	44mm	TI-104.244	104.244
Ø3.5	46mm	TI-104.246	104.246
Ø3.5	48mm	TI-104.248	104.248
Ø3.5	50mm	TI-104.250	104.250
*	Ø3.5	55mm	TI-104.255
*	Ø3.5	60mm	TI-104.260
*	Ø3.5	65mm	TI-104.265
*	Ø3.5	70mm	TI-104.270
*	Ø3.5	75mm	TI-104.275
*	Ø3.5	80mm	TI-104.280
*	Ø3.5	85mm	TI-104.285
*	Ø3.5	90mm	TI-104.290

Dia	Length	Titanium	Stainless Steel
Ø3.5	10mm	TI-104.210-S	104.210-S
Ø3.5	12mm	TI-104.212-S	104.212-S
Ø3.5	14mm	TI-104.214-S	104.214-S
Ø3.5	16mm	TI-104.216-S	104.216-S
Ø3.5	18mm	TI-104.218-S	104.218-S
Ø3.5	20mm	TI-104.220-S	104.220-S
Ø3.5	22mm	TI-104.222-S	104.222-S
Ø3.5	24mm	TI-104.224-S	104.224-S
Ø3.5	26mm	TI-104.226-S	104.226-S
Ø3.5	28mm	TI-104.228-S	104.228-S
Ø3.5	30mm	TI-104.230-S	104.230-S
Ø3.5	32mm	TI-104.232-S	104.232-S
Ø3.5	34mm	TI-104.234-S	104.234-S
Ø3.5	36mm	TI-104.236-S	104.236-S
Ø3.5	38mm	TI-104.238-S	104.238-S
Ø3.5	40mm	TI-104.240-S	104.240-S
Ø3.5	42mm	TI-104.242-S	104.242-S
Ø3.5	44mm	TI-104.244-S	104.244-S
Ø3.5	46mm	TI-104.246-S	104.246-S
Ø3.5	48mm	TI-104.248-S	104.248-S
Ø3.5	50mm	TI-104.250-S	104.250-S
*	Ø3.5	55mm	TI-104.255-S
*	Ø3.5	60mm	TI-104.260-S
*	Ø3.5	65mm	TI-104.265-S
*	Ø3.5	70mm	TI-104.270-S
*	Ø3.5	75mm	TI-104.275-S
*	Ø3.5	80mm	TI-104.280-S
*	Ø3.5	85mm	TI-104.285-S
*	Ø3.5	90mm	TI-104.290-S

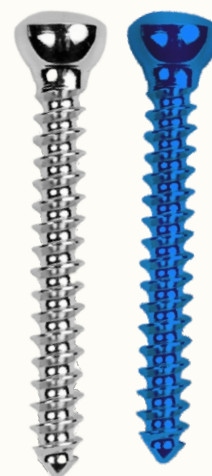

* Sizes not available in Screw Caddy

* Sizes not available in Screw Caddy




2.7mm Wise-Lock Screw, Self-Tapping (Hex Head)



Dia	Length	Titanium	Stainless Steel
Ø2.7	10mm	TI-118.010	118.010
Ø2.7	12mm	TI-118.012	118.012
Ø2.7	14mm	TI-118.014	118.014
Ø2.7	16mm	TI-118.016	118.016
Ø2.7	18mm	TI-118.018	118.018
Ø2.7	20mm	TI-118.020	118.020
Ø2.7	22mm	TI-118.022	118.022
Ø2.7	24mm	TI-118.024	118.024
Ø2.7	26mm	TI-118.026	118.026
Ø2.7	28mm	TI-118.028	118.028
Ø2.7	30mm	TI-118.030	118.030
Ø2.7	32mm	TI-118.032	118.032
Ø2.7	34mm	TI-118.034	118.034
Ø2.7	36mm	TI-118.036	118.036
Ø2.7	38mm	TI-118.038	118.038
Ø2.7	40mm	TI-118.040	118.040
Ø2.7	42mm	TI-118.042	118.042
Ø2.7	44mm	TI-118.044	118.044
Ø2.7	46mm	TI-118.046	118.046
Ø2.7	48mm	TI-118.048	118.048
Ø2.7	50mm	TI-118.050	118.050
Ø2.7	52mm	TI-118.052	118.052
Ø2.7	54mm	TI-118.054	118.054
Ø2.7	56mm	TI-118.056	118.056
Ø2.7	58mm	TI-118.058	118.058
Ø2.7	60mm	TI-118.060	118.060

STERILE

Dia	Length	Titanium	Stainless Steel
Ø2.7	10mm	TI-118.010-S	118.010-S
Ø2.7	12mm	TI-118.012-S	118.012-S
Ø2.7	14mm	TI-118.014-S	118.014-S
Ø2.7	16mm	TI-118.016-S	118.016-S
Ø2.7	18mm	TI-118.018-S	118.018-S
Ø2.7	20mm	TI-118.020-S	118.020-S
Ø2.7	22mm	TI-118.022-S	118.022-S
Ø2.7	24mm	TI-118.024-S	118.024-S
Ø2.7	26mm	TI-118.026-S	118.026-S
Ø2.7	28mm	TI-118.028-S	118.028-S
Ø2.7	30mm	TI-118.030-S	118.030-S
Ø2.7	32mm	TI-118.032-S	118.032-S
Ø2.7	34mm	TI-118.034-S	118.034-S
Ø2.7	36mm	TI-118.036-S	118.036-S
Ø2.7	38mm	TI-118.038-S	118.038-S
Ø2.7	40mm	TI-118.040-S	118.040-S
Ø2.7	42mm	TI-118.042-S	118.042-S
Ø2.7	44mm	TI-118.044-S	118.044-S
Ø2.7	46mm	TI-118.046-S	118.046-S
Ø2.7	48mm	TI-118.048-S	118.048-S
Ø2.7	50mm	TI-118.050-S	118.050-S
Ø2.7	52mm	TI-118.052-S	118.052-S
Ø2.7	54mm	TI-118.054-S	118.054-S
Ø2.7	56mm	TI-118.056-S	118.056-S
Ø2.7	58mm	TI-118.058-S	118.058-S
Ø2.7	60mm	TI-118.060-S	118.060-S

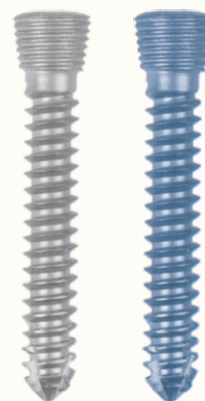



5.0mm Wise-Lock Screw, Self-Tapping (Hex Head)



Dia	Length	Titanium	Stainless Steel
Ø5.0	12mm	TI-119.012	119.012
Ø5.0	14mm	TI-119.014	119.014
Ø5.0	16mm	TI-119.016	119.016
Ø5.0	18mm	TI-119.018	119.018
Ø5.0	20mm	TI-119.020	119.020
Ø5.0	22mm	TI-119.022	119.022
Ø5.0	24mm	TI-119.024	119.024
Ø5.0	26mm	TI-119.026	119.026
Ø5.0	28mm	TI-119.028	119.028
Ø5.0	30mm	TI-119.030	119.030
Ø5.0	32mm	TI-119.032	119.032
Ø5.0	34mm	TI-119.034	119.034
Ø5.0	36mm	TI-119.036	119.036
Ø5.0	38mm	TI-119.038	119.038
Ø5.0	40mm	TI-119.040	119.040
Ø5.0	42mm	TI-119.042	119.042
Ø5.0	44mm	TI-119.044	119.044
Ø5.0	46mm	TI-119.046	119.046
Ø5.0	48mm	TI-119.048	119.048
Ø5.0	50mm	TI-119.050	119.050
Ø5.0	52mm	TI-119.052	119.052
Ø5.0	54mm	TI-119.054	119.054
Ø5.0	56mm	TI-119.056	119.056
Ø5.0	58mm	TI-119.058	119.058
Ø5.0	60mm	TI-119.060	119.060
Ø5.0	65mm	TI-119.065	119.065
Ø5.0	70mm	TI-119.070	119.070
Ø5.0	75mm	TI-119.075	119.075
Ø5.0	80mm	TI-119.080	119.080
Ø5.0	85mm	TI-119.085	119.085
Ø5.0	90mm	TI-119.090	119.090


STERILE

Dia	Length	Titanium	Stainless Steel
Ø5.0	12mm	TI-119.012-S	119.012-S
Ø5.0	14mm	TI-119.014-S	119.014-S
Ø5.0	16mm	TI-119.016-S	119.016-S
Ø5.0	18mm	TI-119.018-S	119.018-S
Ø5.0	20mm	TI-119.020-S	119.020-S
Ø5.0	22mm	TI-119.022-S	119.022-S
Ø5.0	24mm	TI-119.024-S	119.024-S
Ø5.0	26mm	TI-119.026-S	119.026-S
Ø5.0	28mm	TI-119.028-S	119.028-S
Ø5.0	30mm	TI-119.030-S	119.030-S
Ø5.0	32mm	TI-119.032-S	119.032-S
Ø5.0	34mm	TI-119.034-S	119.034-S
Ø5.0	36mm	TI-119.036-S	119.036-S
Ø5.0	38mm	TI-119.038-S	119.038-S
Ø5.0	40mm	TI-119.040-S	119.040-S
Ø5.0	42mm	TI-119.042-S	119.042-S
Ø5.0	44mm	TI-119.044-S	119.044-S
Ø5.0	46mm	TI-119.046-S	119.046-S
Ø5.0	48mm	TI-119.048-S	119.048-S
Ø5.0	50mm	TI-119.050-S	119.050-S
Ø5.0	52mm	TI-119.052-S	119.052-S
Ø5.0	54mm	TI-119.054-S	119.054-S
Ø5.0	56mm	TI-119.056-S	119.056-S
Ø5.0	58mm	TI-119.058-S	119.058-S
Ø5.0	60mm	TI-119.060-S	119.060-S
Ø5.0	65mm	TI-119.065-S	119.065-S
Ø5.0	70mm	TI-119.070-S	119.070-S
Ø5.0	75mm	TI-119.075-S	119.075-S
Ø5.0	80mm	TI-119.080-S	119.080-S
Ø5.0	85mm	TI-119.085-S	119.085-S
Ø5.0	90mm	TI-119.090-S	119.090-S





4.5mm Cortical Screw, Self-Tapping (Hex Head)

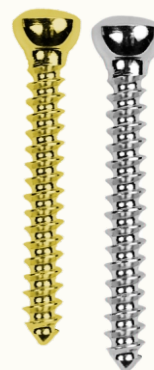

STERILE


	Dia	Length	Titanium	Stainless Steel
*	Ø4.5	12mm	TI-105.212	105.212
*	Ø4.5	14mm	TI-105.214	105.214
*	Ø4.5	16mm	TI-105.216	105.216
*	Ø4.5	18mm	TI-105.218	105.218
	Ø4.5	20mm	TI-105.220	105.220
	Ø4.5	22mm	TI-105.222	105.222
	Ø4.5	24mm	TI-105.224	105.224
	Ø4.5	26mm	TI-105.226	105.226
	Ø4.5	28mm	TI-105.228	105.228
	Ø4.5	30mm	TI-105.230	105.230
	Ø4.5	32mm	TI-105.232	105.232
	Ø4.5	34mm	TI-105.234	105.234
	Ø4.5	36mm	TI-105.236	105.236
	Ø4.5	38mm	TI-105.238	105.238
	Ø4.5	40mm	TI-105.240	105.240
	Ø4.5	42mm	TI-105.242	105.242
	Ø4.5	44mm	TI-105.244	105.244
	Ø4.5	46mm	TI-105.246	105.246
	Ø4.5	48mm	TI-105.248	105.248
	Ø4.5	50mm	TI-105.250	105.250
	Ø4.5	52mm	TI-105.252	105.252
	Ø4.5	54mm	TI-105.254	105.254
*	Ø4.5	55mm	TI-105.255	105.255
	Ø4.5	56mm	TI-105.256	105.256
	Ø4.5	58mm	TI-105.258	105.258
	Ø4.5	60mm	TI-105.260	105.260
	Ø4.5	65mm	TI-105.265	105.265
	Ø4.5	70mm	TI-105.270	105.270
	Ø4.5	80mm	TI-105.280	105.280
*	Ø4.5	85mm	TI-105.285	105.285
*	Ø4.5	90mm	TI-105.290	105.290

	Dia	Length	Titanium	Stainless Steel
	Ø5.0	12mm	TI-105.212-S	105.212-S
	Ø5.0	14mm	TI-105.214-S	105.214-S
	Ø5.0	16mm	TI-105.216-S	105.216-S
	Ø5.0	18mm	TI-105.218-S	105.218-S
	Ø5.0	20mm	TI-105.220-S	105.220-S
	Ø5.0	22mm	TI-105.222-S	105.222-S
	Ø5.0	24mm	TI-105.224-S	105.224-S
	Ø5.0	26mm	TI-105.226-S	105.226-S
	Ø5.0	28mm	TI-105.228-S	105.228-S
	Ø5.0	30mm	TI-105.230-S	105.230-S
	Ø5.0	32mm	TI-105.232-S	105.232-S
	Ø5.0	34mm	TI-105.234-S	105.234-S
	Ø5.0	36mm	TI-105.236-S	105.236-S
	Ø5.0	38mm	TI-105.238-S	105.238-S
	Ø5.0	40mm	TI-105.240-S	105.240-S
	Ø5.0	42mm	TI-105.242-S	105.242-S
	Ø5.0	44mm	TI-105.244-S	105.244-S
	Ø5.0	46mm	TI-105.246-S	105.246-S
	Ø5.0	48mm	TI-105.248-S	105.248-S
	Ø5.0	50mm	TI-105.250-S	105.250-S
	Ø5.0	52mm	TI-105.252-S	105.252-S
	Ø5.0	54mm	TI-105.254-S	105.254-S
	Ø5.0	56mm	TI-105.255-S	105.255-S
	Ø5.0	58mm	TI-105.256-S	105.256-S
	Ø5.0	60mm	TI-105.258-S	105.258-S
	Ø5.0	65mm	TI-105.260-S	105.260-S
	Ø5.0	70mm	TI-105.265-S	105.265-S
	Ø5.0	75mm	TI-105.270-S	105.270-S
	Ø5.0	80mm	TI-105.280-S	105.280-S
	Ø5.0	85mm	TI-105.285-S	105.285-S
	Ø5.0	90mm	TI-105.290-S	105.290-S

* Sizes not available in Screw Caddy

* Sizes not available in Screw Caddy

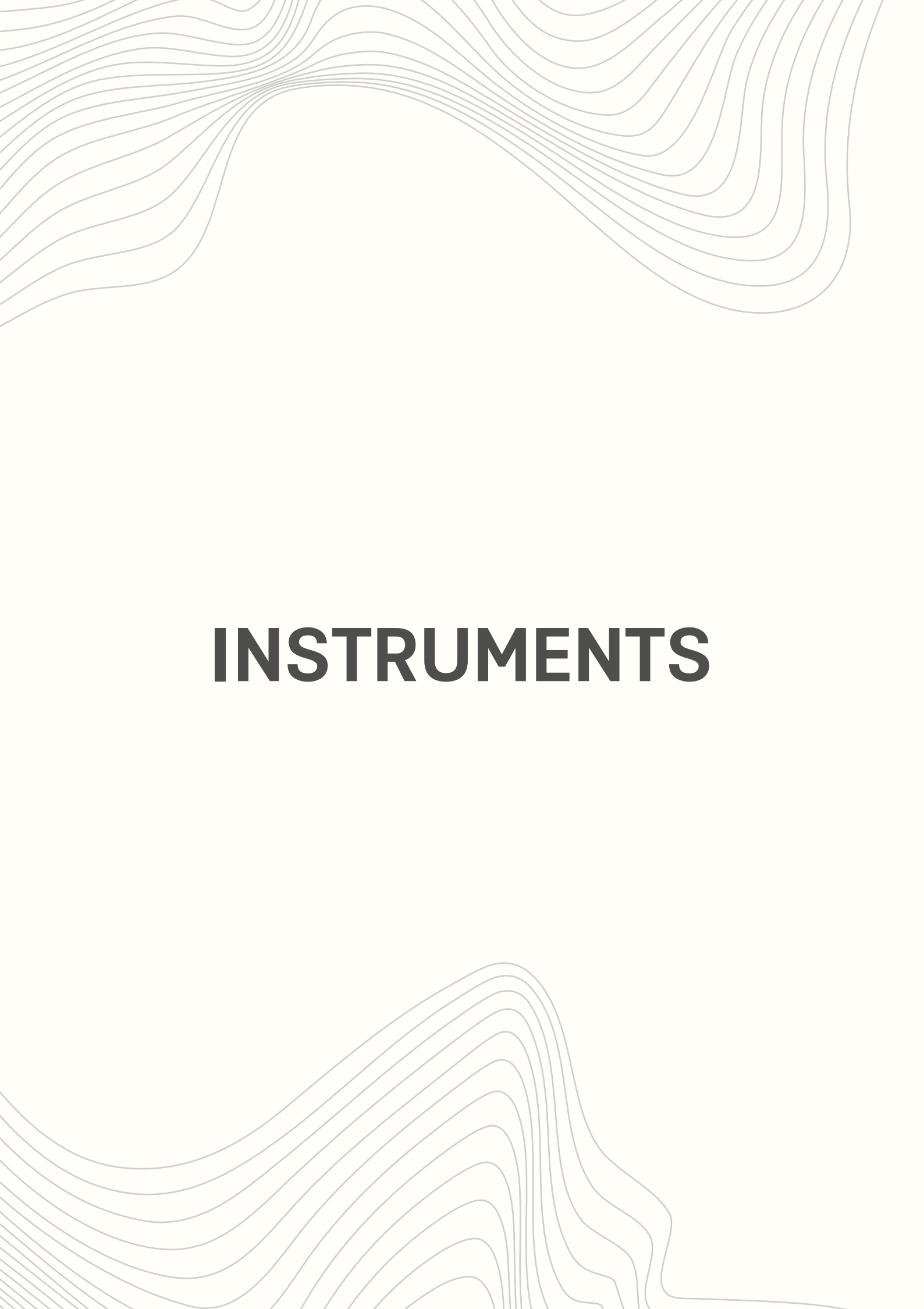The background of the page features a series of thin, light gray, wavy lines that flow from the top left towards the bottom right, creating a sense of movement and depth. These lines are more densely packed in some areas, particularly near the top and bottom edges, and more spread out in the center.

INSTRUMENTS



3443-37

Depth Gauge Measuring Upto 80mm For Small Fragment



3445-2.5

T-handle Screwdriver, Hex 2.5mm, Self-retaining For Small Fragment




3992-035

Allen Key, Hex 3.0mm, For Small Fragment




2100-2.0-110

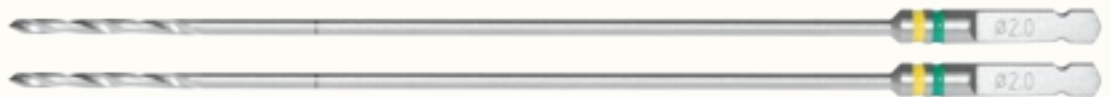Drill Bit With Quick Coupling End, Ø2.0mm X Length 110mm,
For Small Fragment

2100-2.5-112

Drill Bit With Quick Coupling End, Ø2.5mm X Length 112mm,
For Small Fragment


2100-3.5-112

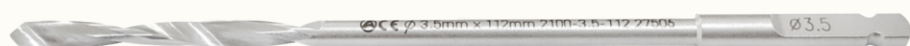Drill Bit With Quick Coupling End, Ø3.5mm X Length 112mm,
For Small Fragment



2103-2.8-165

Drill Bit Quick Coupling With Stopper, Ø2.8mm X Length 165mm, For Small Fragment

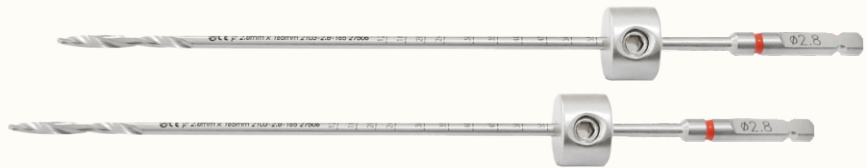

2104-27

Bone Tap Quick Coupling For Wise-lock Screws, Ø2.7mm, For Small Fragment



2104-02

Bone Tap Quick Coupling For Cortical Screws, Ø3.5mm, For Small Fragment



2104-04

Bone Tap, Quick Coupling For Cancellous Screws, Ø4.0mm, For Small Fragment




3400-01

Countersink For Ø3.5/4.0mm Screws, Quick Coupling, For Small Fragment



3408-01


Hexagonal Screwdriver Shaft - 2.5mm Tip, Quick Coupling, For Small Fragment

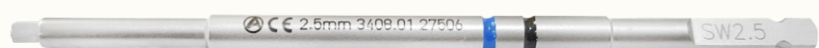



3408-03

Hexagonal Screwdriver Shaft - 2.0mm Tip, Quick Coupling,
For Small Fragment

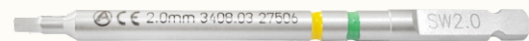

2186-2.5

Hss Drill Bit, Ø2.5mm, For Small Fragment



2106-1.2

Guide Sleeve For Ø1.2mm K. Wires, For Small Fragment



3443-05

Depth Gauge With Protector Measuring Upto 50mm For Small Fragment



3443-39

Trephine For Small Fragment



BT-SF-06

Bending Template, Small, For Small Fragment





BT-SF-08

Bending Template, Medium, For Small Fragment



BT-SF-10

Bending Template, Large, For Small Fragment



3402-000

T-handle With Quick Coupling




1472-054

Quick Coupling Shaft



TDG-2.7

Threaded Drill Sleeve For Drill Bit Ø2.0mm - Small Fragment

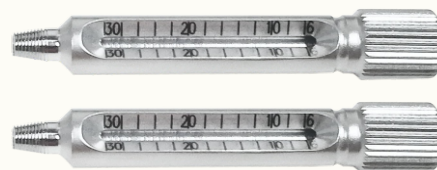

7-050-04

Threaded Drill Sleeve For Drill Bit Ø2.8 For Wise-lock Small Fragment For T-handle





3420-01

Drill Sleeve Insert Ø3.5/2.5mm For Small Fragment



1472-036

Drill Guide 2.0mm For Small Fragment



1472-044

Double Drill Guide Ø2.0/2.7mm For Small Fragment



1472-046

Self-centering Double Drill Guide, Ø2.5/3.5mm, For Small Fragment



3441-16

Drill Guide For Neutral And Loaded Position Ø3.5mm, For Small Fragment



1472-066

Hohmann Retractor, 6.5mm, For Small Fragment





1472-068

Hohmann Retractor, 8.5mm, For Small Fragment



2146-018

Hohmann Retractor, 15.5mm, For Small Fragment



2149-1012

Periosteal Elevator, Straight, 12mm, For Small Fragment



3406-02

Hexagonal Screwdriver - 2.0mm Tip For Small Fragment



3406-02S

Screw Holding Sleeve For 2.0mm Tip Screwdriver, For Small Fragment



3406-03

Hexagonal Screwdriver - 2.5mm Tip, For Small Fragment





3406-03S

Screw Holding Sleeve For 2.5mm Tip Screwdriver, For Small Fragment



3409-03L

Bending Iron, Left For Small Fragment



3409-03R

Bending Iron, Right For Small Fragment



2107-1180

Reduction Forcep, Pointed, Ratchet Lock, 180mm, For Small Fragment



2106-1160

Reduction Forcep, Serrated Jaws, Speed Lock, 160mm, For Small Fragment



2106-190

Self-centering Bone Holding Forcep, Speed Lock, 190mm, For Small Fragment





2150-1006

Periosteal Elevator With Silicon Handle, Curved, 6mm, For Small Fragment



TQ-2.0

Torque Limiting Attachment, 0.8nm, For Small Fragment



TQ-2.5

Torque Limiting Attachment, 1.5nm, For Small Fragment




1472-064

Torque Screwdriver Handle



GW-1.2-230

Guide Wire With Threaded Trocar Tip, Ø1.2mm X Thread Length 10mm X Length 230mm

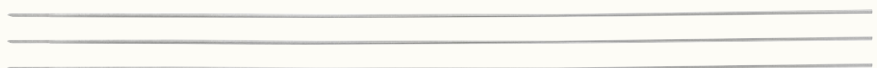

7-050-05

T- Handle For Threaded Drill Sleeve






3415-000

Screw Holding Forcep




ST-007NW

Screw Caddy For 3.5mm Wise-lock Small Fragment System

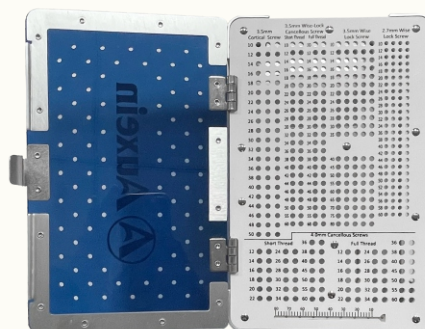


7-050-03

Instrument Trays For 3.5mm Wise-lock Small Fragment Instrument Set

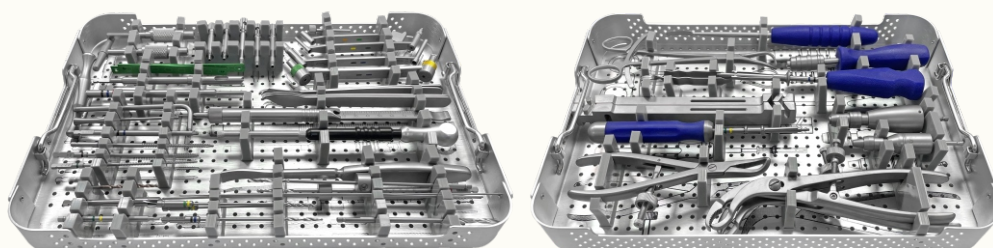

IC-2310-WL

Container For 3.5mm Wise-lock Small Fragment Instrument Set

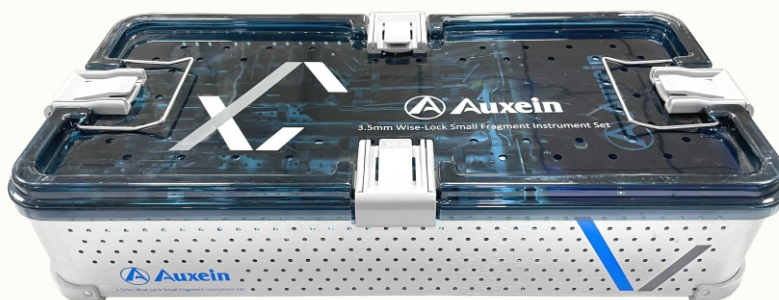



2302-000 : Small Fragment Wise-Lock Instrument Set





ORDER INFORMATION

2302-000	Small Fragment Wise-Lock Instrument Set
----------	---

Code	Set Consisting Of	Unit
3443-37	Depth Gauge measuring upto 80mm for Small Fragment	1
3445-2.5	T-Handle Screwdriver, Hex 2.5mm, Self-Retaining for Small Fragment	1
3992-035	Allen Key, Hex 3.0mm, for Small Fragment	1
2100-2.0-110	Drill Bit with Quick Coupling End, Ø2.0mm x Length 110mm, for Small Fragment	2
2100-2.5-112	Drill Bit with Quick Coupling End, Ø2.5mm x Length 112mm, for Small Fragment	1
2100-3.5-112	Drill Bit with Quick Coupling End, Ø3.5mm x Length 112mm, for Small Fragment	1
2103-2.8-165	Drill Bit Quick Coupling with Stopper, Ø2.8mm x Length 165mm, for Small Fragment	2
2104-27	Bone Tap Quick Coupling for Wise-Lock Screws, Ø2.7mm, for Small Fragment	1
2104-02	Bone Tap Quick Coupling for Cortical Screws, Ø3.5mm, for Small Fragment	1
2104-04	Bone Tap, Quick Coupling for Cancellous Screws, Ø4.0mm, for Small Fragment	1
3400-01	Countersink for Ø3.5/4.0mm Screws, Quick Coupling, for Small Fragment	1
3408-01	Hexagonal Screwdriver Shaft - 2.5mm Tip, Quick Coupling, for Small Fragment	1
3408-03	Hexagonal Screwdriver Shaft - 2.0mm Tip, Quick Coupling, for Small Fragment	1
2186-2.5	HSS Drill Bit, Ø2.5mm, for Small Fragment	1
2106-1.2	Guide Sleeve for Ø1.2mm K. Wires, , for Small Fragment	1
3443-05	Depth Gauge with Protector measuring upto 50mm for Small Fragment	1
3443-39	Trephine for Small Fragment	1
BT-SF-06	Bending Template, Small, for Small Fragment	1
BT-SF-08	Bending Template, Medium, for Small Fragment	1
BT-SF-10	Bending Template, Large, for Small Fragment	1
3402-000	T-Handle with Quick Coupling	1
1472-054	Quick Coupling Shaft	1
TDG-2.7	Threaded Drill Sleeve for Drill Bit Ø2.0mm - Small Fragment	2
7-050-04	Threaded Drill Sleeve for Drill Bit Ø2.8 for Wise-lock Small Fragment for T-Handle	3
3420-01	Drill Sleeve Insert Ø3.5/2.5mm for Small Fragment	1
1472-036	Drill Guide 2.0mm for Small Fragment	1
1472-044	Double Drill Guide Ø2.0/2.7mm for Small Fragment	1
1472-046	Self-Centering Double Drill Guide, Ø2.5/3.5mm, for Small Fragment	1
3441-16	Drill Guide for Neutral and Loaded Position Ø3.5mm, for Small Fragment	1
1472-066	Hohmann Retractor, 6.5mm, for Small Fragment	1
1472-068	Hohmann Retractor, 8.5mm, for Small Fragment	1
2146-018	Hohmann Retractor, 15.5mm, for Small Fragment	2
2149-1012	Periosteal Elevator, Straight, 12mm, for Small Fragment	1
3406-02	Hexagonal Screwdriver - 2.0mm Tip for Small Fragment	1
3406-02S	Screw Holding Sleeve for 2.0mm Tip Screwdriver, for Small Fragment	1
3406-03	Hexagonal Screwdriver - 2.5mm Tip, for Small Fragment	1
3406-03S	Screw Holding Sleeve for 2.5mm Tip Screwdriver, for Small Fragment	1
3409-03L	Bending Iron, Left for Small Fragment	1
3409-03R	Bending Iron, Right for Small Fragment	1
2107-1180	Reduction Forcep, Pointed, Ratchet Lock, 180mm, for Small Fragment	1
2106-1160	Reduction Forcep, Serrated Jaws, Speed Lock, 160mm, for Small Fragment	1

**1138-11****Small Fragment Wise-Lock Instrument Set**

Code	Set Consisting Of	Unit
2106-190	Self-Centering Bone Holding Forcep, Speed Lock, 190mm, for Small Fragment	2
2150-1006	Periosteal Elevator with Silicon Handle, Curved, 6mm, for Small Fragment	1
TQ-2.0	Torque Limiting Attachment, 0.8Nm, for Small Fragment	1
TQ-2.5	Torque Limiting Attachment, 1.5Nm, for Small Fragment	1
1472-064	Torque Screwdriver Handle	1
GW-1.2-230	Guide Wire with Threaded Trocar Tip, Ø1.2mm x Thread Length 10mm x Length 230mm	3
7-050-05	T- Handle for Threaded Drill Sleeve	1
3415-000	Screw Holding Forcep	1
ST-007NW	Screw Caddy for 3.5mm Wise-Lock Small Fragment System	1
7-050-03	Instrument Trays for 3.5mm Wise-Lock Small Fragment Instrument Set	2
IC-2310-WL	Container for 3.5mm Wise-Lock Small Fragment Instrument Set	1

